ENGLISH

Fluorescein Sodium
Ophthalmic Strips USP 1mg

DESCRIPTION

Fluorescein sodium ophthalmic strips have an overall length of 60mm. The holding paper is 7mm in width. The impregnated portion is sandwiched between the holding paper and has an exposed portion of 17.5 mm by 5mm. The end of the tip rounded with a radius of 2.5 mm.

The absorbent paper holding the dye is made of Whatman # 1 or its equivalent. The holding paper is white in color, non-shredding and sufficiently stiff to offer a convenient hold.

Each strip is individually packed and sealed in a non-coated medical grade paper and film pouch to render a sterile product. The GSM of medical grade paper is not less than 60 gsm.

INDICATIONS

It is intended for use when fitting contact lenses.

Each strip is impregnated with 1 mg of fluorescein sodium. The strip is made of an absorbent paper, which is impregnated with the dye. This portion has a smoothly rounded tip that offers comfort during use. The absorbent portion is sandwiched between stiff, non-absorbent holding papers. The holding paper offers rigidity to the strip, making it easy and convenient to use.

CONTRAINDICATIONS

Fluorescein sodium ophthalmic strips are not meant for use with soft contact lenses as these will stain the lenses.

WARNING

- Keep out of reach of children.
- For external use only.
- Discontinue use in case of persistent irritation or burning sensation.
- Do not use if the pouch is torn, damaged or shows traces of leakage.
- For professional use only.

STORAGE

- Store at 10° - 30°
- Keep dry; do not store in moist environment

DIRECTIONS

Handle the exposed strip by grasping the white holding paper. Moisten the impregnated area of the strip with sterile normal saline or tear fluid from the lower fornix. While the patient looks up, stroke the moistened tip along the bulbar conjunctiva. The patient should blink several times to obtain the best results.

HOW SUPPLIED

In boxes of 50, 100 and 300 sterile strips.

NDC 83851-100-05 (Box of 50)
NDC 83851-100-10 (Box of 100)
NDC 83851-100-30 (Box of 300)

DISCLAIMER OF LIABILITY/RETURNS

The firm is liable only for potential manufacturing defect. Return the damaged container in its original packing identified by the lot number, your references and reason for return.

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	Manufacturer
	Date and Country of Manufacture
	Use by date
	Consult instructions for use
	Do not resterilize
	Do not re-use
	Keep away from sunlight
	Do not use if package is damaged and consult Instructions for use
	Keep dry
	Temperature Limit
	Single Sterile Barrier System, Sterilized using Ethylene Oxide
	Single Sterile Barrier System with protective packaging outside, Sterilized using Ethylene Oxide
	Batch Code
	Medical Device
	Keep out of reach of Children

**Manufactured by:**
Madhu Instruments Pvt. Ltd.
A-260, Okhla Industrial Area, Phase-1,
New Delhi-110 020 India
D&B D-U-N-S Number: 91-582-7852
Mfg. Lic. No. S(0116)/MB/18

**Imported by**
Vistamerica USA
20 Perkins Dr, Prescott, AZ 86301

**distributed by**
Made in India

Ref. No.: VISA4/IFU/01, Rev.: 01, Rev. Date: 07.02.2025