

PUREWAY

AMALGAM RECYCLING SYSTEMS



THE EASIEST AND MOST AFFORDABLE WAY TO BE IN COMPLIANCE

IMPORTANT! Amalgam waste CANNOT be discarded in regular trash or in bio-hazard containers with other red bag and sharps waste. Anything that contains amalgam must be stored in a specifically-designated container for proper recycling.

ADA

Stay compliant with local regulations and ADA Best Management Practices by using PureWay Amalgam Recycling Systems to properly dispose of your amalgam waste!

Unlike other amalgam disposal systems on the market, all PureWay systems come with a pre-paid UPS return shipping label, live email tracking updates and automatic electronic recycling certificate.

BENEFITS

- Available in 3 convenient sizes: 1.25 gallon, 2.5 gallon or 5 gallon
- Average cost savings of 20-30% over competing systems
- Meets ADA standard and complies with ANSI 109 requirements
- Approved to properly store and recycle scrap amalgam, chairside traps, disposable amalgam capsules, and extracted teeth containing amalgam fillings
- Includes pre-paid UPS return shipping label — no need to call and request a label
- Live email tracking notifications and electronic recycling certificates sent automatically



PUREWAY AMALGAM RECYCLING SYSTEMS

- 1.25 Gallon System
- 2.5 Gallon System
- 5 Gallon System



ORDER TODAY



(727) 735.1895 or info@coaynemed.com

AMALGAM RECYCLING SYSTEMS

WASTE AMALGAM BUCKETS/CONTAINERS

1.25 GALLON

- 1-3 operatories/chairs
- Will usually last about 6 to 9 months before full

2.5 GALLON

- 4-5 operatories/chairs
- Will usually last about 6 to 9 months before full

5 GALLON

- 6 or more operatories/chairs
- Will usually last about 9 months to a year



AMALGAM WASTE BUCKETS/CONTAINERS NEED TO HAVE ALL OF THESE ITEMS PLACED IN IT

- AMALGAM CAPTURE DEVICE (CHAIRSIDE TRAP) - The entire trap goes in the bucket. They do not need to pick through and find small pieces of amalgam to place in the bucket. I have been asked that question a lot in the past from dental offices.
- CONTACT AMALGAM - Any amalgam waste that has been in contact with the patient at any time during the dental service
- EMPTY AMALGAM CAPSULES - Empty capsules left over after mixing precapsulated dental amalgam
- NON-CONTACT SCRAP AMALGAM - Excess mix leftover at end of dental procedure

You will often hear from dental offices that they do not use amalgam anymore when they perform dental procedures. Explain to them they still need to have a waste amalgam bucket in their office because they will occasionally be removing amalgam fillings from patients.



ORDER TODAY

CoyneMed 
Medical & Dental Equipment
Sales, Service & Repair

(727) 735.1895 or info@coynemed.com