


Handpiece Maintenance Guide

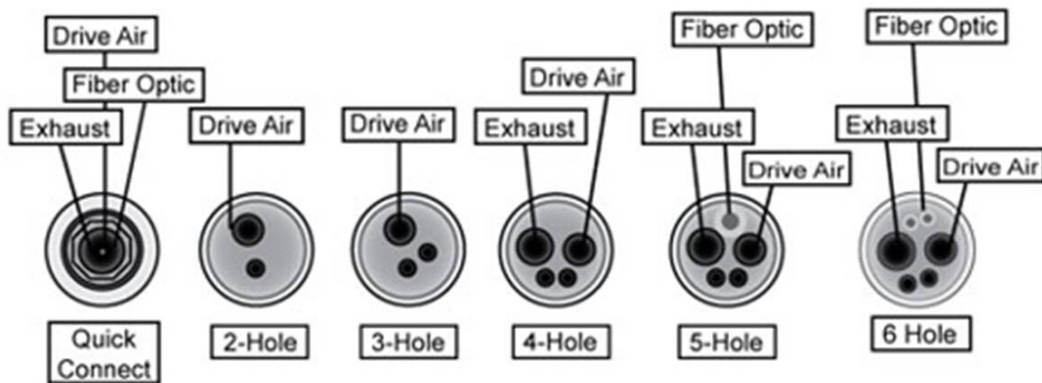

PRE-STERILIZATION

1. Clean external surface of handpiece by wiping with a damp toothbrush or paper towel
 - a. **DO NOT** run under water. Wipe dry after cleaning
2. Insert bur into chuck
3. Spray handpiece cleaner/lubricant into drive air line for 2-3 seconds
4. Rotate the bur back and forth between your fingers to loosen any debris inside the handpiece head
5. Operate handpiece over a paper towel to purge used cleaner/lubricant and debris
6. Repeat steps 3 - 5 until expelled fluid is completely clear
7. Operate handpiece until all cleaner/lubricant has been expelled
8. Wipe external surface with a dry towel to remove expelled fluid
9. Sterilize handpiece according to manufacturer's instructions

POST-STERILIZATION

1. Allow sterilized handpiece to cool gradually to room temperature
2. Verify that the bur is securely in the chuck
3. Apply 2 drops of lubricant into drive air line, or spray lubricant into drive air line for 2-3 seconds
4. Operate handpiece for 30 seconds over a paper towel to expel excess lubricant from the handpiece
5. Wipe external surfaces with a dry towel
6. Handpiece is ready for use

PUSH BUTTON CHUCK MAINTENANCE

Twice a week

1. Remove bur from handpiece
2. Place 2 drops of handpiece oil into the chuck
3. Insert bur into the handpiece
4. Run handpiece over a paper towel at 35 psi to expel excess lubricant
 - a. If your push button chuck is stiff, after step 2 hold the chuck open and work the bur in and out to flush out debris

EXTERNAL MAINTENANCE

- Never use any type of cold sterile, alcohol, or disinfectant on your handpiece
- Do NOT submerge handpiece in running water or ultra sonic cleaner

Why CoyneMed?

We service and sell ALL dental operatory equipment, including, chairs, sterilizers, compressors, vacuums, and **HANDPIECES!**

CoyneMed is a local, family owned business offering professional, quick, and friendly service at reasonable prices.

- * **FREE** estimates
- * **NO CALL OUT** charges
- * **FREE** Loaner Sterilizer
- * **FREE** training for your employees on proper maintenance which extends the life of your equipment and reduces your repairs

We hope you will **call us FIRST** each time you need equipment sales, service or repair!



(727) 735.1895

www.CoyneMed.com