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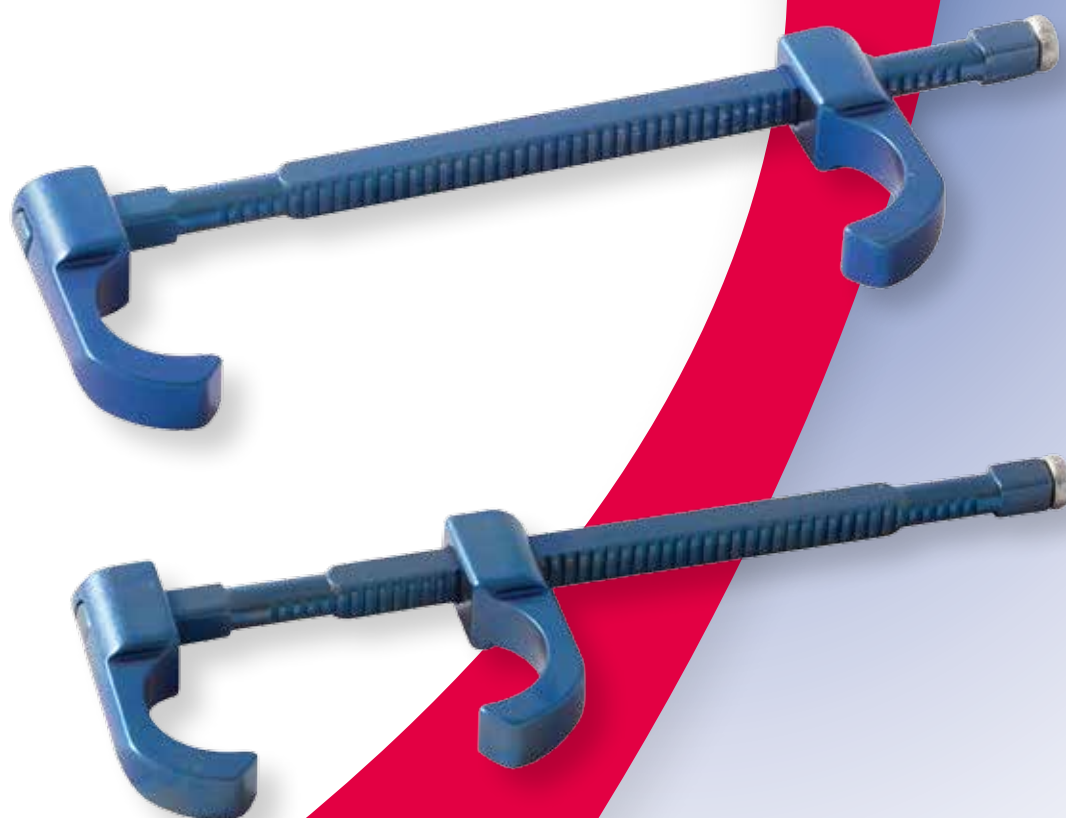
Sternal closure device

Description

Product overview

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The Sternal Closure Device is comprised of two hooks connected by a ratchet rod. One of the hooks is fixed at one end, while the second hook can be moved along the ratchet rod. A sternal closure applying forceps is used to bring the movable hook towards the fixed hook along the ratchet rod.

Compression sternale progressive et parfaite.

Perfect and progressive sternal compression.

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Preventive and curative treatment

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- *Alternative to steel wire for the sternal closure.*
- *Preventive treatment: at risk, elderly, obese or diabetic patients.*
- *Curative treatment: redo surgery, disunion, sternal pseudarthrosis, sternite.*
- *Easy to apply and adapted to the sternal shape.*
- *Biocompatible titanium alloy which enables NMR techniques.*

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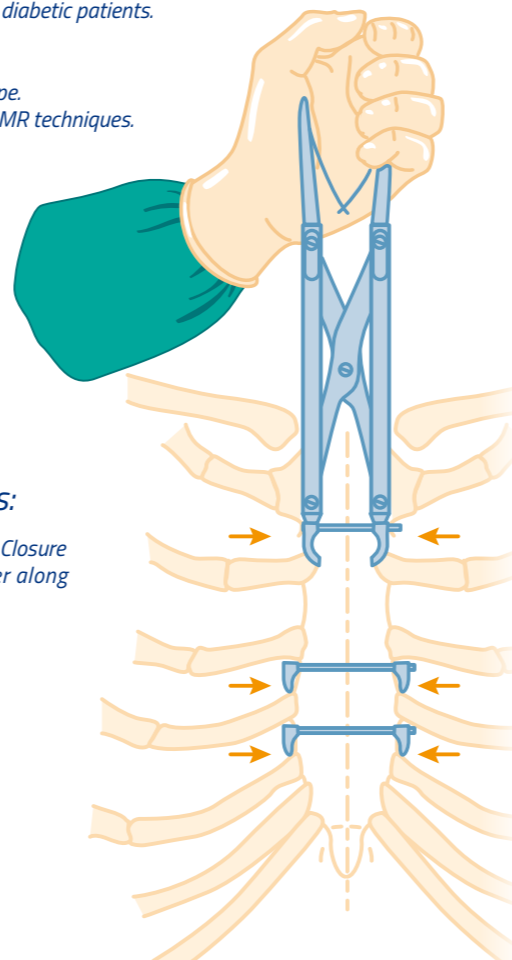
Sternal closure applying forceps:

- *Two jaws to grip the two hooks of the Sternal Closure Device and bring them towards one another along the ratched rod.*

Sternal closure cutting forceps:

- *To cut off the excess ratchet rod at the base of the movable hook and to cut the middle of the ratchet rod in two if needed.*

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Only 2 or 3 sternal closure devices are required:

- *1 is applied to the manubrium and 1 or 2 others to the body of the sternum.*
- *Positioned perpendicular to the major axis of the sternum.*
- *The two hooks are applied laterally to the sternum in the two matching intercostal spaces.*

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- *Use the applying forceps to bring the two hooks towards one another such that the distance between the two hooks allows positioning of the closure device rod against the front surface of the sternum and the hook in the prepared parasternal intercostal space.*

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- *Check the positioning and, if necessary, use the applying forceps until the two sides of the sternum are compressed.*
- *Use the cutting forceps to cut off the excess rod at the base of the movable hook.*

	Description <i>Description</i>	Références <i>References</i>
	<p>Agrafe sternale à crémaillière <i>Sternal closure device</i></p>	<p>ASC010</p>
	<p>Pince porte-agrafe <i>Applying forceps</i></p>	<p>PPA010</p>
	<p>Pince coupe-agrafe <i>Cutting forceps</i></p>	<p>PCA010</p>

Ce produit est livré non stérile
This product is delivered non sterile