



19/02/13

Internal use only!

News letter

ALKANTIS, A World's First!

ALKANTIS Surgery Indications Tutorial

Objectives with this tutorial is to show details about the use of ALKANTIS for different surgery specialities:

Orthopedics, General surgery, Plastic, ENT, Gynecology, Obstetric, Electrophysiology, Cardiac, Vascular, Wound Care,...

- This information is not theoretical but its a return of experience we have collected through our distributors.
- The first page displays the unique and exceptional performance of the product but more important the ALKANTIS concept.



«ALKANTIS opens the way to the hospital to make savings»

Guidelines:



- To prepare a call you need both the ALKANTIS product brochure and this Tutorial.
- The Tutorial displays several indications per speciality.
- Do not hesitate to show the attached cases to surgeons

This newsletter should allow you to accurately describe where and how ALKANTIS is used.








[Ice Sterile]

A World's First!

Introducing in the O.R. a Drug-free Patient's Pain Control by using a Sterile and Dry Cold Therapy

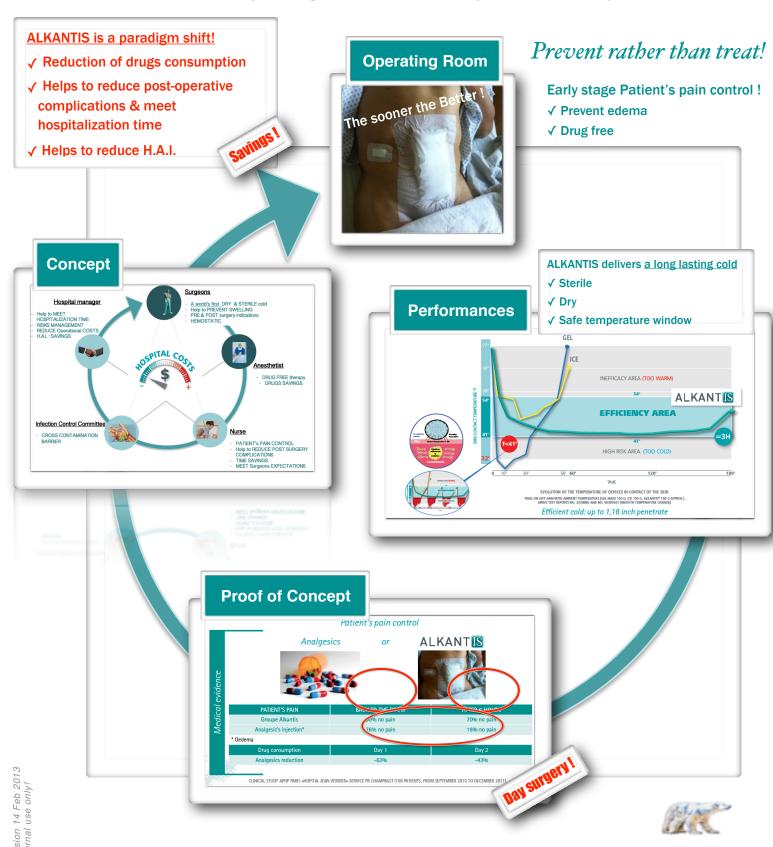

Introducing in the O.R. A Drug-Free Patient's Pain control by using a Sterile and Dry Cold Therapy

A few indications for Orthopedic surgery:




Hip / Knee / Shoulder / Elbow / Hand / Foot / Ankle / Spine / Arthroscopy / Arthroplasty

- Why Alkantis?
- Helps to reduce Pain, inflammation, edema, and hematoma
- Helps to prevent postoperative complications
- Considered a cross contamination barrier for reducing the risk of Hospital- Acquired Infection (H.A.I.)
- Delivers long lasting cold up to 3 hours, between 5°C (41°F) and 12°C (54° F)
- Can be placed directly on the surgical site in the operating
- No burns & no re-vascularization issues



ALKANTIS helps to prevent swelling, and

Analgesics side effect could impact patient's metabolism such as "bowel movements ALKANTIS, is a drug free! Patients could leave the hospital on time!

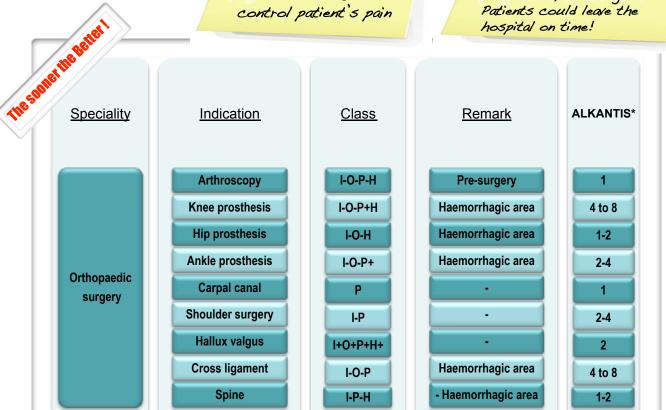




Introducing in the O.R. A Drug-Free Patient's Pain control by using a Sterile and Dry Cold Therapy


A few indications for General surgery:

Hernia / Laparoscopy / Laparotomy / Appendectomy / Gastroplasty / Subcostal incision



- Why Alkantis?
- Helps to reduce Pain, inflammation, edema, and hematoma
- Helps to prevent postoperative complications
- Considered a cross contamination barrier for reducing the risk of Hospital- Acquired Infection (H.A.I.)
- Delivers long lasting cold up to 3 hours, between 5°C (41°F) and 12°C (54° F)
- Can be placed directly on the surgical site in the operating room
- No burns & no re-vascularization issues

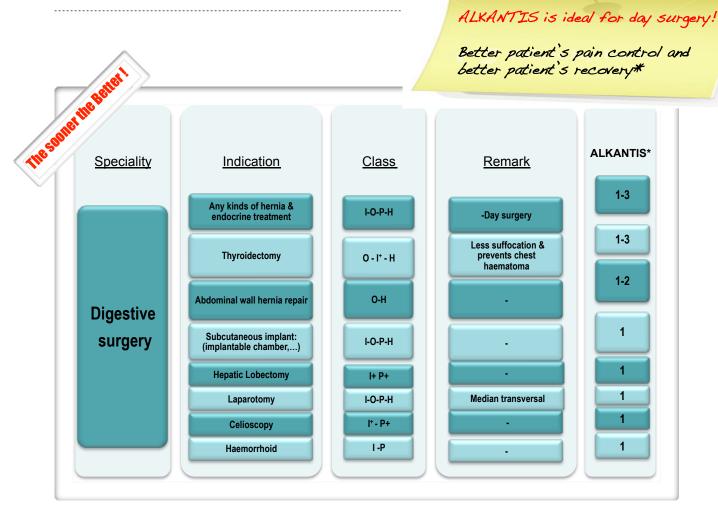

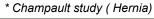





Introducing in the O.R. A Drug-Free Patient's Pain control by using a Sterile and Dry Cold Therapy

A few indications for Plastic surgery, Reconstructive






Blepharoplasty / Rhytidectomy / Rhinoplasty / Liposuction / Breast / Abdominoplasty

- Why Alkantis?
- Helps to reduce Pain, inflammation, edema, and hematoma
- Helps to prevent postoperative complications
- Considered a cross contamination barrier for reducing the risk of Hospital- Acquired Infection (H.A.I.)
- Delivers long lasting cold up to 3 hours, between 5°C (41°F) and 12°C (54° F)
- Can be placed directly on the surgical site in the operating
- No burns & no re-vascularization issues

swelling, which means less tensions on incisions! Better scars recovery and less visible ! Remark **ALKANTIS*** 1-2 1 2-3 Reduce redness 1-2

ALKANTIS help to prevent

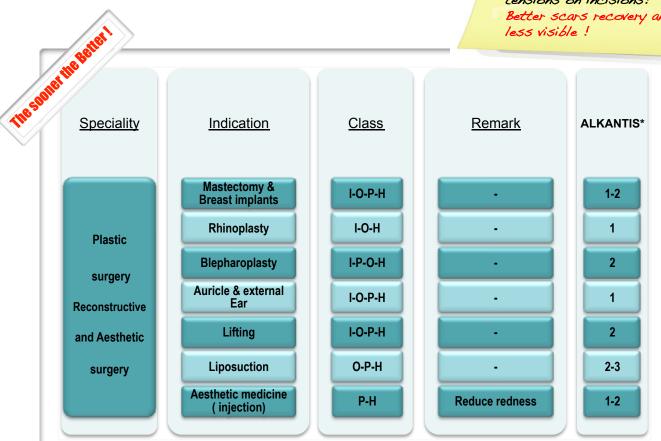



Introducing in the O.R. A Drug-Free Patient's Pain control by using a Sterile and Dry Cold Therapy

A few indications for Gynecology, Obstetric & Urology:




Caesarian / Episiotomy / Hysterectomy / Prostatectomy / Cystectomy / Nephrotectomy

- Why Alkantis?
- Helps to reduce Pain, inflammation, edema, and hematoma
- Helps to prevent postoperative complications
- Considered a cross contamination barrier for reducing the risk of Hospital- Acquired Infection (H.A.I.)
- Delivers long lasting cold up to 3 hours, between 5°C (41°F) and 12°C (54° F)
- Can be placed directly on the surgical site in the operating room
- No burns & no re-vascularization issues



Lady could seat 24 hours after an episiotomy!!

ALKANTIS is a drug free therapy, which means passing less narcotics through breast milk!!

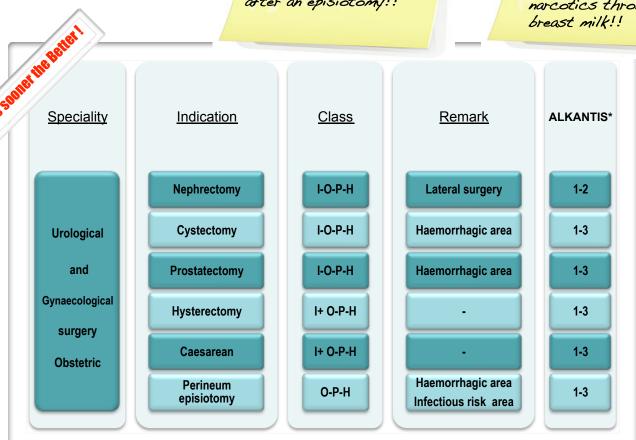





Introducing in the O.R. A Drug-Free Patient's Pain control by using a Sterile and Dry Cold Therapy

A few indications for Electrophysiology and Cardiac:





Pacemaker / Defibrillator / Thoracotomy / Sternotomy / Catheter

- Why Alkantis?
- Helps to reduce Pain, inflammation, edema, and hematoma
- Helps to prevent postoperative complications
- Considered a cross contamination barrier for reducing the risk of Hospital- Acquired Infection (H.A.I.)
- Delivers long lasting cold up to 3 hours, between 5°C (41°F) and 12°C (54° F)
- Can be placed directly on the surgical site in the operating
- No burns & no re-vascularization issues

ALKANTIS could be used in PRE surgery to prevent bleeding. Helpful for patient under "anticoaquiant" for example!

Analgesics side effect could impact patient's metabolism such as "bowel movements" ALKANTIS, is a drug free! Patients could leave the hospital on time!

The souner the Relter I **Speciality ALKANTIS*** Indication Class Remark I+ - H+ - P+ 1 to 2 **Thoracotomy** Haemorrhagic area Sternotony P-H Haemorrhagic area 1 to 3 Cardiac **Pacemaker Pocket Haematoma** I-O-P-H 1 Electrophysiology surgery **Acces: Central** I-O-P Aneurism area 1 Catheter **Stent insertion** I-O-P Aneurism area 1 (PTCA)



Introducing in the O.R. A Drug-Free Patient's Pain control by using a Sterile and Dry Cold Therapy


A few indications for vascular surgery:



Carotid / Variscose vein / Phlebotomy/ Saphenectomy / Vena cava / Coronagraph / Stent / Dilatation / Aorta & Arteries

- Why Alkantis?
- Helps to reduce Pain, inflammation, edema, and hematoma
- Helps to prevent postoperative complications
- Considered a cross contamination barrier for reducing the risk of Hospital- Acquired Infection (H.A.I.)
- Delivers long lasting cold up to 3 hours, between 5°C (41°F) and 12°C (54° F)
- Can be placed directly on the surgical site in the operating room
- No burns & no re-vascularization issues



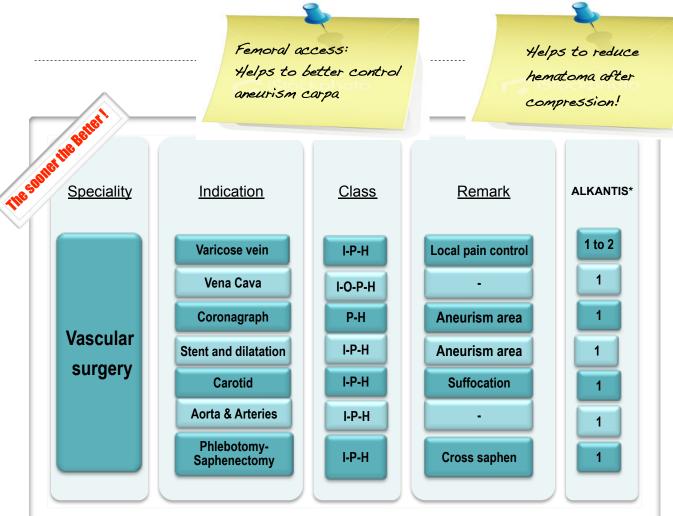





Introducing in the O.R. A Drug-Free Patient's Pain control by using a Sterile and Dry Cold Therapy

A few indications for Wound Care:



Debridement / Skin excision / Skin repair / Skin evacuation / Skin autograph / Dermolipectomy / Lifting

- Why Alkantis?
- Helps to reduce Pain, inflammation, edema, and hematoma
- Helps to prevent postoperative complications
- Considered a cross contamination barrier for reducing the risk of Hospital- Acquired Infection (H.A.I.)
- Delivers long lasting cold up to 3 hours, between 5°C (41°F) and 12°C (54° F)
- Can be placed directly on the surgical site in the operating room

ALKANTIS induces a deep

- No burns & no re-vascularization issues



local anesthesia. No need of drugs! Savings! and safe comfort for the patient! **ALKANTIS*** Speciality Indication Class Remark **Skin Excision** I-O-P-H Pain 1 I-O-P+H **Debridement Debridement** 1 - 2 Skin repair I-O-P-H Pain 1-2 Wound care Skin evacuation I-O-P-H **Debridement** 1-2 Skin infectious I-O-P-H Pain 1 excision Skin autograft I-O-P-H Pain I-O-P-H **Dermolipectomy Pain** 1-2





Congress Participation:

AORN St Diego, March 4-6, 2013 Booth N° 3945

HAS Orlando, March
13-16,2013
Booth N° 408

AAOS Chicago, March 20-22, 2013 Booth Contact US

EHS, Gdansk Poland May 12-15, 2013 Booth Contact US

MESH, Paris France
June 14, 2013
Booth Contact US

ACS Washington, October 6-10, 2013 Booth Contact US













INTERNATIONAL CONTACT:

Patrick Caceres

+33 (0) 620 747 302

p.caceres@alkantis.fr

Jean Claude Riquier

+33(0) 677 079 747

Jean-claude.riquier@innovcare.net



