

Calming Comfort™

WEIGHTED HOT/COLD NECK WRAP



Please read all instructions and warnings before using the ThermoComfort™ Weighted Hot/Cold Neck Wrap. Keep the instruction sheet as it contains important information for future use.



ThermoComfort™
Weighted Hot/Cold
Neck Wrap



ThermoComfort™
Storage Pouch

ThermoComfort™ Weighted Hot/Cold Neck Wrap contains the following Ingredients:

- Clay Beads
- Flax Seeds
- Lavender
- Mint Leaves

CARE INSTRUCTIONS

- Spot clean only.
- Do not place in washing machine or dryer.
- Do not iron.
- Do not bleach.
- Do not wet product.

USING YOUR THERMACOMFORT™ WEIGHTED HOT/COLD NECK WRAP

Heating Instructions:

- Place the ThermoComfort™ Weighted Hot/Cold Neck Wrap into the Storage Pouch and insert into the center of the microwave.
- Microwave for 1 minute and 30 seconds (1:30). For Best Results, Flip the unit over after 0:45 and continue heating for remaining time. Heating time based may vary by Wattage of microwave. Heating time is based on 1100W microwave, for higher powered microwaves reduce initial heating time to 1 minute.
- Carefully touch the wrap to check the temperature. If not warm enough, heat in increments of 30 seconds – **Never exceed a total heating time of 3 minutes.**
- Use caution when removing from microwave as product may be hot. Remove wrap from pouch. Before use, carefully check the temperature before applying the wrap. If it is too hot, allow the wrap to cool.
- Your ThermoComfort™ Weighted Hot/Cold Neck Wrap is now ready to wear!



Cooling Instructions:

- Place the ThermoComfort™ Weighted Hot/Cold Neck Wrap into the storage pouch.
- Chill the ThermoComfort™ Weighted Hot/Cold Neck Wrap in the freezer for at least one hour. Adjust freeze time if additional cooling is needed.
- Remove the ThermoComfort™ Weighted Hot/Cold Neck Wrap from the freezer and pouch. Carefully touch to test.
- If too cold, allow the product to warm up before use.
- Your ThermoComfort™ Weighted Hot/Cold Neck Wrap is now ready to wear!



Use:

Once the wrap is heated or chilled, it is ready to apply. Place the unit over your shoulders and raise the collar around the back of your neck. Sit back and relax as the ThermoComfort™ Weighted Neck Wrap soothes you. The Neck Wrap stays in place while walking, standing, or sitting. **Helpful Hint:** it is best to keep good posture while using the product.

⚠ WARNING

• DO NOT EXCEED TOTAL HEATING TIME OF 3 MINUTES.

- Do not leave ThermoComfort™ Weighted Hot/Cold Neck Wrap unattended while heating.

OVERHEATING MAY CAUSE COVER TO IGNITE.

- Allow ThermoComfort™ Weighted Hot/Cold Neck Wrap to return to room temperature before reheating.
- HOT, take care when removing from microwave oven.
- Microwaved item may actually be much HOTTER INSIDE than indicated by initial touch-so HANDLE WITH CARE.
- Discard if found worn or damaged.
- This is not a children's product. Keep out of children's reach.
- The heating of this item should only be carried out by an adult.
- Due to variations in microwaves, it may be necessary to adjust temperature and time settings. These instructions are based on an 1100W microwave, if you are using a higher powered microwave, then you will need to reduce the heating time.

- Do not use this product on persons with sensitive skin, decreased skin sensitivity, nerve damage, circulatory problems, damaged or broken skin; or on persons who are unable to remove this product such as infants, children and some elderly persons.
- If any medical condition exists such as diabetes, or if you are pregnant, consult a physician before use.

- Do not use on any other parts of the body than otherwise indicated.
- Do not ingest contents.
- Heat in Microwave only - Do not use convection function on your microwave.
- Do not use in microwave where the original turntable has been damaged or removed.

KEEP THIS INSTRUCTION MANUAL FOR REFERENCE AS IT CONTAINS IMPORTANT INFORMATION.

Allstar Products Group is not responsible for abuse or misuse of The Calming Comfort™ Weighted Hot/Cold Neck Wrap.

© 2019 APG
ThermoComfort is a trademark of APG
Made in China