

COZYZONE ANIMALS INSTRUCTIONS FOR USE



The Cozy Zone Animal Leg Supports have been designed to support the body in a straight and comfortable position and can be used during the night or day!

CozyZone Animals should only be used for the purposes for which they have been designed. If the leg support is altered in any way or used in a way in which it has not been designed, Blackstone Innovations cannot take responsibility for any damages or injury caused.


Please note that altering the equipment or using it in a way for which it has not been designed will void the warranty.

Important Information.

These instructions must be kept with the product at all times.

This product is a Prescribed Medical Device.

You must read these Instructions for Use carefully before use.

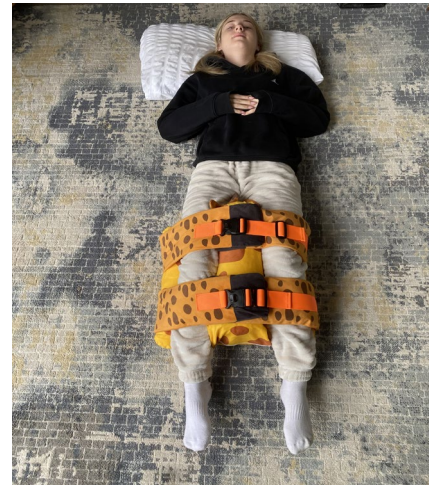

Blackstone Innovations



Manufactured by Blackstone Innovations LTD

BICZUF10001v1

1

INSTRUCTIONS FOR USE CONTENTS



- Important Safety Information
 - Label Information
- Care and Cleaning Instructions
- Maintenance and Inspection
 - Warranty Information
 - Contact Page



Blackstone Innovations



Manufactured by Blackstone Innovations LTD

IMPORTANT SAFETY INFORMATION



CozyZone Animals should only be used following advice/guidance from a Qualified Health Professional. If you have any doubts or concerns regarding this product please contact your Local Healthcare Professional or Blackstone Innovations LTD.



Do not use the leg support system if you become concerned over any of the following: Breathing, Reflux, Temperature, Circulation or Discomfort. Please contact your Qualified Healthcare Professional or Blackstone Innovations LTD immediately.



CozyZone Animals conform to the EU Medical Device Regulation. They have not been tested in conjunction with other manufacturer's products. Do not add any accessories other than those approved by Blackstone Innovations LTD.



Any serious incident that has occurred in relation to CozyZone should be reported to the Blackstone Innovations LTD and the competent authority of the country in which the user and/or patient is established immediately.



It is essential that CozyZone is reviewed and thoroughly inspected at least annually by a Qualified Healthcare Professional for it's continued suitability to the user and the user's needs.



Blackstone Innovations



Manufactured by Blackstone Innovations LTD


LABEL INFORMATION



Blackstone Innovations LTD declares that the product conforms to the UK Medical Device Regulations 2002



Unique Device Identifier (UDI)

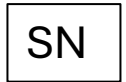

Serial Number



Manufacturer's Address and Date of Manufacture.

Blackstone Innovations LTD, Ground Floor, Sterling House, Outram's Wharf, Long Eaton, DE21 5EL



Batch Code



Blackstone Innovations



Manufactured by Blackstone Innovations LTD



CARE AND CLEANING INSTRUCTIONS

Description	Care and Cleaning Instructions
Printed Cover	The Printed Animal Covers can be machine washed at 75° and Tumble Dried. Please connect all securing straps before washing/drying and turn animal cover inside out. Blackstone LTD can take no responsibility for damage caused by incorrect washing/tumble drying.
Inner Cushions	Remove and Replace cushions through zipped edges of Cozy Zone animals. Wipe Clean with a Mild Detergent. DO NOT remove inner core. DO NOT use Bleach or Biological Cleaners. DO NOT Dry Clean.



MAINTENANCE AND INSPECTION



Any CozyZone Leg Supports returned to Equipment Services should be inspected, recorded, repaired, reissued in line with the national competent authority guidelines for products in their charge. Check the competent body website for the latest bulletins.

General Maintenance and Inspection

Fabrics and PU:

Is any fabric and PU in good, clean condition with no rips, wear or gaping of the cushions?

Buckles and Securing Straps:

Are any Buckles/Straps in good condition and not showing any signs of wear? Check straps for fraying and missing or broken buckles. Ensure all webbing end clips are in place.

Zips:

Check all zips for waer and tear. Are they complete and in good condition and serviceable?

Safety:

Is the product complete, safe and fit for use?

Products should be thoroughly decontaminated in line with national guidelines prior to inspection, storage and re-issue.

STORAGE: DO NOT store CozyZone or any of it's accessories outside, in a damp environment or on hot surfaces.



Blackstone Innovations



Manufactured by Blackstone Innovations LTD

WARRANTY INFORMATION

The Instruction for Use of CozyZone Animal Leg Supports should be followed carefully.

It is impossible to eliminate all risks inherently associated with the use of CozyZone Animal Leg Supports.

Product failure or other unintended consequences may result because of such factors as improper storage, use or maintenance, unauthorised and inappropriate alteration or repair or improper or violent treatment, excessive strain, use of improper replacement parts, or because of normal wear and tear, all of which are beyond the control of the Manufacturer and Seller. Buyer and User assume all such risks.

Manufacturer and Seller warrant that this product will be free from defects in material and workmanship for a period of 12 months from the date of purchase and is reasonably fit for the purposes stated in the Instruction for Use, subject to the inherent risks under normal use conditions.

This Warranty does not extend to the use of the product contrary to the Instruction for Use, or under abnormal conditions, or under conditions not reasonably foreseeable to or beyond the control of the Manufacturer or Seller. Buyer and User assume the risk of any such use.

MANUFACTURER MAKES NO WARRANTIES OF SATISFACTORY QUALITY OR OF FITNESS FOR A PARTICULAR PURPOSE NOR ANY OTHER EXPRESS OR IMPLIED WARRANTY EXCEPT AS STATED ABOVE.

In no event shall Manufacturer or Seller be liable for any incidental, consequential or special damages resulting from the use of this product.

THE EXCLUSIVE REMEDY OF THE USER OR BUYER AND THE EXCLUSIVE LIABILITY OF MANUFACTURER AND SELLER FOR ANY AND ALL CLAIMS, LOSSES, INJURIES OR DAMAGES (INCLUDING CLAIMS BASED ON BREACH OF WARRANTY, CONTRACT, NEGLIGENCE, TORT STRICT LIABILITY OR OTHERWISE) RESULTING FROM THE USE OF THIS PRODUCT, SHALL BE THE RETURN OF THE PURCHASE PRICE OF THE PRODUCT OR THE REPLACEMENT OF THE PRODUCT.

Manufacturer and Seller offer this product, and Buyer and User accept it, subject to the foregoing conditions of sale and limitations of warranty and liability, which may not be modified except by written agreement signed by a duly authorised representative of the Manufacturer.



Blackstone Innovations



Manufactured by Blackstone Innovations LTD



CONTACT PAGE

If you have any concerns over the safe use of your product.
If you would like to order replacement parts.
Or if you just have a question.

Please do contact us:

Email: dominic@blackstoneinnovations.co.uk

Telephone: 07444 238908

Website: www.blackstoneinnovations.co.uk



Blackstone Innovations



CozyTherm

Manufactured by Blackstone Innovations LTD

BICZUFI0001v1

8