

Monitor by Laerdal Resources

How-To-Videos

Monitor by Laerdal - <https://laerdal.com/support/how-to-videos/laerdal-monitor-how-to-videos/>

TruMonitor - <https://laerdal.com/support/how-to-videos/trumonitor-how-to-videos/>

TruVent - <https://laerdal.com/support/how-to-videos/truvent-how-to-videos/>

Sell sheet

<https://bam.laerdal.com/laerdal#!asset/jv5h3v74xtt47g9rqjnm975>

<https://laerdal.com/us/doc/4545/Monitor-by-Laerdal#/Downloads>

Website

<https://laerdal.com/products/simulation-training/manage-assess-debrief/monitor-by-laerdal/>

Frequently Asked Questions

What monitor interface skins are currently available?

- **Monitor by Laerdal** – Generic | Generic 2
- **Zoll** – Zoll X | Zoll R | Zoll R+ |
- **Mindray** - BeneHeart D3 |
- **Physio-Control** - LifePak 15
- **Phillips** - Tempus Pro | IntelliVue MP30

Can the instructor's iPad control two monitors at the same time (one defib and one vent)?

- No, but the instructor can toggle back and forth between the TruMonitor and TruVent easily

How do you present labs on the screen?

- Select the *Investigation* tab and then choose your labs – make sure to enable test in the upper right corner

How is LLEAP for SimPad involved?

- Only one Bluetooth connection can be used at a time. You cannot use LLEAP for SimPad (with RAAST/RAS) and Monitor by Laerdal concurrently. Future integration is not currently planned, but if market demand is high, this can be revisited.

How is SimCapture involved?

- SimCapture and Monitor by Laerdal integration is being developed. We do not have timelines or anything more sharable at this moment.

Can we order without the iPads?


- This is currently planned to release without iPads if customer choose to purchase themselves. On track for December 2022.

What happens at the end of subscription?

- The email associated with the TruVent/TruMonitor account will get an email notification

What if the customer wants both TruVent and TruMonitor?

- Customer can purchase the additional software (separate line item)

Company	Laerdal	Laerdal	iSimulate	Skill Qube	Penguin System
					
Product	Monitor by Laerdal	SimStart	ALSi/Realiti360	Skill Qube	EmerSim
Platform					
IOS	Yes	No	Yes	Yes	Yes
Android	Yes	No	No	No	Yes
Windows	No	Yes	No	No	Yes
Remote learning	Yes, unlimited	No	Yes, up to 6	No	Yes
Remote video screen sharing between instructor and student	Yes	No	No	No	Yes
Integration with fully functional ventilator simulation solution	Yes	No	No	No	No
Integration with video debrief/capture feedback system	Yes	No	No	No	No
Capture/ Track data for student feedback	In development	Yes	Yes	Yes	No
Integration of events in manikin software	In development	No	No	No	No
Automatic Realtime QCPR feedback	Yes	Yes	No	No	No
Prebuild scenario's	Yes	Yes	No	No	Yes
Possibility to create own scenario's	Yes	Yes	Yes	No	No
Ability to record your own sounds and upload these	Yes	Yes	Yes	No	No
Import/ Export scenario's	Yes	No	Yes	No	No
Access camera roll to upload own images	Yes	Yes	Yes	No	No
Portable solution	Yes	No	Yes	No	Yes
Connectivity with Resusci Anne QCPR	Yes	No	No	No	No
Patient Monitor Interface	Yes	Yes	Yes	Yes	Yes
Monitor-defibrillator Interface	Yes	Yes	Yes	Yes	Yes
AED interface generic	Yes	Yes	Yes	Yes	Yes
Brand interface	Yes	No	Yes	Yes	No
Pacing	Yes	Yes	Yes	Yes	Yes
Synchronized cardioversion	Yes	Yes	Yes	Yes	Yes
CPR simulation curve	Yes	Yes	Yes	?	?
PEA simulation curve	Yes	Yes	Yes	?	?
Disarm button	Yes	Yes	Yes	?	?