



# Single-use Surgical Mask

SOFT & EASY TO BREATHE

Non-sterile

**BYD CARE** 




**Product Model**

**FE2311**



## Product Introduction

BYD Single-use Surgical Mask consists of three layers of nonwoven material:

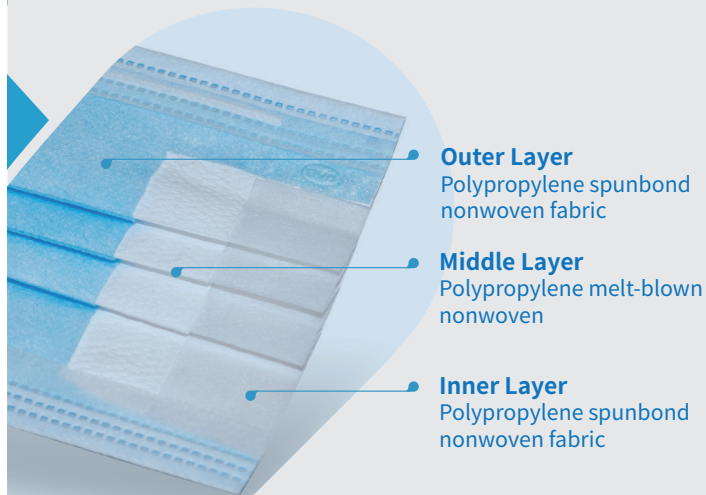

**Outer Layer**  
Polypropylene spunbond nonwoven fabric

**Middle Layer**  
Polypropylene melt-blown nonwoven

**Inner Layer**  
Polypropylene spunbond nonwoven fabric

### MATERIALS

Inner layer	Polypropylene spunbond nonwoven fabric
Middle layer	Polypropylene melt-blown nonwoven
Outer layer	Polypropylene spunbond nonwoven fabric
Ear loops	Polyester/nylon spandex blend
Nose clip	Metal wire with plastic covering

This device is not made with natural rubber latex. Non-sterile product.  
One-time use only.

## Wearing directions



1. Hold the mask by the ear loops with the colored side facing outwards and the nose bridge upwards, and place an ear loop around each ear.



2. Bend the nose clip to match the shape of the nose to prevent unfiltered air from entering.

# Single-use Surgical Mask




SOFT & EASY TO BREATHE (Non-sterile)

## Indications for Use

The Single-use Surgical Masks (Model: FE2311) are intended to be worn to protect both the patient and healthcare personnel from transfer of microorganisms, body fluids and particulate material. These face masks are intended for use in infection control practices to reduce the potential exposure to blood and body fluids.

This is a single use, disposable device(s), provided non-sterile.

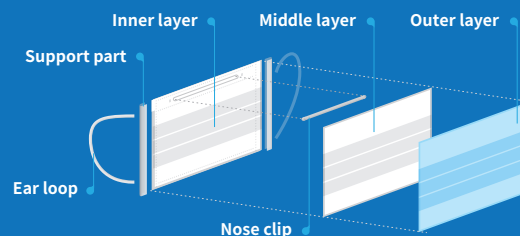

## Product Information

Name	Parameter
Product name	Single-use Surgical Mask
Materials	Polypropylene spunbond nonwoven fabric (for surgical use), polypropylene melt-blown nonwoven, metal wire with plastic covering nose clip, polyester/nylon spandex blend ear loops
Product model	FE2311
Size	175 mm×95 mm / 6.89in.×3.74in.
Expiration date	2 years after production
Packaging specification	10pcs/bag; 50pcs/box; 2000pcs/case
Storage	Store in a dry, clean environment with room temperature
BFE ≥ 98%	PFE ≥ 98%
Flammability: Class I	The masks are tested according to the ASTM F2100 and 16 CFR-1610 standard.

## WARNING

- 1 The mask does not eliminate the risk of contracting any disease or infection.
- 2 Failure to properly use and maintain this product could result in illness or even death.
- 3 Only for one-time use, please destroy and dispose according to regulations after use.
- 4 Please use within the validity period.
- 5 DO NOT use if package is damaged.
- 6 Please refer to the instructions before use.
- 7 For adult use only.

### Time use limitation

If the mask becomes damaged, soiled, or breathing becomes difficult, leave the contaminated area and replace and refit with a new mask.

### Important notice

To the extent permitted by law, BYD shall not be liable for any loss or damage including any loss of business, loss of profits, or for any indirect, special, incidental or consequential loss or damage arising from reliance upon any information herein provided by BYD. Nothing in this statement will be deemed to exclude or restrict BYD's liability for death or personal injury arising from its negligence.

All information presented in this document is based on the latest data available at the time of printing. BYD reserves the right to make changes at any time without prior notice.

ASTM F-2100 Level III EN 14683 Type IIR

Distributed by  
**Bigfoot Inc.**  
7378 W Atlantic Blvd #210  
Margate, FL 33063  
www.BigfootPPE.com  
**Made in China**

