



Single-use Surgical Mask

SOFT & EASY TO BREATHE

Non-sterile



BYD CARE 



Application

Worn by personnel to cover the mouth, nose and chin, the mask provides a physical barrier.



Product Introduction

BYD Single-use Surgical Mask consists of three layers of nonwoven material:



Outer Layer
Polypropylene spunbond nonwoven fabric

Middle Layer
Polypropylene melt-blown nonwoven

Inner Layer
Polypropylene spunbond nonwoven fabric

MATERIALS

Inner layer	Polypropylene spunbond nonwoven fabric
Middle layer	Polypropylene melt-blown nonwoven
Outer layer	Polypropylene spunbond nonwoven fabric
Ear loops	Polyester/nylon spandex blend
Nose clip	Metal wire with plastic covering

This device is not made with natural rubber latex.
Non-sterile product. One-time use only.

Wearing Instructions



1. Hold the mask by the ear loops with the colored side facing outwards and the nose bridge upwards, and place both ear loops around both ears.



2. Bend the nose clip to match the shape of the nose.



3. Pull the mask to your chin to produce a tight seal.

Single-use Surgical Mask


SOFT & EASY TO BREATHE

Non-sterile



Product Specification

- 1 Mask length: 175 mm(6.89in.)
- 2 Mask expansion width: 165 mm(6.49in.). Post-stack width is 95 mm(3.74in.) with three stacks in the middle. Each stack width is not less than 10 mm(0.39in.). The nose clip is located on the upper edge of the mask, and the outside of the mask is darker in color.
- 3 The position of the upper nose clip is not more than 16 mm(0.63in.).
- 4 Nose clip: length > 90 mm(3.54in.); width approximately 3 mm(0.12in.)
- 5 Ear loop: 170 mm(6.69in.) in length and 3.5 mm(0.14in.) in diameter, composed of polyester, spandex and other materials, welded on the inner layer not more than 10 mm(0.39in.) from the edge.

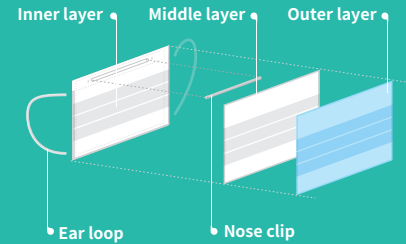

Parameter of BYD Single-use Surgical Mask

Name	Parameter
Product name	Single-use Surgical Mask
Material	Polypropylene spunbond nonwoven fabric, polypropylene melt-blown nonwoven, metal wire with plastic covering nose clip, polyester/nylon spandex blend ear loops
Product model	FE2311X (FLAT)
Size	175 mm×95 mm / 6.89in.×3.74in.
Expiration date	2 years after production
Packaging specification	10pcs/bag; 50pcs/box; 2000pcs/case
Storage	Store in a room with relative humidity level of no more than 80%, no corrosive gas and good ventilation to avoid high temperature.

WARNING

- 1 The mask is not to be used for antimicrobial protection, antiviral protection, or particulate filtration, and it does not eliminate or reduce the risk of contracting any disease or infection.
- 2 Failure to properly use and maintain this product could result in illness or even death.
- 3 Only for one-time use, please destroy and dispose after use.
- 4 Please use within two years of manufacture.
- 5 DO NOT use if package is damaged.
- 6 Please refer to the instructions before use.
- 7 For adults use only.
- 8 Use this product immediately after the package is opened. Avoid hand contact within the interior part of the mask. Do not wash.
- 9 This mask is distributed under FDA's Enforcement Policy for Face Masks and Respirators During the Coronavirus Disease (COVID-19) Public Health Emergency.

Time use limitation

If the mask becomes damaged or soiled, or if breathing becomes difficult, leave the contaminated area and replace and refit with a new mask.


Important notice

To the extent permitted by law, BYD shall not be liable for any loss or damage, including any loss of business, loss of profits, or for any indirect, special, incidental or consequential loss or damage arising from reliance upon any information herein provided by BYD. BYD SPECIFICALLY DISCLAIMS ALL WARRANTIES, EXPRESS OR IMPLIED, INCLUDING (WITHOUT LIMITATION) IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. BYD SPECIFICALLY DISCLAIMS THAT THE MASK WILL PREVENT INFECTION OR THE TRANSMISSION OF VIRUSES OR DISEASE.

All information presented in this document is based on the latest data available at the time of printing. BYD reserves the right to make changes at any time without prior notice.

Approval and Standard

Product model: FE2311X (FLAT) ASTM F2100 Level III YY 0469-2011 standard

EN 14683:2019 Type IIR 



Distributed By: Bigfoot Inc.
7378 W Atlantic Blvd #210
Margate, FL 33063
888.850.7273
www.BigfootPPE.com

