




# Single-use Surgical Mask

SOFT & EASY TO BREATHE

Non-sterile

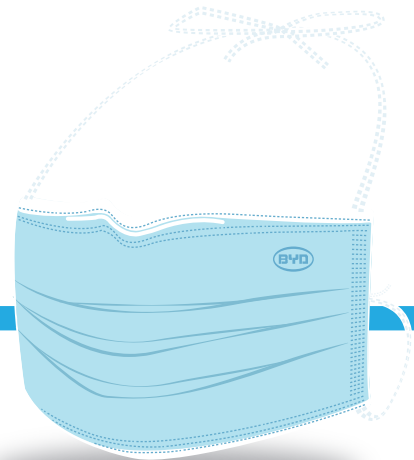

**BYD CARE** 

Product Model

FLAT FE2811



## Product Introduction

BYD Single-use Surgical Mask consists of three layers of nonwoven material:



**Outer Layer**  
Polypropylene spunbond nonwoven fabric

**Middle Layer**  
Polypropylene melt-blown nonwoven

**Inner Layer**  
Polypropylene spunbond nonwoven fabric

### MATERIALS

Inner layer	Polypropylene spunbond nonwoven fabric
Middle layer	Polypropylene melt-blown nonwoven
Outer layer	Polypropylene spunbond nonwoven fabric
Ties	Polypropylene spunbond nonwoven fabric
Nose clip	Metal wire with plastic covering

This device is not made with natural rubber latex. Non-sterile product. One-time use only.

## Wearing directions



1. Ensure the side with nose clip is facing upwards before applying the mask to your face.



2. Before applying the mask to your face, make sure the white side of the mask is facing towards your face while the colored side facing outwards.



3. Pick up the mask by the upper ties, place the ties around the back of your head and attach them together with a bow.



4. Now that the medical mask is in place on your head and face, use your index finger and thumb to pinch the bendable portion of the top edge of the mask around the bridge of your nose.



5. Then take the bottom ties, one in each hand, and secure with a bow at the nape of your neck.



6. Once the mask is completely secured, adjust it to ensure it covers your face and mouth, and so the bottom edge is under your chin.




# Single-use Surgical Mask

SOFT & EASY TO BREATHE (Non-sterile)

## Indications for Use

The Single-use Surgical Masks (Model: FLAT FE2811) are intended to be worn to protect both the patient and healthcare personnel from transfer of microorganisms, body fluids and particulate material. These face masks are intended for use in infection control practices to reduce the potential exposure to blood and body fluids. This is a single use, disposable device(s), provided non-sterile.

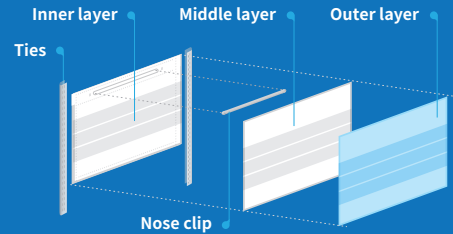

## Product Information

Name	Parameter
Product name	Single-use Surgical Mask
Materials	Polypropylene spunbond nonwoven fabric (for surgical use), polypropylene melt-blown nonwoven, metal wire with plastic covering nose clip, Polypropylene spunbond nonwoven fabric ties
Product model	FLAT FE2811
Size	175 mm×95 mm / 6.89in.×3.74in.
Expiration date	2 years after production
Packaging specification	10pcs/bag; 50pcs/box; 1000pcs/case
Storage	Store in a dry, clean environment with room temperature
BFE ≥ 98%	PFE ≥ 98%
Flammability: Class I	The masks are tested according to the ASTM F-2100 and 16 CFR-1610 standard.

## WARNING

- 1 The mask does not eliminate the risk of contracting any disease or infection.
- 2 Failure to properly use and maintain this product could result in illness or even death.
- 3 Only for one-time use, please destroy and dispose according to regulations after use.
- 4 Please use within the validity period.
- 5 DO NOT use if package is damaged.
- 6 Please refer to the instructions before use.
- 7 For adult use only.

### Time use limitation

If the mask becomes damaged, soiled, or breathing becomes difficult, leave the contaminated area and replace and refit with a new mask.

### Important notice

To the extent permitted by law, BYD shall not be liable for any loss or damage including any loss of business, loss of profits, or for any indirect, special, incidental or consequential loss or damage arising from reliance upon any information herein provided by BYD. Nothing in this statement will be deemed to exclude or restrict BYD's liability for death or personal injury arising from its negligence.

All information presented in this document is based on the latest data available at the time of printing. BYD reserves the right to make changes at any time without prior notice.

YY 0469-2011 standard

ASTM F-2100 Level III

Distributed by  
**Bigfoot Inc.**  
7378 W Atlantic Blvd #210  
Margate, FL 33063  
www.BigfootPPE.com  
**Made in China**

