


Auto-wrapping (tear-and seal) portable toilet

wrappon · TREKKER **WT-4**

User's Manual

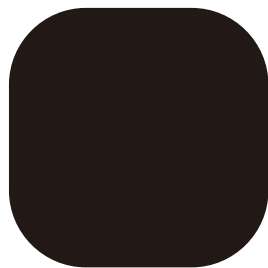

wrappon®

Auto-wrapping (tear-and-seal) portable toilet

Please read this manual very carefully and understand thoroughly before using it.
Please keep this manual clean.

 **NIHON SAFETY CO.,LTD.**

Introduction

Using the
Wrappon Trekker WT-4

Put it Away
when it is not using

When this happens

Table of Contents

Introduction

Before using the Wrappon Trekker WT-4	3
Product Overview	3
About User's Manual	3
To Ensure Safe Use	4
Confirmation of Parts and Accessories and Introduction of Consumable Supplies	9
Name of various parts	10
External dimensions	10
Installed status	10
Top view with cover removed	10
Warning and caution	11

Using the Wrappon Trekker WT-4

Assembling the Main Unit	12
Installing a Film roll	14
'STOP' mark appeared inside the film in the toilet(Film roll change)	16
How to use the toilet	17

Put it Away when it is not using

Maintaining the Product	18
Daily maintenance	18
If the wrapping unit becomes soiled	18
When installing a film roll	18
Storing the product	19
About the storage environment	19

When this happens

Frequently asked questions	20
Troubleshooting	21
Re-wrapping	24
Lights at Remote Control Unit	25
How to take care of tangled film	28
Specifications	30
Warranty and After-sale Service	31

Before using the Wrappon Trekker WT-4

Thank you for purchasing Wrappon Trekker WT-4, auto-wrapping (tear-and-seal) portable toilet. This User's Manual explains how to correctly handle Wrappon Trekker WT-4 as well as simple explanation and maintenance procedures. Please read this User's Manual thoroughly before using Wrappon Trekker WT-4 for safe use.

Product Overview





- Wrappon Trekker WT-4 is an auto-wrapping portable toilet designed for hygienic features. It also designs for elderly and handicapped persons and handicapped persons who have walking difficulties. But, please do not use for other purposes.
- Wrappon Trekker WT-4 is a clean, simple and odorless toilet that uses special coagulant and wraps the waste automatically.
- Wrappon Trekker3 cannot be wrapped by itself alone. The designated film roll must be used.
- The film roll is sold separately.
- When assembling the product, be careful not to get your fingers caught in the main unit.
- For any questions on how to use the Wrappon Trekker WT-4, contact the company you purchased.






About the User's Manual

- Keep this User's Manual close at hand for easy references by users. Should this User's Manual be lost or damaged, purchase a new copy from the company you purchased.
- The manufacturer may modify this User's Manual at any time without prior notice.
- User's Manual and warning labels do not necessarily show the shapes of items and sometime those are the icons. Note that the display values shown in the diagrams are only examples.
- If you transfer this product to another party, be sure to transfer the User's Manual and the Quick Reference Guide.
- The intellectual property rights for this User's Manual belong to the manufacturer. No part of this User's Manual may be reproduced without written permission from the manufacturer.

To Ensure Safe Use

Before using the product, read the "To Ensure Safe Use" section thoroughly to ensure correct use. The precautions described here represent important safety-related issues and must be observed.

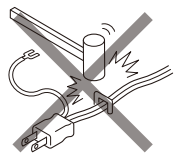Precautionary symbols and its meanings		Various symbols used to indicate and explain the items that must be observed.	
 Warning	Using the product incorrectly and ignoring this symbol may cause death or serious injury		This type of symbol means "PROHIBITED". The one shown on the left means "Do not disassemble."
 Caution	Using the product incorrectly by ignoring this symbol may cause injury or property damage		This type of symbol means "REQUIRED:" Left sign means "MUST BE OBSERVED:"

 Warnings		
<p>! MUST BE OBSERVED</p> <p>When using the AC adapter, connect it directly to an outlet(AC100V-240V)and securely insert the plug.</p> <p>If the adapter is not plugged in securely, electric shock or fire may result due to overheating.</p>	<p>! MUST BE OBSERVED</p> <p>Use only designated AC adapter.</p> <p>Use only the specified AC adapter. If not, it may damage the product or pose a danger.</p>	<p> PROHIBITED </p> <p>Do not use staples, nails, or wires to fasten the power supply cord.</p> <p>May damage the power supply cord causing electric shock, short circuit, or fire.</p>
<p> DO NOT TOUCH WITH WET HAND</p> <p>Do not touch with wet hand .</p> <p>May cause electric shock if you touch the electric plug and switch with wet hand.</p>	<p> DO NOT WET WITH WATER</p> <p>May cause short in the electrical circuit and corrosion on the product.</p>	<p>! MUST BE OBSERVED</p> <p>Reinforcement bar must be installed.</p> <p>Without it, the product may fall resulting injury.</p>

⚡ Warnings

▲ PROHIBITED

Be careful not to damage the cords, plugs, etc.

Do not damage, do not modify, keep away from high temperature, do not forcibly bend, do not twist, do not pull, do not place under a heavy object, or do not bundle them.

- A damaged cord or plugs, if used, may cause electric shock, short circuit, or fire.
- When unplugging, hold the plug itself and pull out straight.
- If the power supply cord becomes damaged, ask the seller to repair it.

▲ PROHIBITED

When the main power supply is ON, do not put your hand in the film or toilet.

The rollers or seal bar may trap your hand causing severe injury.

▲ PROHIBITED

Do not insert hand into the opening area from the bottom of the product.

Inside the product, there are moving parts which may damage and burn your hands.

▲ DO NOT DISASSEMBLE

Do not attempt to disassemble or repair the product.

High voltages are present inside the product, which may cause electric shock, fire, or product malfunctioning result in injury.

▲ PROHIBITED

Use the product only as a toilet and do not use for other purposes.

- Do not place inside a vehicle and use it as a seat.
- Do not use the product as a step.
- Do not use the product as a pallet.
- Do not use the product as a chair while taking the toilet seat off.

The above may cause the product to topple or be damaged, causing injury.

! MUST BE OBSERVED

Regularly clean the dust off from the power supply plug.

Accumulated dust may cause an insulation failure, resulting in fire due to overheating. Disconnect the power supply plug.

! MUST BE OBSERVED

If you hear thunder, do not touch the main unit or power supply plug. May cause electric shock.

▲ PROHIBITED

Do not place a heavy load on the product.


- Do not stand on the product.
- Do not move the product while person is on the product.
- Do not use two persons at the same time.

! UNPLUG

Before maintenance or inspection, disconnect the power supply plug.

If not unplugged, you may get an electric shock or an injury.

To Ensure Safe Use (continued)

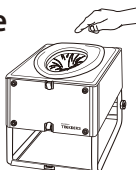⚠ Warnings		
<p>△ NO OPEN FLAME</p> <ul style="list-style-type: none"> • Do not install the product near an open flame. • Do not throw flammable material or lit cigarettes into the toilet. <p>Deform the product may cause short circuit or fire which may result in waste leakage. Do not use the product near a stove.</p>	<p>▲ PROHIBITED</p> <p>Children or adults who require assistance in moving must not use the product by themselves.</p> <ul style="list-style-type: none"> • May result in injury if mischief movement is done by a child. • May result in falling down of the product. 	<p>▲ PROHIBITED</p> <p>Do not use the person heavier than 150kg. It might be damaged and may result in injury.</p>
<p>! MUST BE OBSERVED</p> <p>Do not put the coagulant into your mouth. The coagulant is not food. Check the precautions for use.</p>	<p>! MUST BE OBSERVED</p> <p>The product must be used on a flat and stable location. If installed in an unstable location, the product may fall down and cause injury.</p>	<p>! UNPLUG</p> <p>If the product is not to use for a long time, disconnect the power supply plug from the electrical outlet. Insulation deterioration may cause electric shock or fire.</p>
△ Caution		
<p>! MUST BE OBSERVED</p> <p>Before use, make sure that you check if the film roll is installed.</p>  <p>If used without film roll installed, waste leakage may cause the product failure. Also no-load running may cause the product failure.</p>	<p>▲ PROHIBITED</p> <p>Do not throw large quantity of toilet paper into the toilet. When wrapping the film, the toilet paper may get tangled with the film, resulting bad seal with a leakage.</p>	<p>▲ PROHIBITED</p> <p>Do not install, use and store at the following places.</p> <ul style="list-style-type: none"> • Lots of dust and dirt. • Places such as bathroom over 80% humidity. • Locations exposed to rain and direct sun light. <p>If you install, use and store the product in these places, you may get the trouble.</p>

△ Caution

▲ PROHIBITED

Do not put in more than the specified amount of coagulant.

The coagulant may become trapped between the rollers which may cause the product to fail.

For the coagulant information, please refer the coagulant note on use.

▲ PROHIBITED

Do not use anything except the designated coagulant.

Using any other coagulant may cause insufficient coagulation, clogging, wrapping failure, or product failure.

! MUST BE OBSERVED

The user must sit squarely on the toilet seat.

If the user sits close to the front edge, the waste may leak and cause the product to fail.

! MUST BE OBSERVED

Throw used toilet paper into the middle of the toilet.

If not observed, it may cause clogging and wrapping failures.

▲ PROHIBITED

Do not press the push button while seated on the toilet.

The film will get caught in the toilet seat, causing product failure or wrapping failure.

▲ PROHIBITED

Use the product only if wrapped film does not remain on the bottom cover.

If wrapped film remains, it may cause product failure or wrapping failure.

! MUST BE OBSERVED

When installing the film roll, the power supply cord must be disconnected.

If not observed, it may cause clogging and wrapping failures.

▲ PROHIBITED

Do not use other film roll than designated film roll.

Using other film roll may cause it to get trapped inside the product, and may cause waste leakage or product failure.

! MUST BE OBSERVED


Install the toilet seat to the body properly.

Wrong installation may cause the product fall or product malfunction. This may result in product failure.


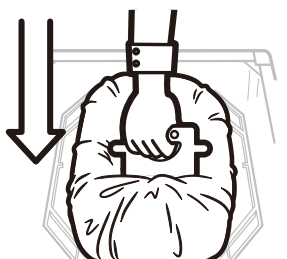
! MUST BE OBSERVED

Make sure that you press the "Yellow" button for initializing after installing a new film roll.

If you do not run the initial film-bonding operation, the waste may leak, causing the product to fail as the film is not sealed.

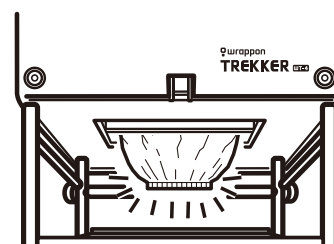



To Ensure Safe Use (continued)

<div>  注意 </div>		
<p>! MUST BE OBSERVED</p> <p>Push down the film installation jig while aligning it with the protrusions on the top plate.</p>  <p>If the product is used without a film installation jig, film may be dislocated from rollers and unbalanced film feeding which may cause the wrapping failure.</p>	<p>▲ PROHIBITED</p> <p>To clean the product, use a diluted solution of mild kitchen detergent. Do not use any of the following : toilet cleaner , benzene, solvent, or cleanser.</p> <p>Using such a prohibited product may damage the product, causing injury. It may also cause product deformation or discoloration.</p>	<p>▲ PROHIBITED</p> <p>Do not stack up more than five units.</p> <p>Products may fall over and may be broken resulting injury.</p>
<p>▲ PROHIBITED</p> <p>Do not relocate the product while connecting the power source plug.</p> <p>You may trip on the cord resulting the disconnection of the plug or short circuit.</p>	<p>▲ PROHIBITED</p> <p>Do not move without wrapping.</p> <p>Waste may leak. The floor and the product is soiled. So make sure that you move the product after completing the wrap cycle.</p>	<p>! MUST BE OBSERVED</p> <p>Do not place the product on the uneven floor.</p> <p>The product may get broken.</p>
<p>! MUST BE OBSERVED</p> <p>Reinforcement bar must be installed.</p> <p>Without it, the product may fall resulting injury.</p>		


The Parts and Accessories List and Description of Consumable Supplies

The Parts and Accessories List


- The parts and accessories shown below are pre-packaged with the product (those indicated by an asterisk(*) are installed inside the product).
- Before using the product, check to make sure all of the accessories are intact. Should any accessory be missing or damaged, please notify the seller of the product immediately.

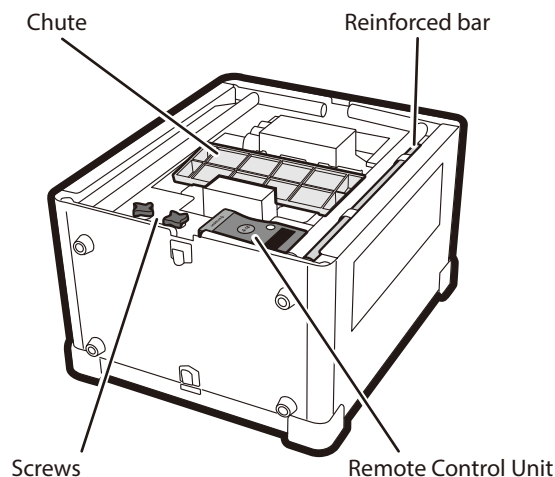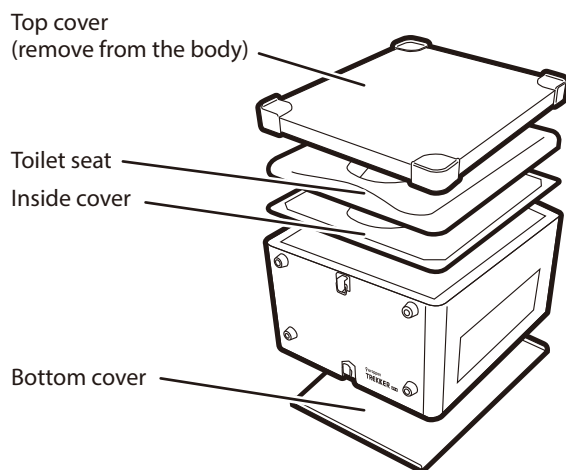

Accessories List

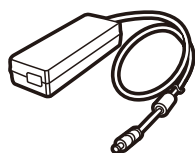


AC adapter (1)

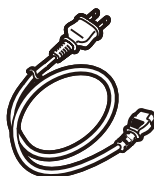

Power supply plug (1)



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
User's Manual (1)



Quick Reference Guide (1)




Hook-and-loop tape(1)
To fix the remote control unit.
Place it any where

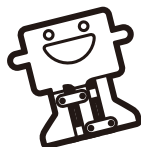

Film Installation jig (1)

Description of Consumable Supplies

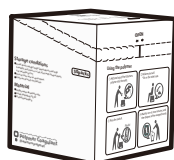

Special coagulant (50 packets/box)



Film roll (50 times per roll)




Moist tissue (100 sheets per bag)


Name of various parts

External dimentions(cm)

To the toilet seat

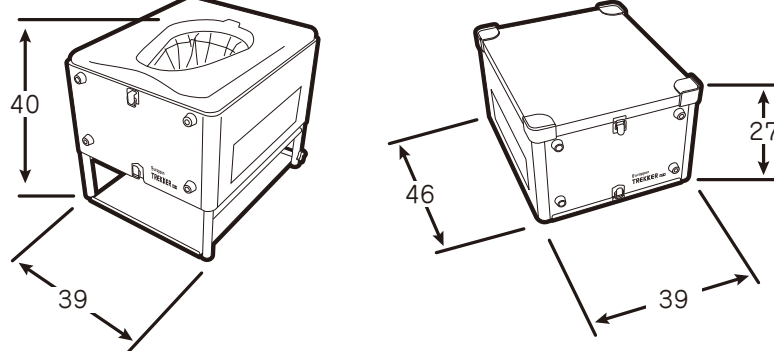

When stored, you can stack five units.

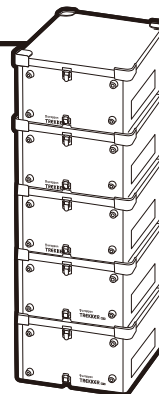

Installed status

Top cover
(remove from the body)



Toilet seat

Inside cover


Handle
(back side)

Legs

Bottom
cover

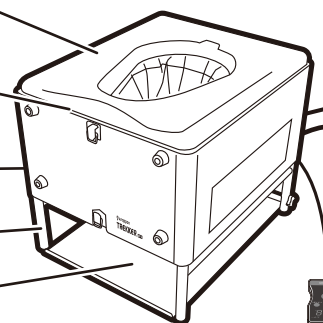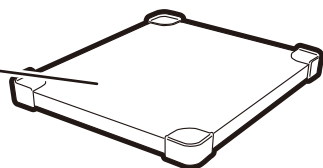

AC adapter • Power supply cord

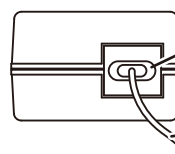

Plug in area

Power supply
cord

Remote Control Unit

Power
supply light
(Green)


Display
window

Flush button

Condition
light
(Green/Red/
Yellow)

Error light
(Red)

Film feed
button

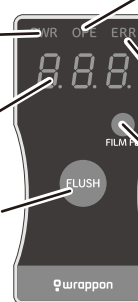

Top view with the toilet seat removed


Film guide

Top plate

Heater (located inside)

Roller


Press bar (located inside)

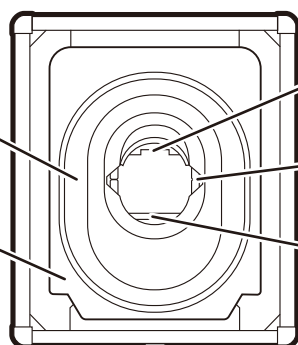


Assembling the Main Body

- Read 'Ensure the safe use' on page 4~8 before assembling.
- Check 'Parts and accessories list' on page 9 to make sure all are available.
- Use outlet with a ground terminal to plug in.


1 Place the product with the bottom.

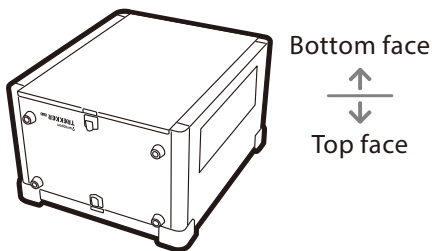

2 Remove the bottom cover by unlockin the two locks.

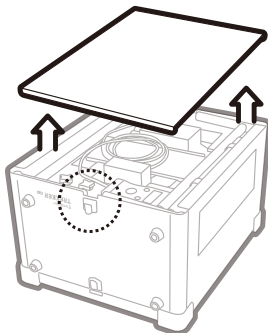


3 Pull up the both legs and connect the both AC adapter cord.

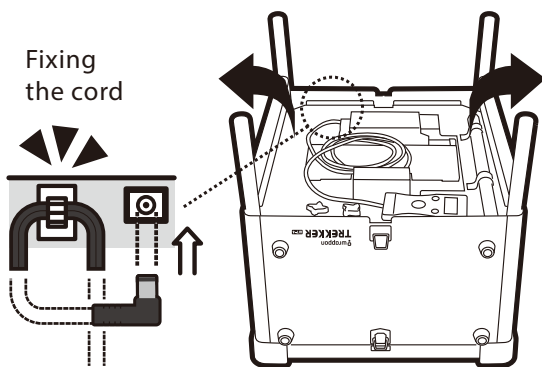


*The back side is the side with handle.

- ① Insert the plug without twisting.
- ② After connecting, fixing the plug cord.


4 Pull and place the cords of AC adapter Cover up and the Remote Control Unit through the backside.

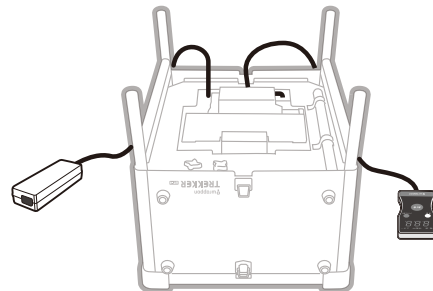

5 Remove the reinforced bar and remove two screws from the bar ends.

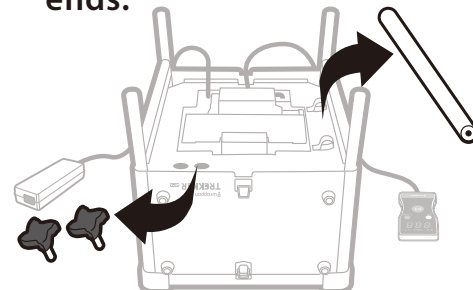


6 Install the reinforced bar with Legs with screws.

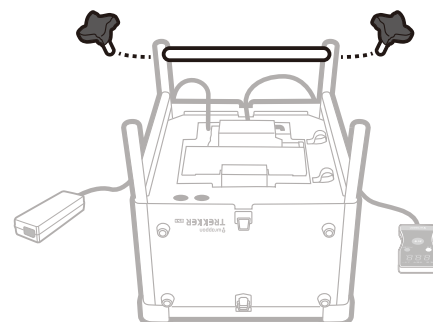


Pull and place the cords of the AC Adapter and the Remote control unit through the backside.

Assembling the Main Body (continued)


- 7** Pull up the chute by removing.
If the chute is dislocated, you can re-install the chute by putting it back.

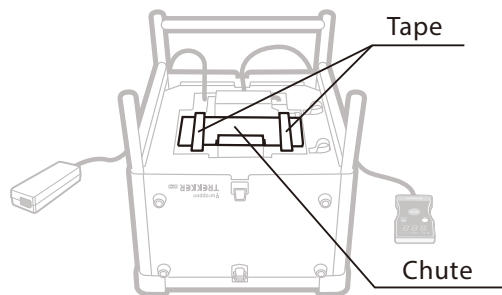


- 9** Set the bottom cover in between the legs.

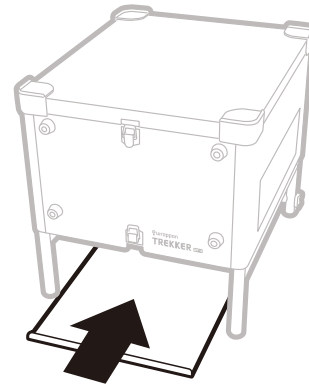


- 8** Put the main body back to the normal position.
Connect the electric power cord to AD adapter.

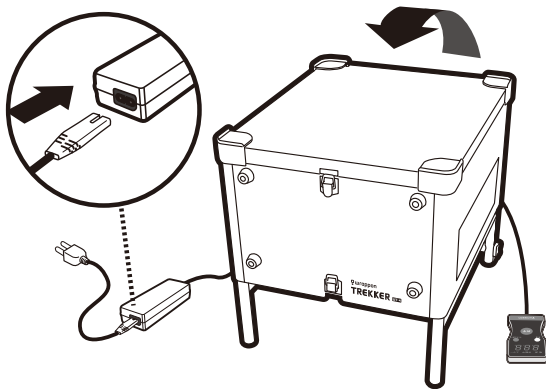

- 10** Unlock and remove the Top cover.

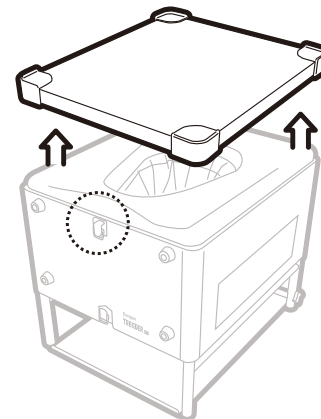

By this the assembling is finished.
Do not connect to the power supply
before installing the film roll.

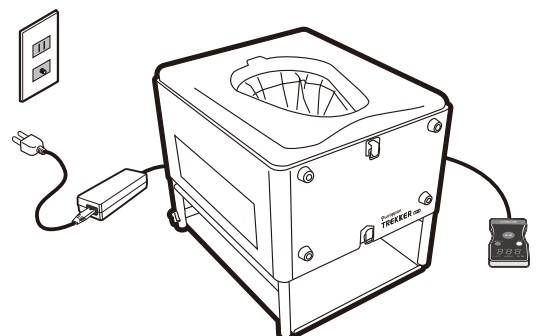


Installing a film roll

- Before installing a film roll, check 'assembling the main unit' on P11.
- While the plug is on, do not insert your hand inside the toilet.
- To function wrapping mechanism properly, a film roll must be installed properly.
- Do not use other than the designated film roll.

1


Make sure that the power supply is disconnected from the outlet.

Remove the toilet seat and inside cover.

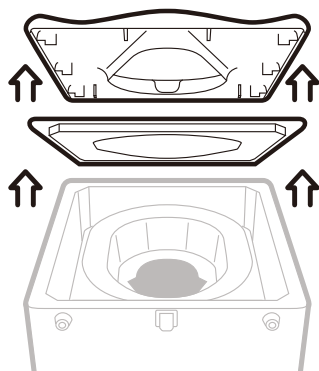


2

Unlock two clips and remove the film guide at upward.

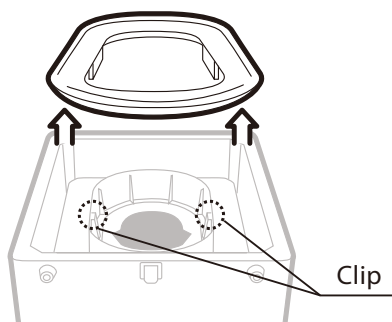


3

Install a film roll along with film guide.

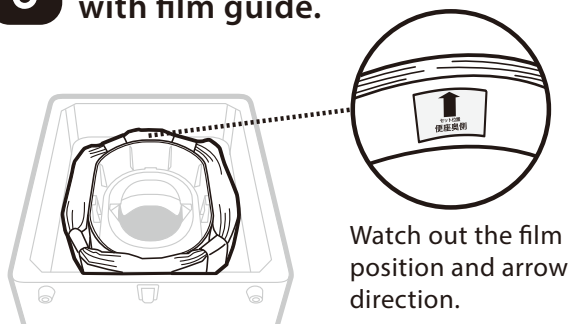

4


Install the film guide which you took at 2.

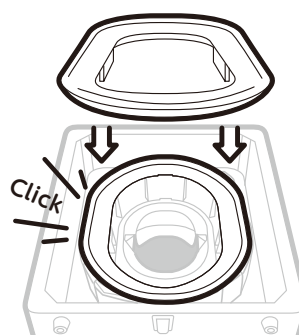

Push it until you hear the click sounds.

5


Remove Clamping bands at four locations of the film roll.

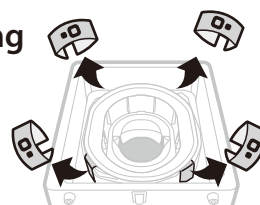


6

Take the film cassette cover out.


- ① Lift up the uppermost edge of the film roll from the upper end of the film guide by approximately 30cm.

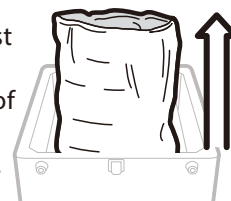

- ② Grip the upper portion of the film roll in a lump.


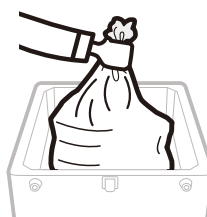


- ③ Push the lump in the toilet as far as you can.

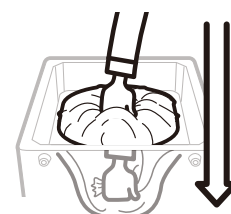


Installing a film roll (continued)

- 7** Push down the film installation jig while aligning it with the protrusions on the top plate.

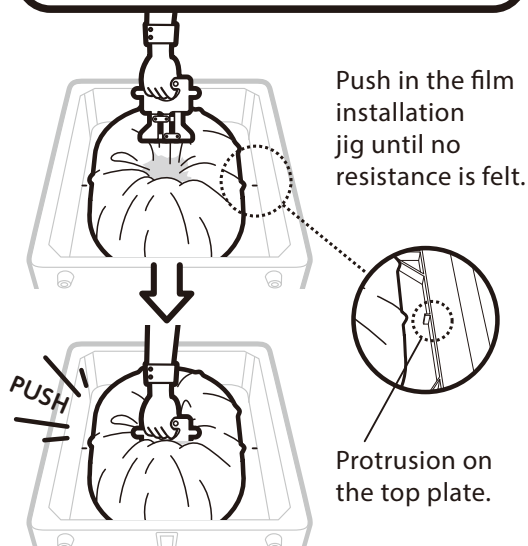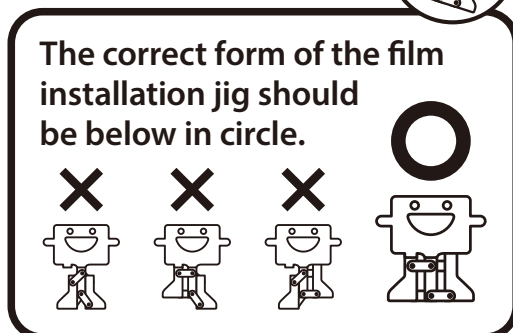


- 8** Pull up the film installation jig. Check if the film is properly set or not.

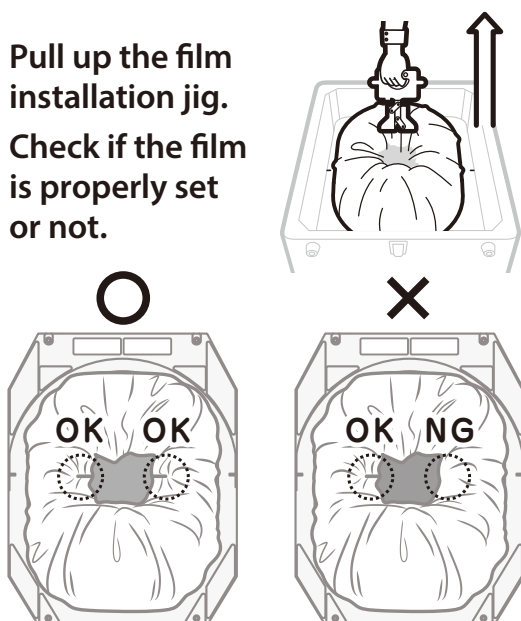

Film must be sandwiched both left & right set of rollers.


Film is sandwiched with one set of rollers.

- 9** Put the power plug into an outlet. Make sure that the power light is green.

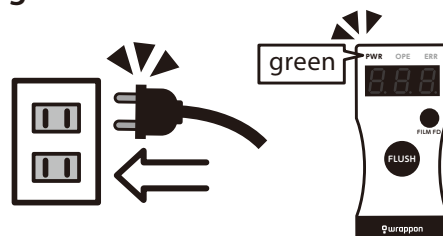


- 10** When you push the flush button, the initial bonding starts. While wrapping, condition light flashes green.

※Without this, the bottom of the bag is not sealed.

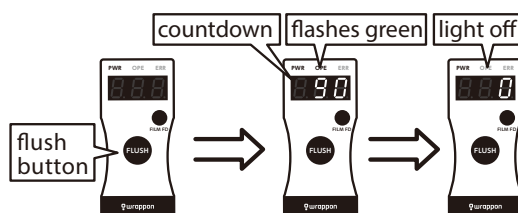


Wrapping takes about 90 seconds. Display screen shows an estimated time remaining to complete the wrapping (approximately 90 seconds countdown) When completed wrapping, you hear beep sounds. Condition light of remote control is off.

- 11** After condition light is off, take out the separated film. This completes film setup.

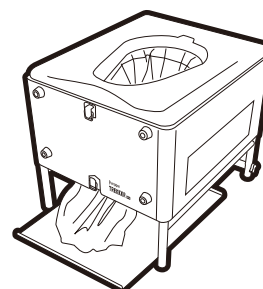


When you see the 'STOP' mark in the film (Changing the film roll)

- 1** **Change the film roll immediately.**
If you continue using the film, the film roll is running out, resulting a cause of waste leak.

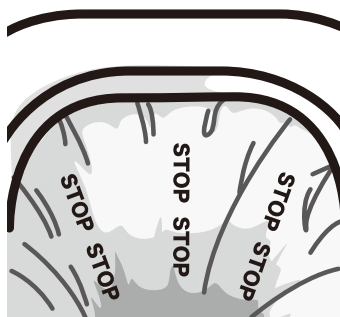


- 4** **Take out the toilet seat and inside cover.**

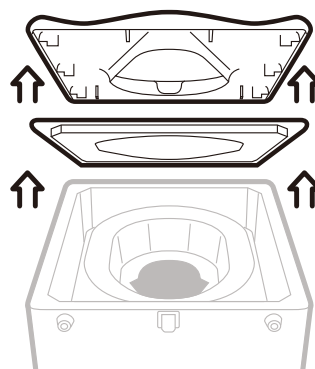

- 2** **After removing the toilet seat and the inside cover, press film feed button(blue).**

By holding down the film feed button, the film is sent out for about 20cm. You keep pulling the film roll below gently until the film is running out. If you did not pull the film, it may cause the film wrapped around the rollers.

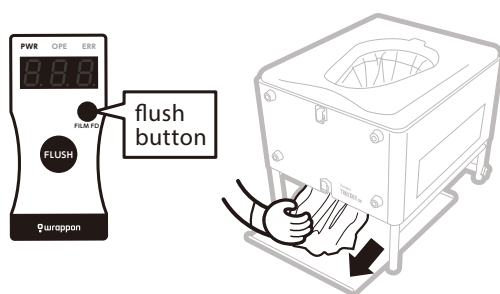

- 5** **Install a film roll.**
Refer to the process(2-11) of 'Installing a film roll' on page13.

- 3** **Pull the electric power cord from the Outlet.**


How to use the toilet

- Before using, make sure that the plug is connected to electric power outlet.
- Do not insert hand into the toilet.
- Sit down on the toilet as Western-style toilet.
- Do not stand and urinate. You should sit down on the toilet and urinate.
- Wrapping cycle takes about 90 seconds.
- When finished, you can hear the sound of beep, beep, beep. Then, the condition light turns to green.
- Do not throw cigarette and sharp-edged items in the toilet. You can throw human waste, wet tissue and sanitary goods.
- If you drop something in the middle of wrapping, wait until the cycle finishes. Then, you open the wrapped film and take it out.
- There is a possibility of waste leaking as the wrapped film is not 100% sealed due to the environment and condition. Also make sure that you use the coagulant and the tray.

★ Check before use ★

- Make sure that operation light of remote control is off.
- Make sure that you check if the film roll is installed. If the STOP mark is appeared on the film, replace it with a new film roll(refer P15).
- Check if there is no damage to the part of toilet seat that touches the body. If there is a damage, please do not use it.

- 1** **Pour one packet of special coagulant.**
Before you do defecation, make sure that you pour the coagulant.

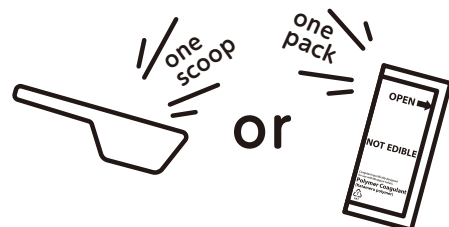

- 2** **Relief yourself.**
Sit on the soft toilet seat in the correct position.

Push down the toilet paper so it is not sandwiched with press bar.




3

When finished, stand up and press and hold the operation button on the remote control for one second.

Make sure that you are away from the toilet seat.

After about 90 seconds, you hear Beep sounds.
The wrapped film is automatically cut.


Do not start the next cycle until the film is completely cut off.

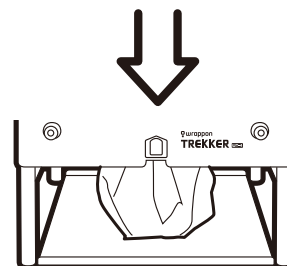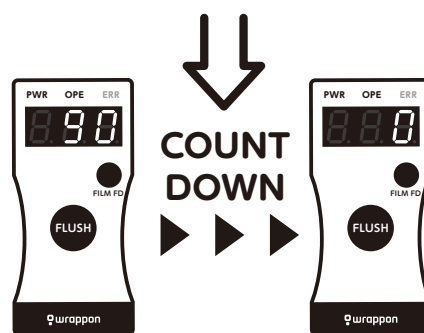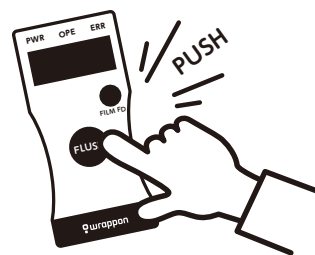

4

After checking that the condition light is green, pull the bottom cover out.
and Discard the wrapped film


Disposal of wrapped waste film must follow the local government.

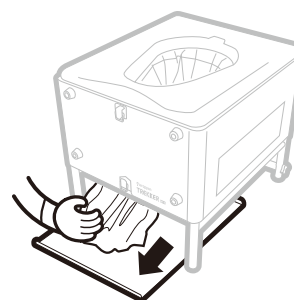

Selector switch of input power supply

Wrappon Trekker(WT-4) can use AC adapter(accessory),battery (sold separately) and DC cable for car (sold separately) as an input power supply.

※ If the input power supply and selector switch do not match, it detects voltage of input power supply which may result in unusable. So,please make sure that you select proper switch against input power supply.

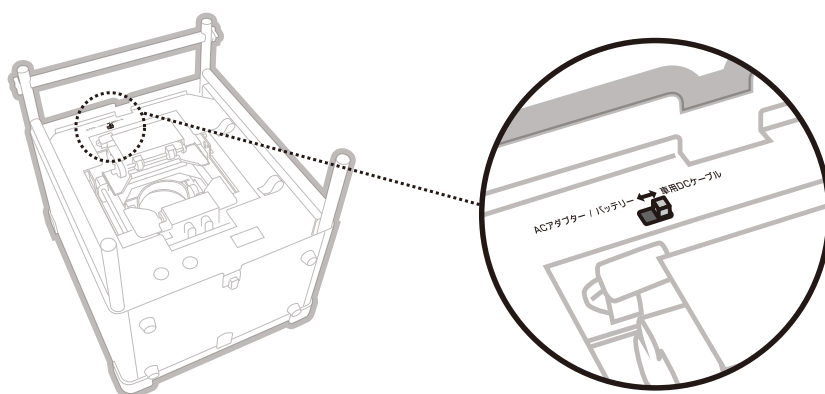

Input Power Supply

When you use AC adapter (accessory) and the dedicated battery(sold separately), please set the selector switch to the DC cable for car.

AC ADAPTER / BATTERY ↔ DEDICATER CABLE FOR CAR



When you use the dedicated DC cable for car, please set to the dedicated DC cable(sold separately) to the side of the selector switch as shown below.

AC ADAPTER / BATTERY ↔ DEDICATER CABLE FOR CAR



Maintaining the Product

Maintaining the product regularly to ensure hygienic use over a long period.


- Before performing maintenance or inspection, disconnect the power supply plug.
- Regularly dust the power supply cord.
- Keep the product clean.
- When cleaning, never use toilet cleaner, benzene, solvent or cleanser.
- After you finish cleaning, reinstall the soft toilet seat.

Daily maintenance

- For light soiling, wipe with a wet soft cloth lightly.
- For heavy soiling, wipe with a soft cloth soaked with diluted mild kitchen detergent.


Inside toilet is soiled

If the inside toilet is contaminated with human waste, do not insert hand into toilet. Stop the operation immediately. Pull off the power cord from power source. Please call the shop or the distributor.

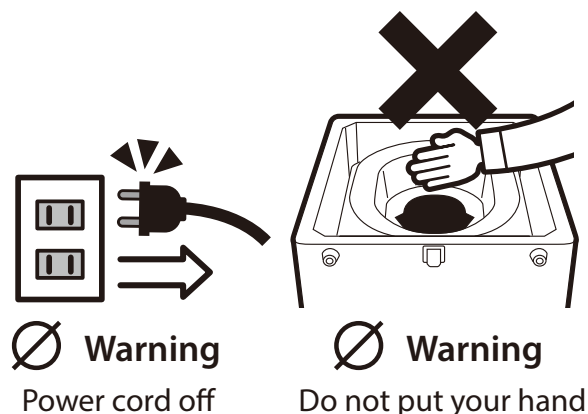

Installing the film roll

When you change a film roll, it is recommended to wipe the back side of soft toilet seat and the top plate. Wash with soft cloth soaked with diluted mild kitchen detergent. After that, wipe with water. Inside cover is consumable. Purchase a new one if it is soiled or broken.

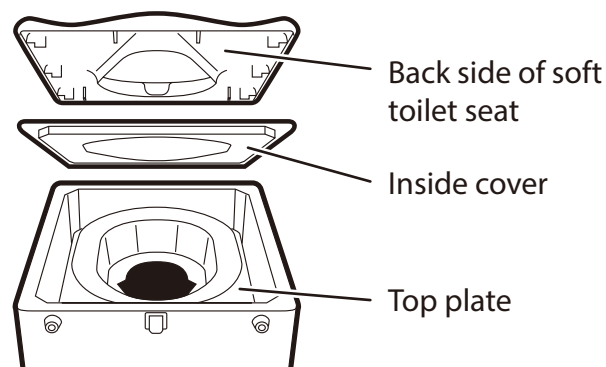

Storing the product

- Before storing the product, clean the toilet seat and other parts by referring 'Maintaining the product'.
- When the product is not used immediately, disconnect the power supply plug from the electric power outlet.
- If wrapped film remains on the bottom cover, remove and discard it.
- If wrapped film is stored for a long time with waste inside, odor and smell occur. Earlier disposal of the wrapped film is recommended.


About the storage environment

An ideal storage location is an indoor with temperature between 0°C~40°C with low humidity.

Caution

Do not use the location as listed below.

- Dusty location
- Locations such as bath room where the humidity exceeds 80%.
- Locations exposed to rain or direct sunlight.


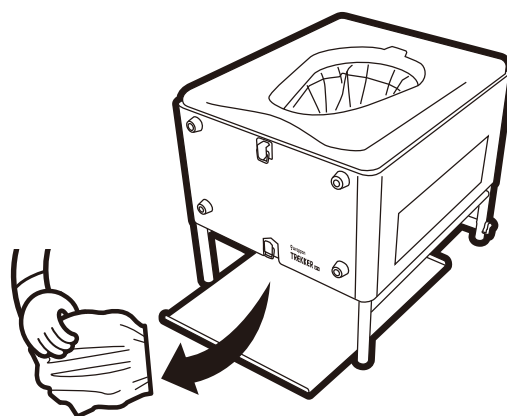



Frequently asked questions



Question	Answer
Will the wrapped waste really not emit odor even if left as is.	If wrapped film is stored for a long time, wrapped waste in the bonded area may deteriorate and cause odor leakage. So, dispose it without delay.
Do I have to wash the tray every time?	It should not become soiled because waste is wrapped after each use. Should the tray become soiled, wash it with water.
Isn't replacing the film roll difficult?	The film roll is designed to change without difficulty. Just follow the film roll replacement section in this manual or quick reference.
Can I dispose of the wrapped waste as combustible waste?	Like disposable diapers, wrapped waste is treated as combustible waste. However,, because the specific rules vary among individual local governments so check with your local government.
How should I clean the product?	Gently wipe with a soft cloth soaked with diluted mild kitchen detergent and rinse with a damp cloth dipped in water and rung out. The unit contains a heater for wrapping the film. Therefore, never pour or spray water on the unit when cleaning the unit. Otherwise, the heater may malfunction.
Why do I have to use the wet tissue?	If light paper such as toilet paper is used, it may become trapped during the wrapping operation and cause waste leakage. Therefore, use only designated flushable wet tissue.
Is it OK to flush waste that has been coagulated with paper coagulate down a regular flushing toilet?	Yes, the volume that can be flushed at one time is about one cup(200CC). So, you may have to flush the coagulated waste in several times.
Is it OK to flush waste that has been coagulated with the polymer coagulated down a regular toilet?	No. Because the polymer is water-insoluble, do not flush it down a regular flushing toilet.

About the display window and condition lights on the remote control unit.


Display when the toilet is bging used.

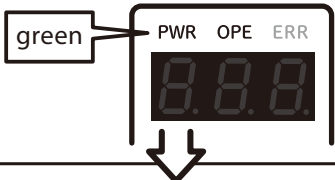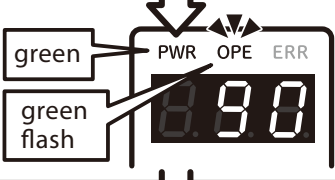Display	Status	Explanation
	Ready	Power supply light is green.
After relieving, press and hold the flush button of remote control unit for one second.		
	Wrapping	Operating light flashes green. The approximate remaining time to complete wrapping will be displayed on the display screen. Please wait for while as it is wrapping. *Wrapping takes approximately 90 seconds.
	Ready	The operating light goes off and beep sounds are heard. After taking out the wrapped film from the bottom cover you can use the toilet again.



Display when re-wrapping

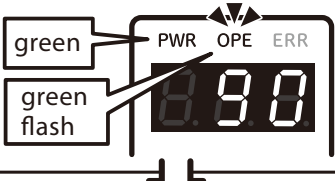Display	Status	Explanation
	Re-Wrapping	When the operation is interrupted for some reason while wrapping, wrapping motion will be resumed after the power is turned on again. Power light up in green and operation light flashes green.
	Ready	Wrapping is completed when the operation light is turned off and beep sounds heard. Remove the wrapped film, then you can use it again.


Economic mode and standby mode

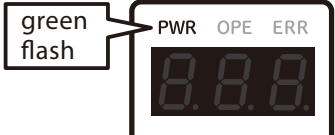Display	Status	Explanation
	Standby	After one minute and five minutes, it becomes an economic mode as the power light flashes. Then, if you do not use more than 60 minutes, the power is turned off automatically. To restart, you push any button, then the power is on and after initaial operation, it can be used again.

Display after film roll is installed

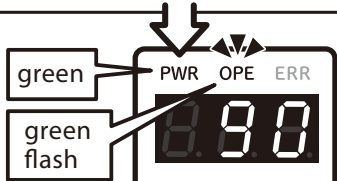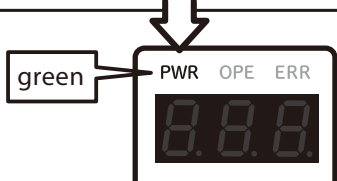Display	Status	Explanation
Press the operation button		
	machine is in operation	<p>Operation light flashes green.</p> <p>Estimated remaining time to complete the wrapping process is displayed on the display screen.</p> <p>So, please wait for a moment.</p> <p>*Wrapping takes approximately 90 seconds.</p>
	Ready	<p>Wrapping is completed when the operation light is turned off and beep sounds are heard.</p> <p>Remove the wrapped film, then you can use it again.</p>


When the error occurs

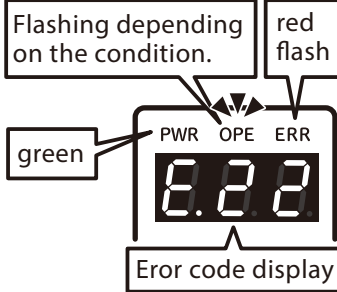Display	Check the following	Countermeasures
	Displayed error codes are E.22, E.23 and E.24.	<p>⇒ Buzzer sounds.</p> <p>Remove the foreign articles near the sealed area and push the operation button.</p>
	Other than the above	<p>⇒ Please contact the seller.</p>

Troubleshooting

If the product does not work correctly after you have tried the steps described in the table below, contact the seller.

Remote Control Unit

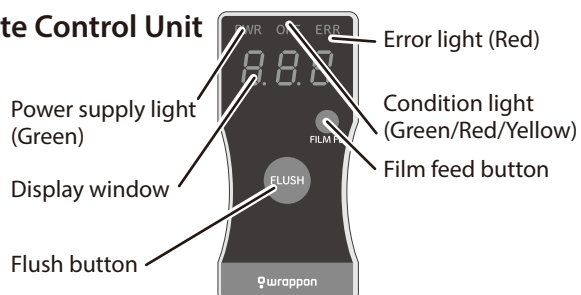

Explanation of each button

Flush button

Keep pressing operation button, then the processing.

Film feed button

Film feed button is to send out the remaining film.

Trekker WT-4 status	The power does not turn on(the power supply light on the remote control unit is not lit.)	Is the power off ?	⇒ If you are not using more than 60 minutes. the power is automatically turned off. Push any button of remote control, then, the power is on.
		Has there been a blackout or tripped breaker?	⇒ If the circuit breaker has tripped, unplug the power supply cord from the electrical outlet, and reset the circuit breaker. Plug the power supply cord. Turn on the main power supply.
		Has the AC adapter been disconnected?	⇒ Securely plug in the AC adapter into the main unit.
		All other cases	⇒ Call the seller
	Waste does not get wrapped even when I press the flush button on the Remote Control Unit.	Is the power supply light (green) of the remote control unit OFF?	⇒ The power supply is not on. See 'The power does not turn on' in the above.
		Did you press the flush button on the remote control unit immediately after turning on the main power supply?	⇒ You must wait for about 10 seconds after turning on the main power supply to make sure the condition light on the Remote Control Unit is green. Then, press and hold the flush button on the Remote Control Unit.
		Is the error light on the remote control unit flashing?	⇒ Refer to "Display window and error lights on the Remote Control Unit" on page 23.
		All other cases	⇒ Call the seller
	The display window of the remote control unit shows something that is not described in "Display window" (page 22-23)		⇒ Call the seller

When this happens		Check the following	What to do
During in use	The Trekker WT-4 wobbles when a user sits on it.	Is the reinforced bar installed?	⇒ Refer to Steps ⑤ and ⑥ under “Assembling the Main Body” on page 11.
		Are the screws for installing the reinforced bar loose?	⇒ Tightly fasten the screws at both ends of the reinforced bar.
		All other cases	⇒ Call the seller
	The film inside the toilet is torn.		⇒ Stand up, press and hold the button for one second(the amount of film for a single operation will feed). Note: Do not press the button on the remote control unit while seated. This may cause the film to tear. You must press the button after you have stood up.
	The toilet was used without a film roll installed.		⇒ DO NOT USE! Pull the electric power cord from the Outlet. Call the seller for assistance.
	The red “STOP” mark has appeared on the film inside the toilet.		⇒ Replace the film roll. Refer to “Replacing the film roll” under “Installing the film roll” on page 15.
	The user forgot to put in the coagulant polymer.	Before the button on the remote control unit is pressed.	⇒ Add the one packet of polymer coagulant and then press the button on the remote control unit.
		After the button on the remote control unit has been pressed.	⇒ Follow the procedure described under “Re-wrapping” on page 26.
	Blackout during in use.		⇒ When the power is restored, wrapping operation is resumed. Do not pull the film before it cuts away after wrapping is completed.
	Wrapped film is not cut away.	Look into the toilet to see if wet tissue is stuck in it.	⇒ Press and hold the flush button on the remote control unit for one second(one cycle length is rolled with empty inside).Follow the procedure of the ‘Re-wrapping’ on page 26.
		Is the film stuck to the heater?	⇒ Peel off the stuck film from the heater and wipe with dried cloth.
		All other cases	⇒ Call the seller


Troubleshooting (continued)

When this happens		Check the following	What to do
During in use	Waste has leaked.	Did you forget to press the flush button after changing the film?	⇒ Follow "Correct re-wrapping" on page 26.
		Is there a hole in the wrapped film?	⇒ Follow "Correct re-wrapping" on page 26. If you press the button while seated, this may cause the film to tear. You must press the button after standing up.
		Did you pull the wrapped film forcefully before the film is cut?	⇒ Follow "Correct re-wrapping" on page 26. Wrapping takes about 90 seconds. After completing the wrapping cycle, you can hear the beep sounds. The flash light of the remote control unit becomes off.
		Did you forget to pour the polymer coagulant in?	⇒ This product does not seal completely. Depending on the operating condition and condition such as wrong film setting, the waste inside the film may leak so make sure you pour polymer coagulant. After that, refer to "Correct re-wrapping" on Page 26.
		All other cases	⇒ Call the seller
	Waste is not wrapped correctly.	Has the film come off the rollers?	⇒ Follow the steps ⑦, ⑧ under 'Installing the film roll' on page 14, use the installation jig to insert the film between the rollers.
		All other cases	⇒ Call the seller

Re-wrapping

- 1 Open the film that has been incorrectly wrapped.
- 2 Firstly, pour a scoopful of coagulant. Then, put the film of 1.
- 3 Press and hold the flush button of remote control unit for one second.

- If Wrappon Trekker WT-4 becomes soiled with waste, refer to 'Maintaining the Product' on page 19.
- Should the waste get on your hand, wash it thoroughly with water and disinfecting soap, and then, disinfect with alcohol.

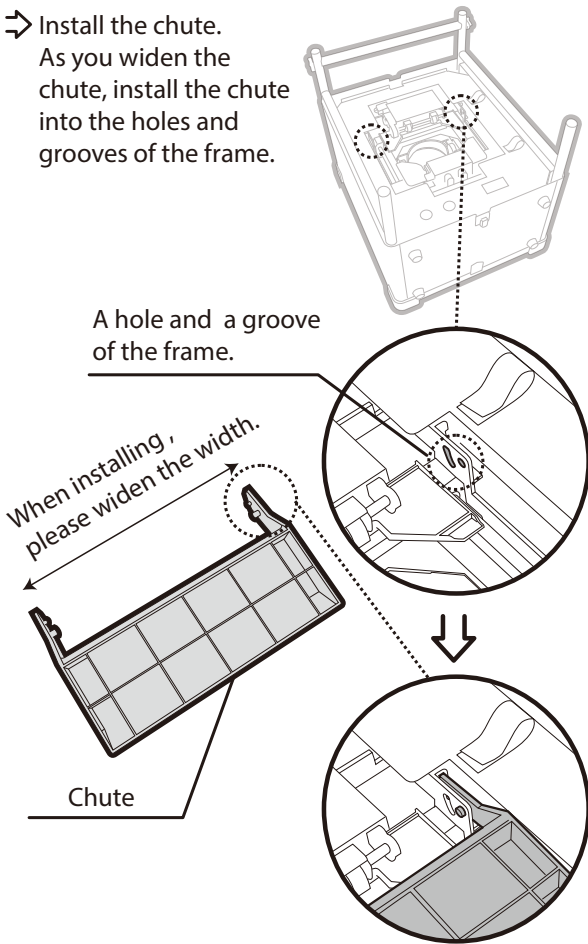When this happens		Check the following	What to do
Accessory & Consumables	Some accessories are missing.	Check the accessories against the parts and accessories list and consumable supplies on page 9.	⇒ If some items are missing, call the seller.
	Some accessories have been lost or damaged. I ran out of consumables.		⇒ Call the seller.
Burning smell or smoke is coming out of the product.			⇒ Stop using the product immediately and turn off the main power supply and unplug the power supply cord. Then, call the seller.
Someone accidentally ingested the paper coagulant.			⇒ This coagulant is not harmful to people, but it is suggested that you see the doctor soon if you are not feeling well or abnormal.
Someone accidentally ingested the Polymer coagulant.			⇒ Remove it immediately, rinse the mouth with water thoroughly, and see a physician.
Something was dropped accidentally into the toilet.			⇒ Wait until the wrapping cycle completed. Then, open it to take something out.
The chute came off			<p>⇒ Install the chute. As you widen the chute, install the chute into the holes and grooves of the frame.</p> 

How to take out the film while in use


· Before working, please unplug the power plug from the outlet.

1 Prepare scissors



2 Lift the outside of the film.


Lift up to a height of about 40cm from the top of the film guide.

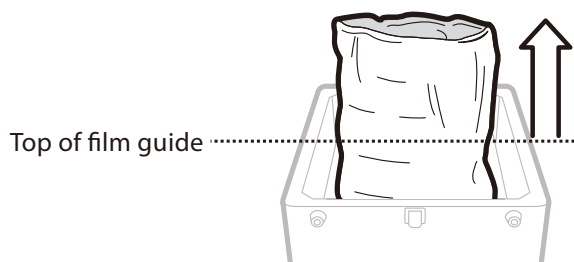

4 If the contents are inside the film, make sure to tie the inside film tightly to prevent the contents from spilling.


Please reduce the amount of air inside as much as possible.

Tie the top tightly

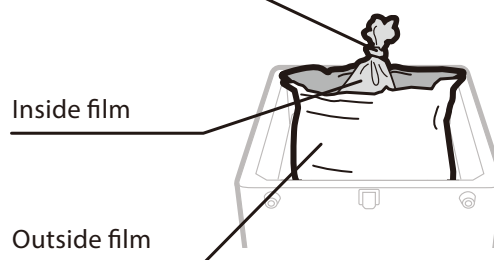


3 Cut the top of film with scissors.

The film is divided into two parts, inside and outside.

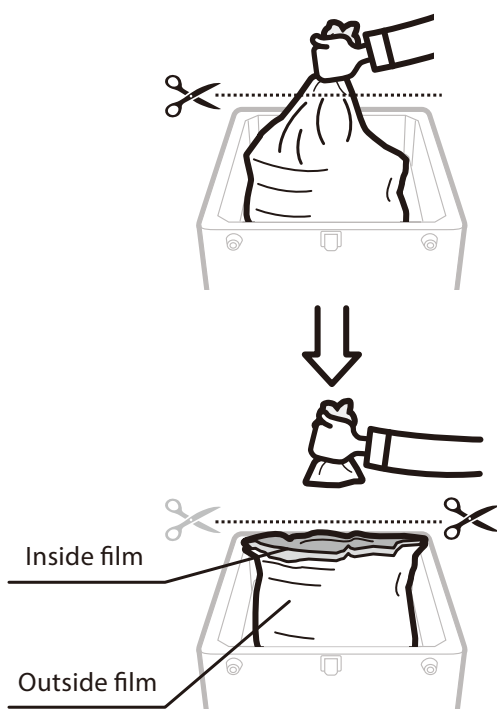

5 Pull out the inside and remove it from the roller.

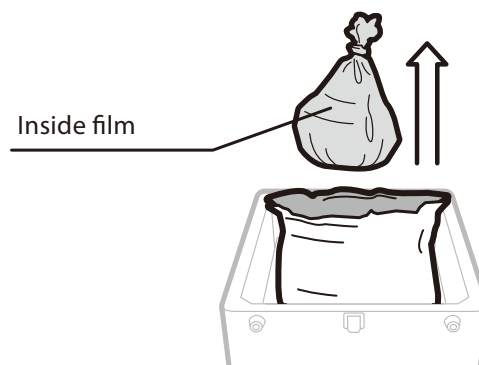

6 When installing a film, please refer to item 6 of "Install the film roll" on page 13.

Specification

Item		Specification
Product Name		Auto-wrapping(tear-and-seal) portable toilet Wrappon Trekker WT-4
Model No.		WT4SE002JH
Materials	Frame	Aluminium
	Film guide	Polypropylene resin
	Toilet seat	Polypropylene resin
	Leg	Aluminium
Power supply	Input voltage	AC100V±10V(AC adapter) Or DC12V±0.6V
Power consumption		75W or less during operation and 10W less in the standby state, 1W less at economic mode.
Dimensions	External dimensions	39cm(Length) x 46cm(Depth) x 27cm(Hight)
	Seat width	39cm
	Height to the toilet seat	40cm
Weight		Main body: Approx. 12kg, AC adapter Approx. 0.6kg
Maximum load capacity		Less than 100kg
Ambient temperature	When operating	0°C~+40°C
	When not operating	-20°C~+60°C
Ambient humidty	When operating	20~80% (No condensation)
	When not operating	5~95% (No condensation)
Ambient atmosphere		Corrosive gas must not be present.
Safety function		Pinch protection*1

*1 Pinch protection is the function to stop the movement when noticed the foreign material other than film during sealing movement. Do not put your hand or body into the toilet.

These specifications may be modified without prior notice.

Warranty and after service

If you have not registered as a user, the warranty is void.
Please make sure that you register either the enclosed user registration card or the website below. <https://wrappon.com/register>

Warranty(attached)

Please receive the warranty card from the store after confirming that the date of purchase and the store name are written. Please the contents carefully and keep it in a safe place.

Warranty period is written in a warranty card

Periodic replacement parts

Estimated replacement of periodic replacement parts(rubbers at heater and pressure bar) is approx. 2 years (expected use is 4 to 5 times a day).

- Heater and pressure bar which require thermo compression bonding are consumable parts. Because parts that deteriorate with increasing use, it is necessary to replace these parts. Please call the store or the safety call.

Periodic inspection

Correct operation and regular maintenance are recommended in order to use the product safely. For periodic inspection and maintenance costs, please call the store or Wrapponn safety call.

When requesting repair

Please check the pages from 24 to 27. If you still have a problem, stop using and make a note of the error display status on the remote control. Unplug the power cord from the main unit and call the store you purchased or call Wrapponn safety call.

During the warranty period

Please present the warranty card when repairing. According to the provisions of the warranty, we will take over and repair.

When the warranty period has expired

If it can be repaired and used, we will repair it if you wish.

Repair fee, how it works

Technical fee	Fees of diagnosis, parts replacement, adjustment, inspection at the time of repair completion, technical labor cost, technical education, equipment costs such as measuring equipment and general and administrative expenses.
Parts fee	Price of parts used for repair. It may include related parts for repair.
Business trip fee	The fee when dispatching a technician to the place where the product is located. A parking fee may be charged separately. *Basically, it is pick-up repair.

Even within the warranty period, below are paid repairs.

- 1) If this warranty is not presented.
- 2) If this warranty is blank for serial number, date of purchase, customer name, the store name. Or the words were rewritten.
- 3) Failure or damage caused by transportation, move, drop, and other improper handling after the purchase.
- 4) Failure or damage caused by fire, earthquake, salt damage, gas damage, wind and flood damage, lightning strike, abnormal voltage and other natural disaster.
- 5) Failure or damage caused by unjust repair by anyone other than our company.
- 6) Failure or damage caused by improper handling or usage which are different from the instruction manual.
- 7) Damage and dirt for accessories, attachments and consumables.
- 8) Failure or damage caused by power supply(Voltage, frequency) fluctuation.
- 9) For the case of repairable on-site, the customer requests on-site repair.

Before you call Wrappon safety call, please prepare the folloing.


1.Warranty

Please give us serial number, date of purchase and the store name which are written on the warranty.

※ If it breaks down within the stated warranty period, repair without charge is done if you show the warranty.
And,we charge repair fee for the repair not applicable of free repair stated in the warranty even within the warranty period.

2.If there is no warranty

Please check the serial number(10 digit number starting T4) which is affixed on the product.
It is listed on either inner left back after removing toilet sheet or on the back side of the main body.
After confirming, please inform serial umber,date of purchase and the store name.

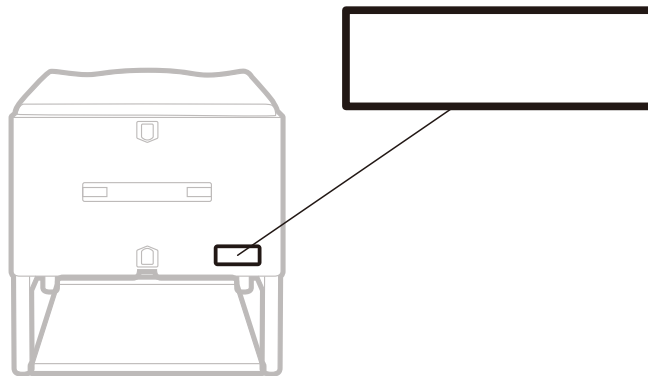



■ Manufacturer

 **NIHON SAFETY CO.,LTD.**

<http://www.wrappon.com>

Wrappon Division,Nihon Safety Co.,Ltd.
11F,Ichibancho Tokyu Bldg.
21 Ichibancho,Chiyoda-ku,Tokyo 102-0082 JAPAN
Phone : +81-3-6369-2223
E-mail:kaigo@nihonsafety.com
 is a registered trademark of NIHONSAFETY CO.,LTD.

■ Agent
