

# ANULON<sup>®</sup> PATRANS

Safe patient handling & moving




Anulon Patrans is a product that gives zero stress to the patient while transferred between beds, tables, and trolleys. It also makes the nursing work simple, easy, and stress-free. This product is mandatory in some of countries and is taken for granted in others.

It is simple to use and it reduces the difficulty in nursing. It is the most cost-effective and lightweight product.

Anulon Patrans not only relieves pain and adverse effect on patients but also relieves the hardship of Doctors and Nurses while moving the patient.

Nursing involves a lot of physical work, which leads to backache on the part of a nurse while moving the patient, particularly. It is crucial for the patient to take up the pain while they are being transferred from beds to the X-ray table, operation table, etc. Transferring patients may take place several times a day and it is particularly challenging for critical patients who require special care.

It is a special Product because it reduces the risk associated with patient handling and it also increases the efficiency of nursing staff. Hence, Anulon Patrans is beneficial for the nurses as well as patients.

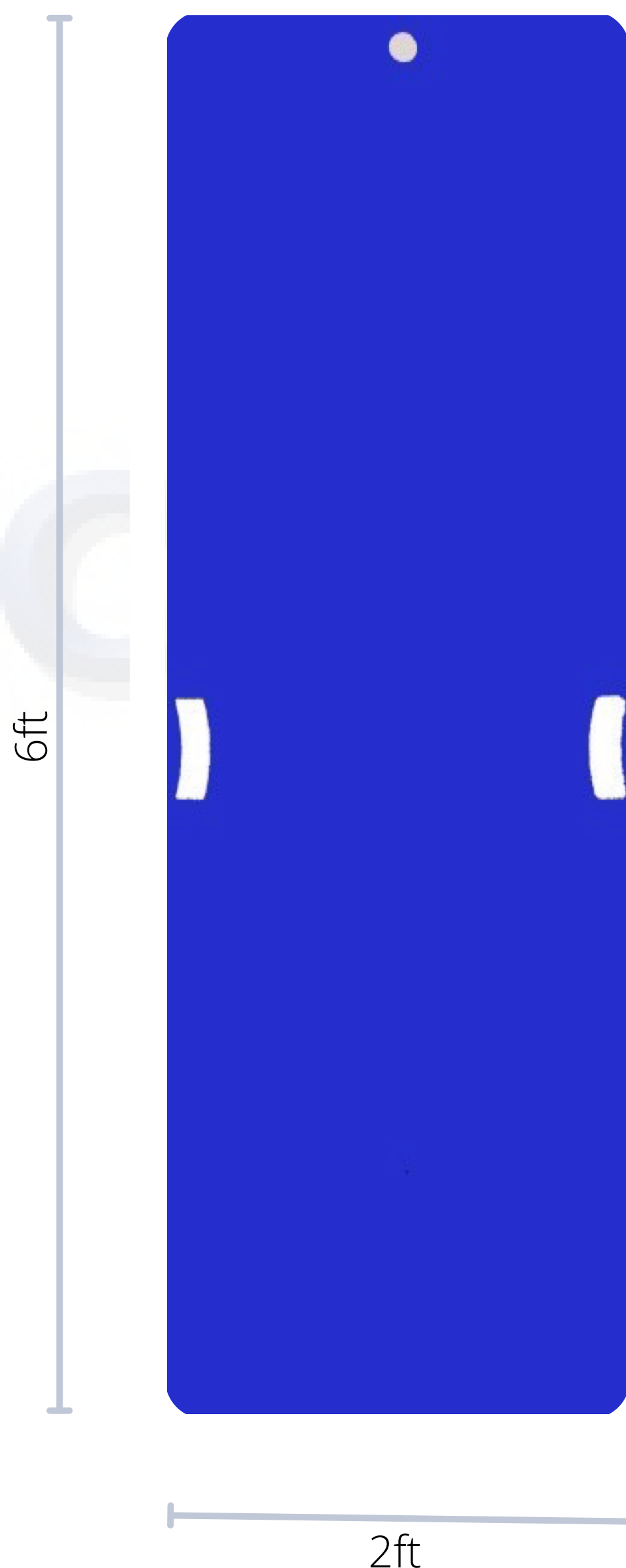

## PLASTIC ABHIYANTA

PLASTIC ENGINEERS AND CONSULTANTS

AN ISO 9001:2015 CERTIFIED COMPANY

25A, CHANDNI CHOWK STREET  
KOLKATA-700072

INDIA

✉ anulon@plasticabhiyanta.com

📞 +91-8240692783

# ANULON<sup>®</sup> PATRANS

Safe patient handling & moving



## USAGE

Step 1-First, position the stretcher next to the bed, and ensure locks being applied to both the stretcher and the bed.

Step 2-One care taker stands next to the patient and the other care taker should carry Anulon Patrans and position themselves on the other side of the bed where the patient will be moved towards.


Step 3- The care taker alongside the stretcher lifts the patient (one half turn) just enough to place the Anulon Patrans under the body of the patient. Meanwhile the other care taker slides the Patrans partway under the patient when he is being half turned. (Anulon Patrans should extend on to the bed to form a bridge between the bed and stretcher).

Step 4- Then the care taker who is on the side of the bed need to hold the sheet which is under the patient and hold it with the palms up grip keeping their elbows close to their body.

Step 5- Care taker alongside the stretcher places his hand against the patients shoulder and hip area with his fingers pointing upwards and forearms parallel to the bed. Then the care taker from the other end holding the sheet pulls the sheet on the count of 3 and the care taker from the opposite side pushes the patient slowly so as to make the transfer smooth.

## Video Guide

<https://youtu.be/-pmfOTFPw0o>

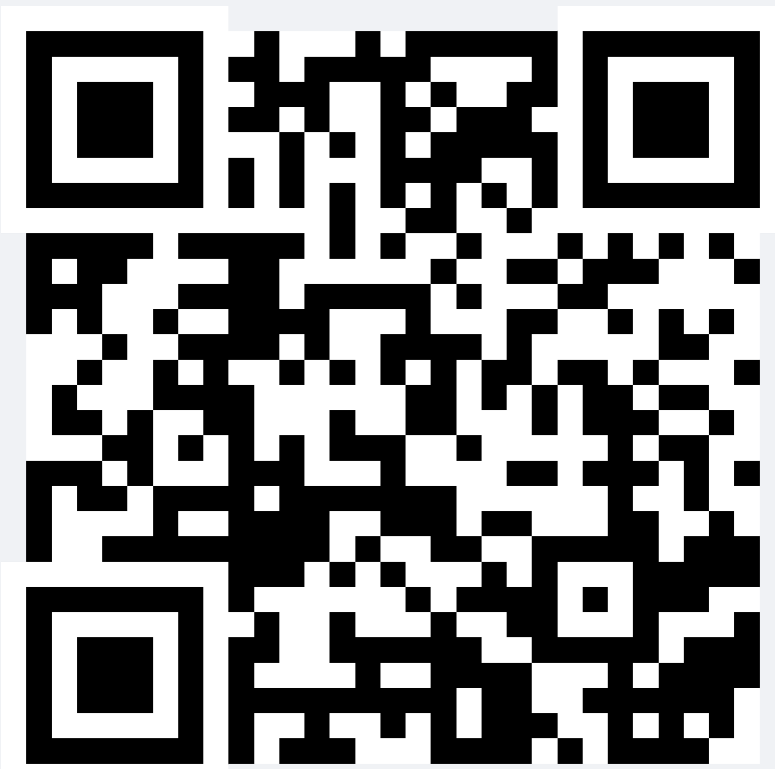

**PLASTIC ABHIYANTA**

PLASTIC ENGINEERS AND CONSULTANTS

AN ISO 9001:2015 CERTIFIED COMPANY

25A, CHANDNI CHOWK STREET  
KOLKATA-700072

INDIA

✉ [anulon@plasticabhiyanta.com](mailto:anulon@plasticabhiyanta.com)

☎ +91-8240692783