

Real Results



BEFORE

Photos Courtesy of F. Bopp, MD



AFTER

8 Treatments



BEFORE

Photos Courtesy of T. Humes, DO



AFTER

1 Treatment



BEFORE

Photos Courtesy of R. Green, MD, FACS



AFTER

7 Treatments



BEFORE

Photos Courtesy of M. Aggarwal, MD



AFTER

4 Treatments



BEFORE

Photos Courtesy of The Pink Door



AFTER

3 Treatments

Individual results may vary.

The perfect treatment
for any skin type.

Ask your provider if
Laser Genesis right for you.

Laser Genesis™

Indications for Use: The xeo+, excel V+ and excel HR (1064 nm Nd:YAG - Laser Genesis Treatment) is indicated for: removal of unwanted hair; stable long-term, or permanent hair reduction through selective targeting of melanin in hair follicles; treatment of pseudofolliculitis barbae (PFB); coagulation and hemostasis of benign vascular lesions such as, but not limited to, hemangiomas, port wine stains, telangiectasias, rosacea, venous lakes, leg veins, spider veins and poikiloderma of Civatte; treatment of wrinkles such as, but not limited to, periocular and perioral wrinkles; reduction of red pigmentation in hypertrophic scars where vascularity is an integral part of the scar; treatment of warts; treatment of mild to moderate inflammatory acne vulgaris. **Important Safety Information:** The following treatment-related expected transient side effects and possible adverse events may occur during or following treatment with xeo+, excel V+ and excel HR (1064 nm Nd:YAG - Laser Genesis Treatment): Erythema; Edema; Purpura; Hyperpigmentation; Hypopigmentation; Burns; Erosion; Epidermal crusting; Blistering; Scarring; Deep tissue injury; Prolonged wound healing; Temporary or permanent gray hair; Stimulation of terminal hair growth; Undesired hair loss in hair-bearing areas; Red rash/bumps; Hemosiderin staining; Textural changes; Cutaneous indentations; Itching/dry skin; Sun exposure, tanning beds, and artificial tanning may increase the risk of side effects and adverse events. Some discomfort may be experienced during treatment. Pain may include the feeling of burning, stinging, and radiating pain. **Contraindications:** Pregnant patients; patients undergoing treatment for skin cancer. In Canada, contraindications also include dysplastic nevi; Accutane if taken in the prior 6 months; photosensitizing drugs (Tetracycline, etc.); bleeding disorders or history of coagulopathies. **Warnings:** Do not treat over dysplastic nevi or questionable pigmented lesions. Do not treat over open wounds. Do not treat over or close to tattoos or permanent make-up. Do not treat over or close to metal or electronic implants. Do not treat over tanned skin. Treatment of tanned skin should be delayed to avoid complications.

CUTERA® ©2025 Cutera, Inc. All rights reserved. AP004281 rA (10/25)

Laser Genesis™

Radiant, Healthy
Looking Skin,
Your Way



Model. Not actual patient.

SIGNATURE TREATMENT

Laser Genesis™

Say “hello” to radiant skin

Laser Genesis also known as the “Hollywood Facial,” delivers gentle laser energy to help activate collagen production to give your skin a smooth and glowing appearance with virtually no downtime.

Laser Genesis helps to address:

- Diffused redness
- Fine lines
- Skin quality
- Scars
- Acne scars



Model. Not actual patient.



All Skin Types

Safe, controlled energy delivery across all skin types



Quick Treatment

Fit a treatment into your lunch break. 15-minutes or less.



Treatments for Face and Body

Popular for the face, neck, chest, and other areas of the body.

FREQUENTLY ASKED QUESTIONS

What can I expect before the treatment?

Before the treatment, you will be asked to remove your make-up or moisturizers and, in some cases, you may be asked to shave the area to be treated. Your physician will review and assist with all pre-treatment requirements.

Is the treatment painful?

Treatments can feel like progressive heating of the skin, which most patients find tolerable. Individual experiences may vary.

How many treatments are required?

A series of four to six Laser Genesis treatments are recommended to achieve an optimal outcome. Improvements are gradual and will be seen over a period of few treatments. Results will continue to improve for months after the last treatment. To sustain results over time, ongoing maintenance treatments are encouraged.

Are there any side effects or downtime?

Typically, you can wear make-up and return to normal activities immediately. Redness may be expected following treatment. Follow your providers post-care instructions and avoid the sun after treatment.