

WHAT YOU NEED

SafeStrip

FENTANYL TEST



Substance



SafeStrip Fentanyl Strip



Plastic bottlecap
2 TSP (10 ml)



OR



01 CRUSH AND SHAKE




02 MEASURE AND DILUTE



Fentanyl can be unevenly distributed within the sample. It is recommended to fill the scoop up with substance from multiple areas of the sample


02 30mg of substance required for accurate results

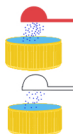

3 Scoops of any substance

1 Scoops of any substance

03 STIR AND TEST

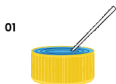

Do not submerge test strip beyond the water line



03



Positive



FYL
FYL

Negative



FYL
FYL

Test Control



Negative



Invalid

FOR MORE INFORMATION AND INSTRUCTIONS VISIT WWW.SAFESTRIP.US

Tests are for forensic use.