



JUNE MEDICAL Ltd, Innov8, Queen Alexandra Road, High Wycombe, Buckinghamshire, HP11 2GZ  
T.+44 1628 330010 W.junemedical.com Registered in England and Wales. Company Reg no.08612384

## Fascia Needle & Handle Sterilisation

Dear Customer,

Please find below sterilisation information regarding Neomedic (DIMA) reusable surgical Instruments.

The sterilisation information listed in the product IFU's are suggested guidelines only This is because each country has its own definition of what constitutes appropriate sterilisation and the IFU must cover them all. The manufacturer certificates state safe parameters from a product design, material and manufacturing perspective. To clarify for UK purposes please see the information below.

### **Sterilisation process**

All reusable surgical instruments manufactured by DIMA can undergo decontamination cycles through steam sterilisation as per the HTM 01-01 Part C.

### **Washing**

All reusable surgical instruments manufactured by DIMA can be processed through an automated washing machine and disinfectant as per the HTM 01-01 Part D. The passer needles and handles can also be cleaned in either Alkaline or PH neutral detergents.

### **Temperature**

The Neomedic surgical instruments are made of stainless steel, therefore higher temperatures than those suggested will not negatively affect these devices (i.e.: the material will not be degraded).

Please also see attached certification from DIMA

- (1.a) Steam steriliser declaration
- (1.b) Washer disinfectant sterilisation
- (1.c) Reusable instrument detergent