

### Splitfit Overview and Summary

Product	SplitFit
Product Code	JM7-510-50
Product Description	CO2 Gas connector
Replaces other product?	NO – Is an additional product to help eliminate risk
Reason for product (tick all that apply)	<input checked="" type="checkbox"/> Eliminates Risk <input checked="" type="checkbox"/> Increased patient Safety <input checked="" type="checkbox"/> Less operating time <input checked="" type="checkbox"/> Easier procedure
CE mark	<input checked="" type="checkbox"/> Yes
Cost	£1299
Ordering information	Order@junemedical.co.uk Phone: 01628 330010 Fax: 0872 1116710

### What is SplitFit?

SplitFit is a shuttle valve system for Laparoscopic Insufflation CO2 Gas

### Product Features

The shuttle Valve, high pressure hose and Yoke system is designed to be used with Laparoscopic insufflators to reduce the likelihood of running out of gas during laparoscopic surgical procedures.

### How is SplitFit used

SplitFit is connected to the gas inlet port on the Laparoscopic equipment. The 2 connectors are then connected to the Gas bottles. To optimize use, one of the bottles should remain off and switched on when the other bottle is empty. The red empty labels should be placed over the upright valve portion when it is empty. When time permits replace the empty bottle.

NOTE: Be sure to only turn 1 cylinder on whilst the other remains off to ensure optimal performance of the product.

Turning both bottles on at once will result in a NO GAS condition, which is what the product has been designed to prevent.

### Customer service and support

JUNE MEDICAL has a dedicated customer service team who can answer any pricing or shipping queries via email

[contact@junemedical.com](mailto:contact@junemedical.com) or telephone 01628 330010.

If you would also like to text or talk over video, please WhatsApp us on 07898 780418.

