

DMD

Digital Micropigmentation Device
"World class Micropigmentation / Permanent Make Up Device"



DMD FACT SHEET

Woori Mecha's

Micropigmentation technology (DMD Series) is
 Most advance technology yet!

Micropigmentation-sometimes referred to as "permanent make-up"- is a technique in which minute, metabolically inert pigment granules are implanted below the epidermis for cosmetic and/or corrective enhancement. This ancient Asian beautification practice has gained popularity around the world.

Micropigmentation is used to enhance facial features like eyebrows, lashes and lips; to improve conditions like scarring and vitiligo (uneven pigmentation); or to reconstruct the nipple/areola breast area.

EFFECT

Micropigmentation		Auto-MTS	
<ul style="list-style-type: none"> • Lips • Shading • Camouflage • Small Scars (Acne) enhancement • Eyebrows • Areola 	<ul style="list-style-type: none"> • Hairlines • Eyelines • Lashes 	<ul style="list-style-type: none"> • Acne scar • Collagen induction • Cellulite treatment • Skin rejuvenation • Improving wrinkle, Skin pigmentation, Skin scar, and Alopecia • Skin pore. 	<ul style="list-style-type: none"> • Hyperpigmentation • Skin tone • Face lifting • Stretch mark

BENEFITS

FEATURES	BENEFITS
Automatic	Very easy to use, and also safe to use since the needle tips are hidden inside the guide.
UANV Technology	DMD is much quieter and smoother in operation than the regular micropigmentation device because, of our unique UANV (Ultra Anti Noise & Vibration) technology. You won't even know if the device is on!
Ergonomic design	Ergonomically-designed DMD can help alleviate the discomfort caused by Repetitive Strain Injury (RSI), Carpal Tunnel Syndrome (CTS)
Safe using disposable needle tips	It is safe to share the hand-piece with other patient. Be sure to change sterilized safety disposable needles every use..
Economical	DMD provides dual function with one pen. Permanent make up and Auto-MTS
Less Pain	The needle tips are inserted to your skin perpendicularly. Therefore, pain is minimized.
Adjustable Needle length	The needle can be adjusted (From 0.1mm~2.0mm) in to different sizes depends on the areas or the purpose of the treatment.
Efficient & Effective	It is more efficient & effective to treat narrow areas such as nose, around eyes, or mouth, this hand-piece device allows you to treat areas in whichever angles you want to.

IMPROVEMENTS



SPEED | DEPTH | PINS


Region	Speed	Length	Pins
Hairline	Over 6000RPM	1.2mm	4,6
Eyeline	4000RPM	0.8mm~1.0mm	1,3,5
Eyebrow	5000RPM	0.8mm~1.0mm	4,6
Scar	Over 6000RPM	1.4mm	1,MTS
Lip	5500~6000RPM	0.8mm~1.0mm	9M
Aeola	5000RPM	1.2mm	1,3

BEFORE AND AFTER



HOW TO USE(DMD-H)

Select speed steps from 1~5 according to your needs. After turning on power, Increase speed by pressing the button. Turning power off during procedure, device remembers last setting. When removing the power adapter, setting value begin at step 1.

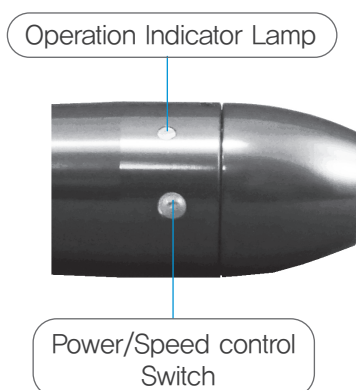


Step	Speed
Step1	4400RPM
Step2	5000RPM
Step3	5500RPM
Step4	6000RPM
Step5	6400RPM

MANUAL MODE & USER MODE (DMD-P)

Manual Mode: From 4500RPM ~ 6400RPM in total of 6 steps.

User Mode: From 4000RPM ~ 6400RPM in total of 11 steps.

Select speed steps according to your needs.

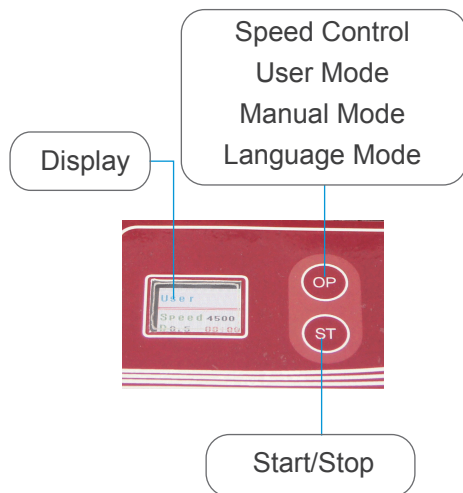

Manual Mode

Mode1	4500RPM	Mode4	5500RPM
Mode2	5000RPM	Mode5	6250RPM
Mode3	5250RPM	Mode6	6400RPM

User Mode

Step1	4000RPM	Step7	5500RPM
Step2	4250RPM	Step8	5750RPM
Step3	4500RPM	Step9	6000RPM
Step4	4750RPM	Step10	6250RPM
Step5	5000RPM	Step11	6400RPM
Step6	5250RPM		

SPECIFICATION

DMD-P		
Weight	Hand Piece	69.5g
	Control Box	300g
Input	AC220V/60Hz	
Output	DC 6V	
Dimention	Hand Piece	128.35Xø19.5mm
	Control Box	148.5X61X104.5mm

DMD-H	
Weight	73.8g
Input	AC220V/60Hz
Output	DC 6V
Dimention	133.5Xø19.5mm



WOORHI
MECHATRONICS CO., LTD.

#301, Daekwang Plaza, 1027-15, Hogae-Dong, DongAn-Gu,
Anyang-City, Kyunggi-Do, Korea #431-848
Tel : +8231-457-5032 / Fax : +8231-457-5031
E-mail : saleswrmc@gmail.com
Http : www.woorhimecha.com