

# DMD-S

Digital Micropigmentation Device

*"World class Permanent Make Up / AutoMTS Device"*



## DMD FACT SHEET

Woori Mecha's

Micropigmentation technology (DMD Series) is

Most advance technology yet!

Micropigmentation-sometimes referred to as "permanent make-up"- is a technique in which minute, metabolically inert pigment granules are implanted below the epidermis for cosmetic and/or corrective enhancement. This ancient Asian beautification practice has gained popularity around the world.

Micropigmentation is used to enhance facial features like eyebrows, lashes and lips; to improve conditions like scarring and vitiligo (uneven pigmentation); or to reconstruct the nipple/areola breast area.

### EFFECT

| Micropigmentation                |             | Auto-MTS  |                     |
|----------------------------------|-------------|---|---------------------|
| • Lips                           | • Hairlines | • Acne scar   | • Hyperpigmentation |
| • Shading                        | • Eyelines  | • Collagen induction  | • Skin tone         |
| • Camouflage                     | • Lashes    | • Cellulite treatment   | • Face lifting      |
| • Small Scars (Acne) enhancement |             | • Skin rejuvenation   | • Stretch mark      |
| • Eyebrows                       |             | • Improving wrinkle, Skin pigmentation, Skin scar, and Alopecia |                     |
| • Areola                         |             | • Skin pore.  |                     |

## BENEFITS

| FEATURES                          | BENEFITS  |
|-----------------------------------|---|
| Automatic                         | Very easy to use, and also safe to use since the needle tips are hidden inside the guide.   |
| UANV Technology                   | DMD is much quieter and smoother in operation than the regular micropigmentation device because, of our unique UANV (Ultra Anti Noise & Vibration) technology. You won't even know if the device is on! |
| Ergonomic design                  | Ergonomically-designed DMD can help alleviate the discomfort caused by Repetitive Strain Injury (RSI), Carpal Tunnel Syndrome (CTS)   |
| Safe using disposable needle tips | It is safe to share the hand-piece with other patient. Be sure to change sterilized safety disposable needles every use..   |
| Economical                        | DMD provides dual function with one pen. Permanent make up and Auto-MTS   |
| Less Pain                         | The needle tips are inserted to your skin perpendicularly. Therefore, pain is minimized.  |
| Adjustable Needle length          | The needle can be adjusted (From 0.1mm~2.0mm) in to different sizes depends on the areas or the purpose of the treatment.   |
| Efficient & Effective             | It is more efficient & effective to treat narrow areas such as nose, around eyes, or mouth, this hand-piece device allows you to treat areas in whichever angles you want to.                           |

## IMPROVEMENTS



## DEPTH/PINS

| Region   | Length      | Pins  |
|----------|-------------|-------|
| Hairline | 1.2mm       | 4,6   |
| Eyeline  | 0.8mm~1.0mm | 1,3,5 |
| Eyebrow  | 0.8mm~1.0mm | 4,6   |
| Scar     | 1.4mm       | 1,MTS |
| Lip      | 0.8mm~1.0mm | 9M    |
| Aeola    | 1.2mm       | 1,3   |

## SPEED

| Step  | Speed   |
|-------|---------|
| Step1 | 5000RPM |
| Step2 | 6000RPM |
| Step3 | 7000RPM |

## BEFORE AND AFTER



## SPECIFICATION

| DMD-S     |                    |
|-----------|--------------------|
| Weight    | 70g                |
| Input     | AC100~240V/50~60Hz |
| Output    | DC 5V 1A           |
| Dimention | 137 X Ø20          |



**WOORHI**  
**MECHATRONICS CO., LTD.**

#301, 48,LS-ro 91beon-gil,DongAn-Gu,  
Anyang-City, Kyunggi-Do, Korea #431-848  
Tel : +8231-457-5032 / Fax : +8231-457-5031  
E-mail : saleswrmc@gmail.com  
Http : www.woorhimecha.com