

haloPocket

Personal Portable Halogenerator


Information Guide

haloPocket arrives charged, sanitized, and ready to use.

What's in the Box



haloPocket Parts

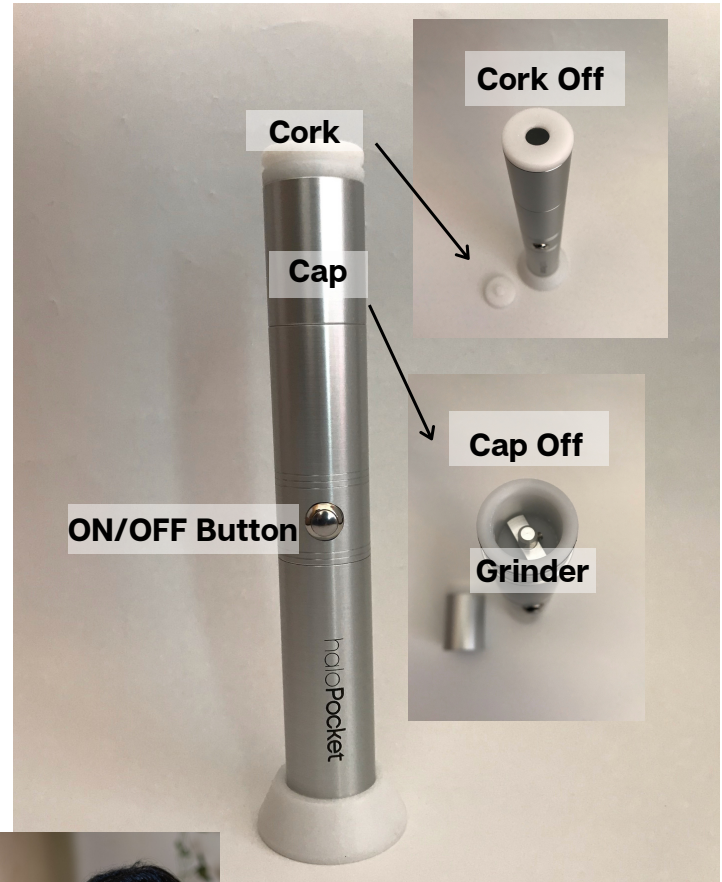


Using haloPocket

1. Open the cap by pulling up with a gentle twist.
2. Add 4 measuring spoons of salt in the cavity when beginning with an empty grinder. This amount of salt will last several sessions. Only pharmaceutical grade salt is to be used in this device.
3. Push the cap back in place.
4. Press the ON/OFF button for 10 seconds to grind the salt. Do this 3 times.
5. Remove the cork by separating the white disks.
6. Point the top of the haloPocket toward your nose/mouth area, slightly tilted, 2-3 inches away.
7. Press the ON/OFF button to activate the flow of salt air and breath in.
8. Release the ON/OFF button on the exhale.
9. Repeat for a total of up to 12 times.
10. Replace cork when finished and store on stand or in carrying case.

For subsequent sessions, add 1 measuring spoon of salt. The amount of salt in the grinder should not exceed the blade height.

Periodic cleaning of the blade area cavity should be carried out using the brush provided. Do not clean with any liquid.

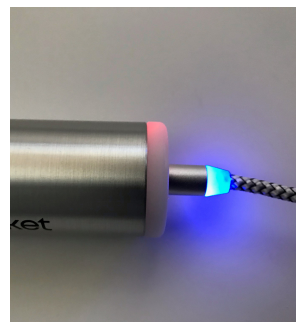

Charging haloPocket

A rechargeable lithium ion battery is pre-installed in the haloPocket. Connect the charging cable to a USB plug and connect the magnetic end to the base. A full charge will take approximately 90 minutes. When the charging cable is attached, the unit is non-operational. Only the supplied charging cable should be used.

While charging, the red light will be illuminated on the haloPocket and light will turn blue when fully charged.

Warranty

2 year warranty - parts and labor from date of purchase from the manufacturer.