



Congratulations on buying the NitNipper

What's in the BOX

One NitNipper Electric Brush
One Ultraviolet Torch
One NitNipper Comb

NitNipper™ Instructions

The system works in 3 easy steps

1

Use the enclosed UV torch to identify the location of Lice & Nits and/or infestation

2

There are three settings on the brush

Warm/Test- Not Used

(L)- low setting for young kids up to 9 Years old

(H) –High setting for children 10 Years old and above

Select the appropriate setting for the child, then slowly comb the hot air brush gently through the hair (approximately 5 seconds/cm of hair). The specially designed hot prongs actively kill the Lice instantly / the hot air dries out the eggs (Nits) and prevents them coming back. Gently pull the ears forward to access the hotspot areas. Keep doing this process for approximately 20-30 mins. The brush is set at a temperature best designed to kill lice/nits

3

Finally use the enclosed nit comb to remove the dead Lice & Nits leaving the hair free of the annoying creatures.

Enjoy a Lice & Nit free Life!

How it works

The Ultraviolet light detects and helps to impede the Nits growth
Development

The Electronic Brush is regulated at a specific temperature and airflow rate to dehydrate and kill active Lice & Nits

The Comb removes dead/dying Lice and Nits from the hair strand.



REGISTERED ADDRESS;
FSP LTD
Wellington, Somerset, TA21
www.NitNipper.info

Pesticide & Chemical
Free Treatment
Patent Pending
110-250V 50/60HZ



THE  **NIPPER™**



IMPORTANT

Treatment **MUST** be administered by an adult.

It is recommended to treat all family members even if not showing signs of irritation

It is recommended that the above process is repeated at least once each day or as necessary. It is also recommended that the child's pillowcase is washed to a minimum of 60 degrees centigrade to kill any Lice/Nits on the fabric and to avoid being reinfected

Rinse the metal nit comb (NOT BRUSH) with hot/boiling water after each use

DO NOT USE IN THE SHOWER / BATH OR ON WET HAIR.

CAUTION: THE BRUSH CAN BECOME HOT TO TOUCH AFTER PROLONGED USE, DO NOT USE MORE THAN 30MINS AT A TIME

Product disclaimer: Always read the label. Follow the directions for use. If symptoms persist, talk to your healthcare professional.

Model no.Q18

