

How To Use the E-Z Graph® Wound Assessment System



STEP 1

Position the **E-Z Graph®** with the dot centered in the wound. Trace the wound margins using a **BLUE** marker, erythema using a **RED** marker, and eschar using a **BLACK** marker.




STEP 4

Complete the top part of the **E-Z Graph®** by filling in or circling appropriate information using the black marker.



STEP 2

Peel away and discard the soiled plastic backing.

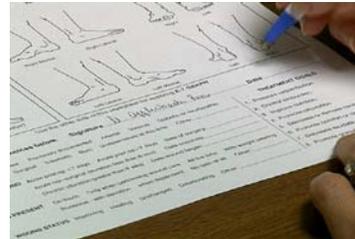

STEP 5

Complete the back page of the worksheet using an ink pen.



STEP 3

Place the self-adhering traced portion of the graph onto the front side of the worksheet.

STEP 6

Upload to electronic health record.
Easily scans and faxes.