


How to Test Step by Step with iPhone

1. Open SoberSense Application



2. Enter Login Credentials

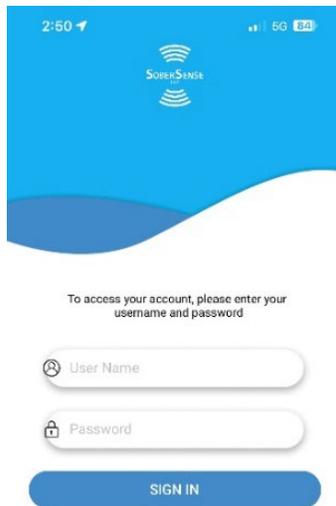

Username: firstnamelastname

Password: sober

All Lowercase with No Spaces

3. Turn on Device by holding the gray button down for about 3 seconds. You will not need to touch the button again until after your session has been submitted. Hold again for 3 seconds to turn it off.



How to Test Step by Step with iPhone

4. You can now select your session. You will do this by tapping on the green session



5. Tap the middle of the screen to connect the breathalyzer with the smart phone.




You will **NEVER** have to pair the device with a code. Tapping on the middle of screen will start the connecting process.

6. Tap on “Take Test” at the top right.



How to Test Step by Step with iPhone

7. You will now see a countdown on the top right of the screen. Once it is complete it will say “Blow Now”

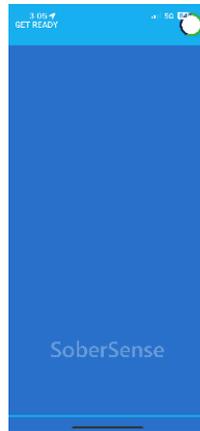

8. Blow steadily for about 6 seconds until you feel a click inside the breathalyzer.



9. Once completed you will see a submit results button. Please tap on that to submit.

10. Tap on the top right refresh arrows to confirm that the session has disappeared.