

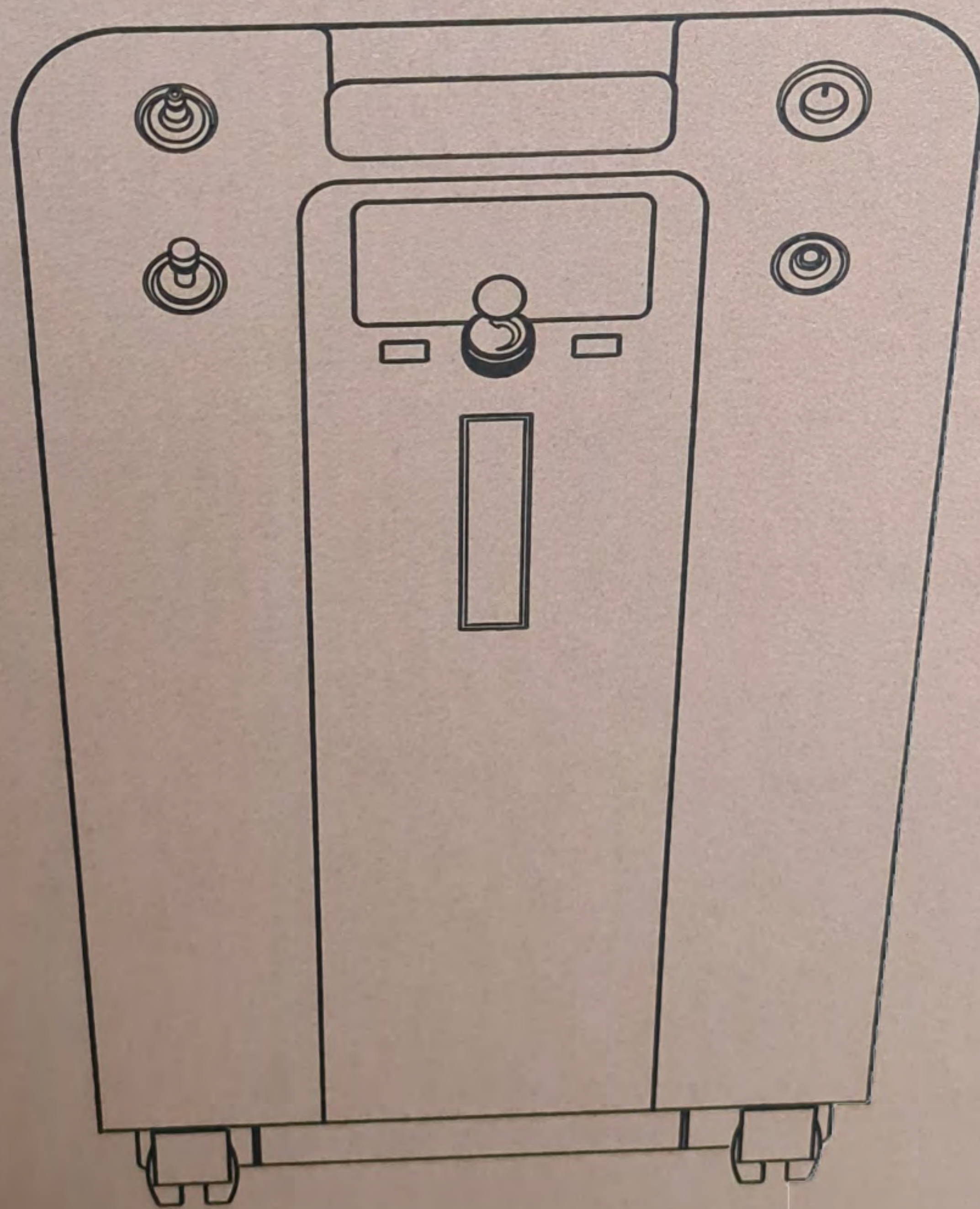


# OXYGEN CONCENTRATOR

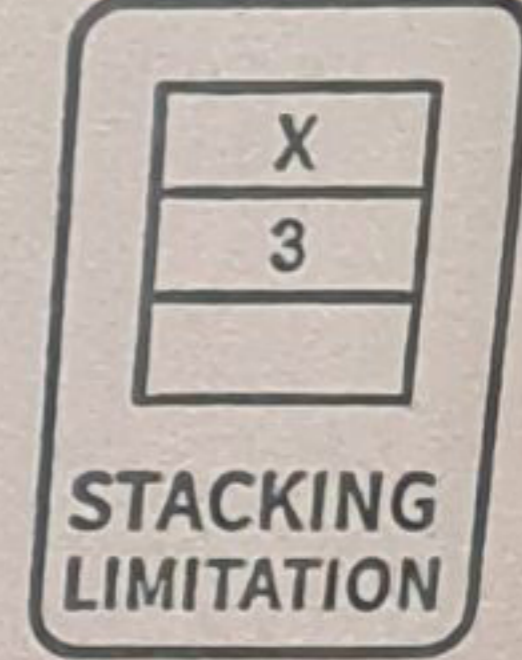

The above product picture is only for reference,  
specifications are subject to the actual product

1. You will receive the carton box

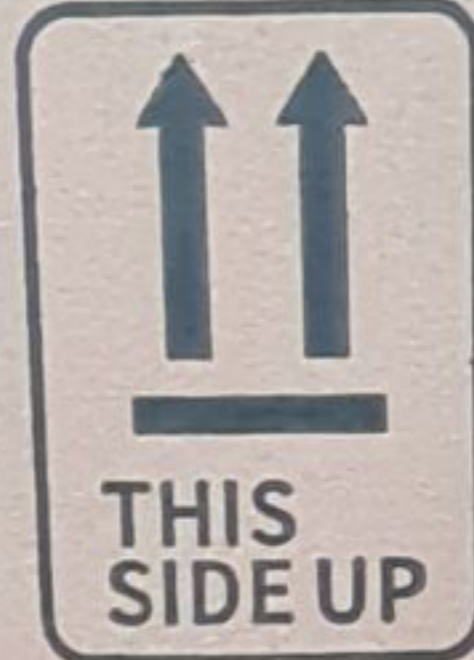

PRODUCT SPECIFICATION	
PRODUCT NAME	OXYGEN CONCENTRATOR
PRODUCT MODEL	
OXYGEN FLOW RATE	(1-5LPM)
OXYGEN PURITY	93%±3% (1-5LPM)
POWER SUPPLY	PLEASE REFER TO THE NAMEPLATE
POWER CONSUMPTION	≤400W
SIZE	402mm×318mm×642mm
NET WEIGHT	15.5 KG
CONDITIONS FOR TRANSPORTATION AND STORAGE	AMBIENT TEMPERATURE: -20°C ~ +55°C
	RELATIVE HUMIDITY: ≤95%
	ATMOSPHERIC PRESSURE: 50kPa~106kPa

PLEASE READ MANUAL BOOK OR UNDER THE GUIDANCE OF MEDICAL PERSONNEL BEFORE USE, FOR OTHER DETAILS, PLEASE REFER TO THE MANUAL BOOK.



2. This is the rear side.



3. Open it Carefully and remove the small box and keep it aside



4. Small box will have power cable, one nasal cannula, one nebulizer kit, one user manual with extra filters

5. This is the power cable



6. One Nasal Cannula for connecting the concentrator output to user



# 7. One Nebulizer kit for atomization



8. One user manual with black colour primary filter and white colour secondary filter for future use





oxygen outlet

power on /off



Reset button

Nebulizer Port

9. Take out the main unit and remove the wrapper

The image shows the rear view of a white oxygen concentrator. The top is covered with clear plastic wrap. A red line points from the text 'Humidifier bottle' to a white rectangular component on the top right. Another red line points from 'Secondary Filter' to a white rectangular component on the left side. A third red line points from 'Primary Filter Housing' to two vertical grilles at the bottom. A blue label is visible at the bottom center.

Humidifier bottle

10. This is the rear end. You will see the two slots of primary filter, above the left primary filter, you will see the secondary filter housing. Top right you will see the humidifier bottle and its placement.

Secondary Filter

Primary Filter Housing


11. The Power Button on the front will be protected by a plastic seal. Remove it and keep it aside.



12. This step is very important, tilt the device and you will see two velcro straps. Unhook it and then only you should switch on. This is to protect the compressor during transport.



13. Fill the Humidifier bottle with purified water or distilled water between minimum and maximum markings.

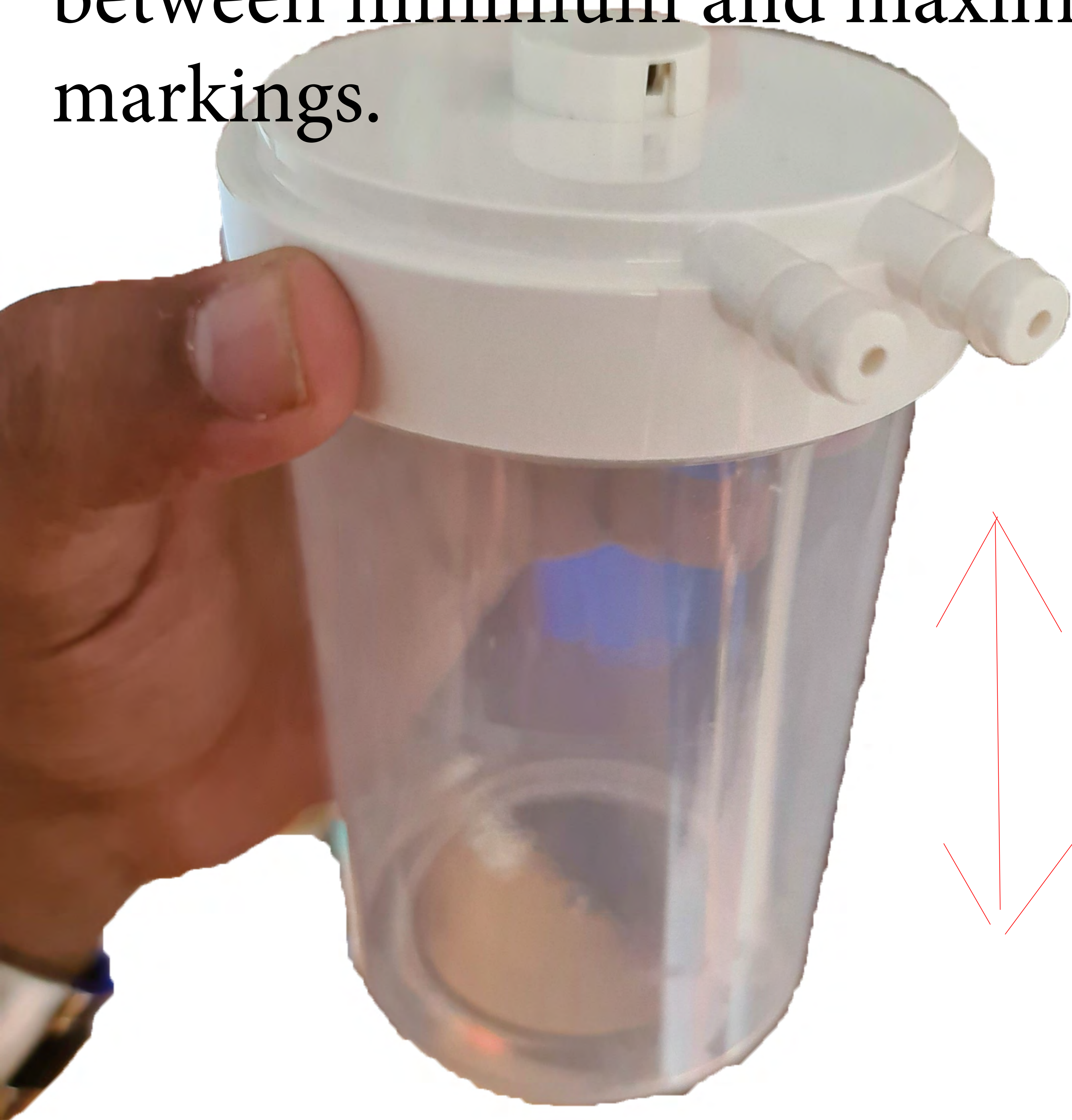

# 14. Switch on the machine



# 15. Side view





TIME-



TIME+

16. Main display indicating Hours & Minutes used, Digital Flow Rate, Oxygen Purity. You can use timer buttons to schedule timer. Adjust the knob to increase / decrease flowrate



# 17. Rear View with Humidifier



18. Clean the primary filters once a week. Replace it when fully dusty and clogged.



**Oxygen Concentrator**  
Model: 2407204  
Flow Capacity: 5LPM  
Oxygen Purity: 93%  
Max. Recommended Flow: 5LPM  
Oxygen Concentrator  
Flow Capacity: 5LPM  
Oxygen Purity: 93%  
Max. Recommended Flow: 5LPM  
Production Date: 20200120  
Production Date: 20200120

19. Remove secondary filter and dust it once in 2 weeks. Replace it when necessary and when it is fully clogged.



20. Best Purity of Oxygen checked with industrial oxygen analyzer.



21. Book your orders on  
[www.entrusted.in](http://www.entrusted.in) & call us at  
7397449991