


OXYGEN CONCENTRATOR

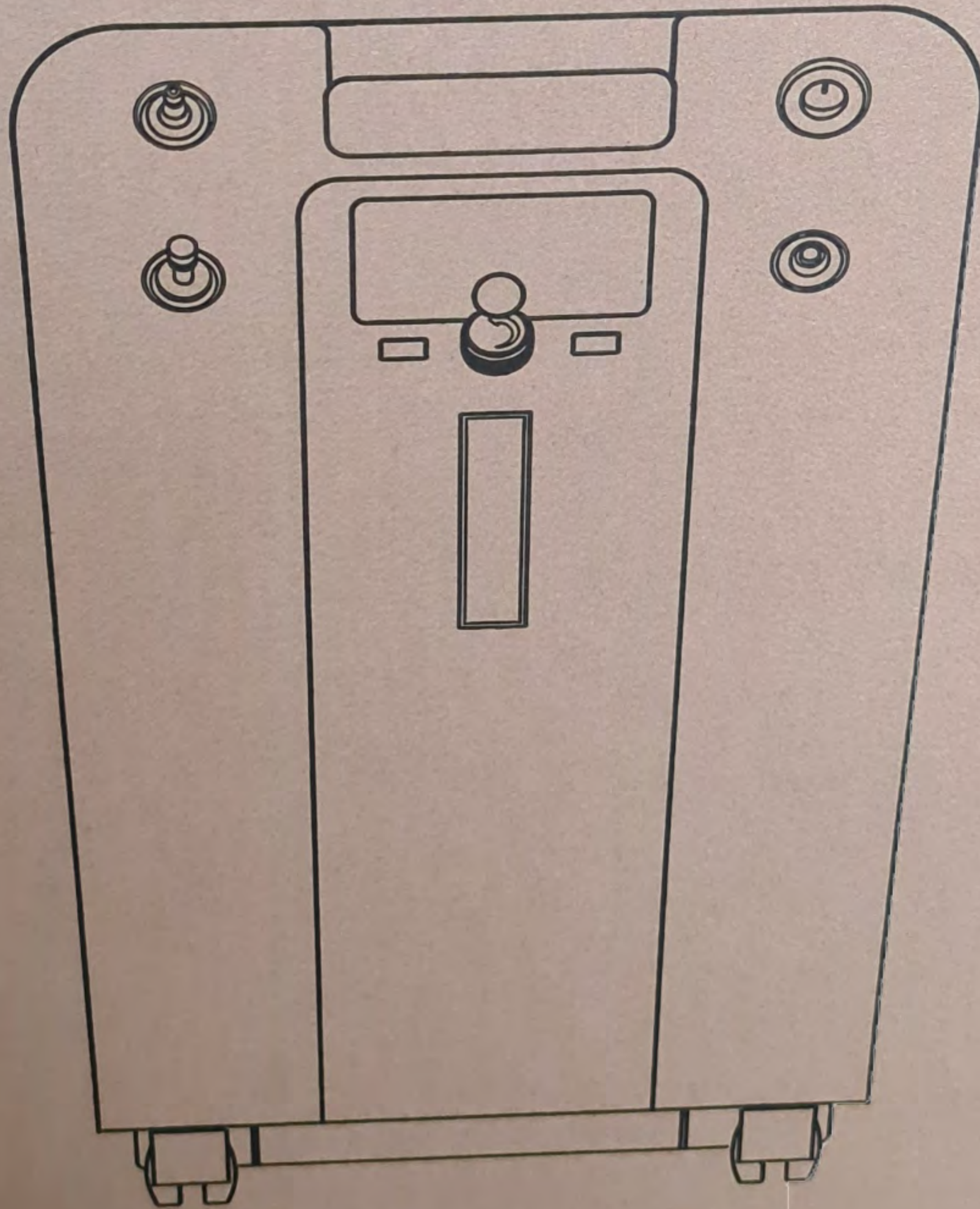

The above product picture is only for reference,
specifications are subject to the actual product

1. You will receive the carton box

2. This is the rear side.



3. Open it Carefully and remove the small box and keep it aside



4. Small box will have power cable, one nasal cannula, one nebulizer kit, one user manual with extra filters

5. This is the power cable



6. One Nasal Cannula for connecting the concentrator output to user



7. One Nebulizer kit for atomization



8. One user manual with black colour primary filter and white colour secondary filter for future use



oxygen outlet

power on /off



Reset button

Nebulizer Port


9. Take out the main unit and remove the wrapper

Humidifier bottle

10. This is the rear end. You will see the two slots of primary filter, above the left primary filter, you will see the secondary filter housing. Top right you will see the humidifier bottle and its placement.

Secondary Filter

Primary Filter Housing

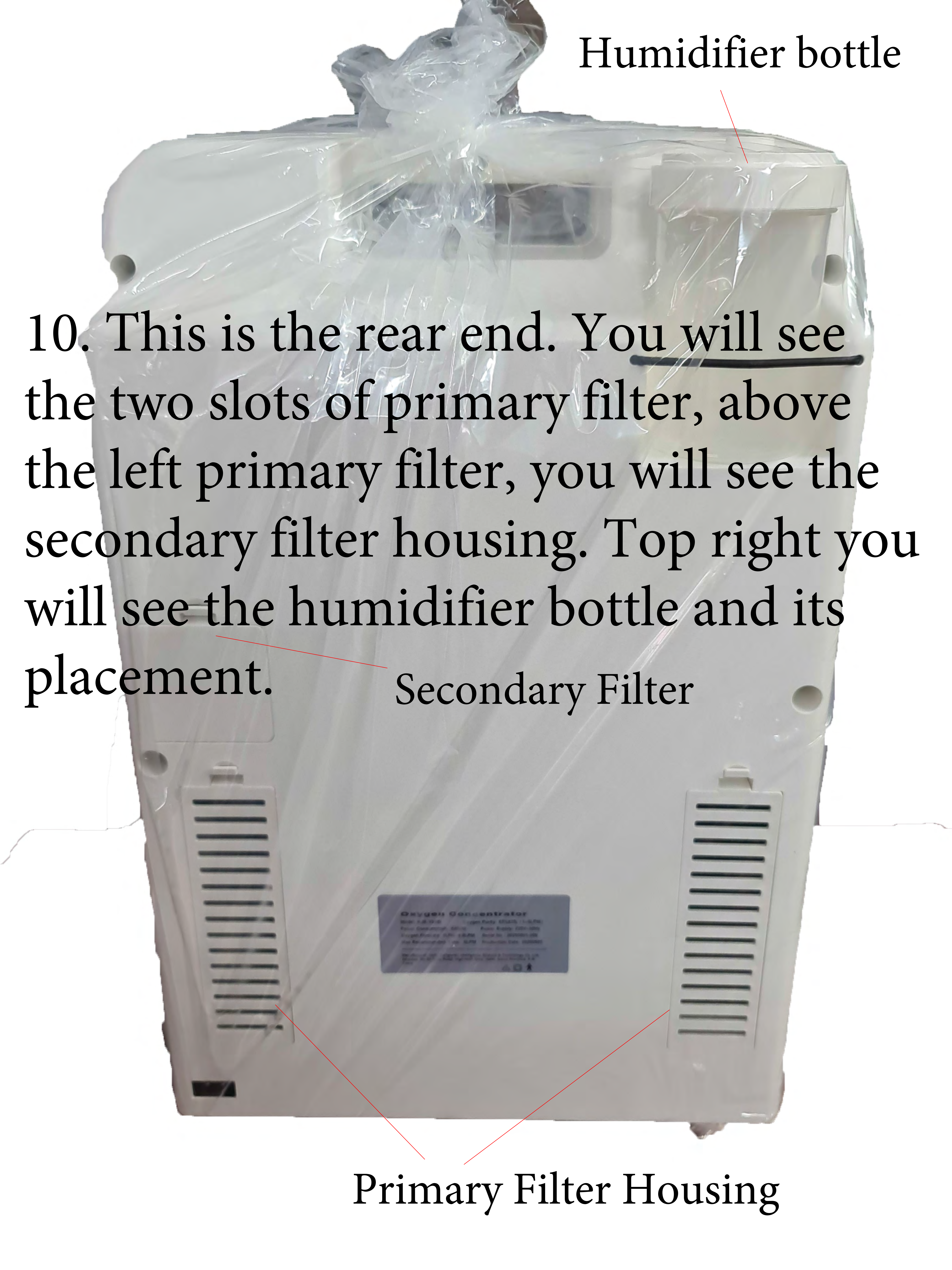

11. The Power Button on the front will be protected by a plastic seal. Remove it and keep it aside.



12. This step is very important, tilt the device and you will see two velcro straps. Unhook it and then only you should switch on. This is to protect the compressor during transport.



13. Fill the Humidifier bottle with purified water or distilled water between minimum and maximum markings.



14. Switch on the machine



15. Side view





TIME-



TIME+

16. Main display indicating Hours & Minutes used, Digital Flow Rate, Oxygen Purity. You can use timer buttons to schedule timer. Adjust the knob to increase / decrease flowrate

17. Rear View with Humidifier



18. Clean the primary filters once a week. Replace it when fully dusty and clogged.



19. Remove secondary filter and dust it once in 2 weeks. Replace it when necessary and when it is fully clogged.



20. Best Purity of Oxygen checked with industrial oxygen analyzer.



21. Book your orders on
www.entrusted.in & call us at
7397449991