



Use/Precautions

- Consult your health care professional concerning your condition and the use of this brace.
- The MyQuest soft AFO brace provides dorsiflexion through the swing phase to assist in walking and other activities, but is not recommended for use that may require precise foot and toe control.
- New England Foot Drop Center will NOT be responsible for any damage to the shoe resulting in attachment, removal or use of the MyQuest soft AFO brace.
- New England Foot Drop Center cannot control the design, materials, workmanship or condition of the footwear worn by the purchaser. Even in cases when a customer's shoe appears to be appropriate for the MyQuest soft AFO brace, we cannot know for certain that the interface is safe initially, or that it will remain safe over time. Therefore, we are not responsible for any damage, injuries or product failure resulting from failure of the shoe material where the MyQuest brace has been attached.
- If used with waterproof footwear, the shoe may no longer be waterproof. If you are unsure about the appropriate shoes for the MyQuest brace, refer to the shoe guide provided or contact us at info@myquestorthosis.com.
- Before using the MyQuest, check to ensure the hardware, plastic clips, elastics and velcro strap are attached correctly and securely. When attaching clips, do not use excessive force. If positioned properly per the instructions, clips will connect easily and securely. Forcing clips together can cause them to break. We are not responsible for broken plastic clips due to misapplication.
- The amount of continuous time the brace can be worn comfortably will vary by individual. New users should begin wearing no more than 4-6 hours for the first couple of days and then extend longer each day as comfortable. If discomfort occurs, check elastic cord tension. If discomfort persists, temporarily discontinue use of the brace.
- The MyQuest AFO Brace and components must not be used for purposes other than those described in this product's instructions. Any alterations of a component, or any application other than provided in the instructions is a misuse or misapplication of the product. New England Foot Drop Center is not liable for injuries or damages from misuse or misapplication of the device.
- The fabric ankle wrap and components are subject to normal wear and tear. The elastic cords and velcro strap should be inspected periodically for replacement. Visit www.nefootdropcenter.com to order replacement parts.
- The MyQuest AFO package includes small parts which should be kept away from small children. This product is not a toy.

Care

- Hand wash with warm water and mild soap. Pat dry and let air dry before wearing. Do not use high temperatures to wash or dry. High heat can melt or deform the plastic parts.