LOWER BACK BRACE

IDEAL BACK CARE

PROTECT TO AVOID SECOND INJURY WHEN:

- long time standing
- long time lifting
- long time seating
- long time riding
- after exercise

BENEFITS:

- breathable materials
- adjustable compression
- soft and flexible support
- 6 plastic correct lumbar
- stay in place during movement

American design

Made in China

AVESTON®
AS

My name is Bogdana, and this is my family. Have you ever suffered from the back pain and spasms so that you couldn't sleep at night? That's what I experienced after giving birth to my twins and taking care of them. My chiropractor told me that after pregnancy my back muscles became weak and I had to wear a support belt. I did not manage to find a belt with strong support, thin and nifty at once.

Thus I invented a new design, and having established the women's business; I decided to sell the belts under the trademark AVESTON. I work directly with production, and I check personally every lot and every unit. FDA certifies my back support belt. In this way, I'd like to give you the best price for the best quality. Thank you very much for choosing the AVESTON back support belt.

SO, THIS IS AN INSTRUCTION OF HOW TO PUT ON THE AVESTON BACK SUPPORT BELT

Step 1. Take the upper part of the belt. The upper part is completely flat.

Step 2. Unstrap the compression bands, put the belt around the required area of the back.

Step 3. Fasten the belt around the desired area, regulating the necessary force and girth direction.

Step 4. Pull ahead and fasten side bands to achieve the compression you want.

Step 5. Take pleasure in easy work.

As the belt is quite rigid, we recommend to wear it over 100% Cotton T-shirt. Due to the considerable difference in waistline and rib line the belt during movement or while seating may go up. For such cases, we recommend to put on the belt on a thick jacket.

The belt may sink into ribs. In this case, we recommend wearing the belt only over the clothes.

Do not wear the belt for more than 8 hours; the back muscles must work and have natural load. Belt has a phyletic function; it reduces pain, but it does not cure a source of trouble.

In the case of skin irritation or any other discomfort stop wearing the belt.

The belt is not intended for pregnant women and kids. But it helps mothers to recover after the pregnancy.

Never wear the belt before sleep.

The Aveston belt is intended only to be worn in the lower region of the vertebral column.

Always consult your doctor before using the belt.

CONTACTS:
AVESTONGROUP@GMAIL.COM

To reward our valued and loyal customers we are giving you

30%

discount for your next purchasing.

- 1) Search Amazon for the keywords **back support** o **back brace**
- 2) Find the model branded AveSton
- 3) Enter your Coupon **AVE30OFF**

Voice your opinion.

You like this belt and want to share your experience, please let the world know!

Create a product review on Amazon for other customers! ★★★★★

BELT FUNCTIONALITY:

- Decreases load on spinal column.
- Provide strong support to weak muscles of the back.
- Unique construction keeps the back in direct position.
- Belt prevents dislocation of the vertebra.
- Reduces pain and accelerate rehabilitation.
- Prevents spasms and pain in the back.

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PRODUCT CERTIFICATION:

The **AVESTON** trade mark is under registration now.

All rights reserved.

**CAUTION: IN CASE OF ANY SKIN ALLERGY OR ITCHINESS,
PLEASE DISCONTINUE USE AND CONSULT WITH A DOCTOR.**

To understand what size is needed, you need to measure the volume of your body at the waistline. Size Guide

RECOMMENDED SIZE	WAISTLINE
S	28-33"
M	32-38"
L	37-44"
XL	45-52"

Composition: 50% Polyester, 25% Rubber
15% Nylon, 10% Plastic

Do not put into the washing machine
Air-dry
Do not iron
Gently wash by hand using warm water
and with mild detergent

Manufacturer
Avestongroup.com

THE BATCH NUMBER
160890