



## **Cleaning & Calibration**

### **STADIOMETERS/LENGTHBOARDS & PARTS**

Avoid products containing alcohol. We recommend a mild sanitation wipe such as Clorox bleach wipes or a hydrogen peroxide product such as Clorox or PDI hydrogen peroxide germicidal disinfecting wipes. The more harsh detergents, sprays and products containing alcohol can cause a cloudy haze in the magnifier and damage to pin holes.

[NOTE: Our cleaning recommendations in light of COVID-19 remain the same. The products we recommend are on the EPA's Registered Antimicrobial Products for Use Against Novel Coronavirus SARS-CoV-2, the Cause of COVID-19 ([https://www.epa.gov/sites/production/files/2020-03/documents/sars-cov-2-ist\\_03-03-2020.pdf](https://www.epa.gov/sites/production/files/2020-03/documents/sars-cov-2-ist_03-03-2020.pdf))

When cleaning with a bleach solution, we recommend mixing:

- 5 tablespoons (1/3rd cup) bleach per gallon of water or
- 4 teaspoons bleach per quart of water

### **CARRYING CASES**

We recommend the same for cleaning our lengthboard carrying cases made of marine vinyl. Although there are no alcohol/chemical damage restrictions on the cases, we still recommend a mild antibacterial germicidal wipe. We can only recommend that you use a product listed on the EPA's Registered Antimicrobial Products for Use Against Novel Coronavirus SARS-CoV-2, the Cause of COVID-19]

### **CALIBRATION**

Our length boards and stadiometers are calibrated during manufacture and do not require additional calibration.