


When Should I Replace My NopoGuards?

NopoGuards can be sterilized and reused according to the Instructions For Use (IFU). As with other reusable devices and equipment in your clinical environment, NopoGuards may experience some cosmetic degradation or suboptimal performance over time that would necessitate replacement. The exact life expectancy of your NopoGuards depends on the use, care, and processing of the device.

Below are some examples of NopoGuards that have undergone regular clinical use, having been exposed to 250-500 cleaning, drying, and sterilization cycles in accordance with the IFU. As shown in the images, NopoGuards may experience loss of clarity in discrete areas, small cracks (visible within the material itself), or even breaks in areas of repeated stress. All of the examples shown are still functional and safe for use but do not perform as new and should be replaced.

 It is important to thoroughly inspect your NopoGuards and ensure they are functioning properly prior to each use (DPL assumes no liability in the case of disregard).

