



- Fine Lines and Wrinkles
- Elasticity & Firmness
- Skin Tone Evenness & Vibrancy
- Skin Texture
- Skin Lift & Tighten
- Hyperpigmentation
- Oily/Congested Skin
- Enlarged Pores
- Advanced Signs of Aging
- Acne & Aging Skin

SAFETY INFORMATION

Thank you for choosing our products! This user instruction is specially designed for the safety of you and others. Please read the instructions carefully before installing or using the machine and follow them when using.



COMPARISON BEFORE AND AFTER USE



Back Acne
After 2 Treatments

Skin Hydration Significantly Increased
After 1 Treatment

Facial Acne
After 2 Treatments



Sun Damage Improved
After 3 Treatments

Skin Hydration Significantly Increased
After 1 Treatment

Facial Acne
After 2 Treatments

1.1 Electrical Safety

- Before installing the machine, make use that your local electric network complies with the voltage and frequency indicated on the tag on the rear panel. Otherwise, the machine may be damaged in case of excessive current or voltage rise.
- Never touch the machine with damp or wet hands or feet.
- It is dangerous to change or try to modify the function of this machine in any way.
- Do not attempt to operate this unit if the power cord has been frayed or broken
- Do not spill water or other liquids into or on to your unit.
- During long periods of non-use, disconnect the unit's main power.

1.2 Safety Rules When Installing

1.3

- Place the machine on a stable and secure base. It should not be subjected to any jolts or vibrations.
- Never expose the unit to too high ($>40^{\circ}\text{C}$) nor too low temperature ($<5^{\circ}\text{C}$).
- NEVER EXPOSE THE UNIT TO RAIN OR ANY OTHER HUMIDITY.
- DO not expose the unit to direct sunshine or install the unit close to a source of heat (central heating, radiator...); otherwise the screen may be damaged.
- Do not cover or block the ventilator behind the unit.

1.4 Safety Rules When in Use.

- Well clean areas to treat before treatment.
- Never touch the machine with damp or wet hands or feet.
- Never place the cavitation handpiece on the liver, kidney area.

INTRODUCE OF THE FUNCTION



1, Hydro dermabrasion / diamond dermabrasion

Hydradermabrasion (Hydra Facial)- the latest in skincare technology. Hydradermabrasion utilizes the natural healing powers of water and oxygen to effortlessly exfoliate the skin without the use of hard crystals or abrasive textured wands, generating deeply hydrated and healthier looking skin.

Application

1 Rejuvenate sun damaged skin-face, neck, shoulders, back, arms and legs.

2 Reduce age spots

3 Minimize of blotchy skin coloring

4 Reduce acne and superficial scars from past injury

5 Remove blackheads and white heads

6 Reduce oily skin

7 Improve overall skin health

2, Spray treatment Mechanism of spray oxygen Operating steps

Mix 2 ~ 3 drip SPA essential oil diluted with purified water into small cup of the sprayer, water should be about 2 / 3 of the cup, and then tighten the lid tightly. Oxygen sprayer can only with liquid products, other states product will cause pinhole jam.

Beautician hold the spray gun, press the control button to release oxygen, And pull backwards to spray water mist.

Spray oxygen from top to bottom starting from the forehead to the neck, careful not to spray into the eyes or nose, in the spray oxygen process, the beautician can appease, lift up, stretch wrinkles with the other hand, more helpful for nutrition products penetrate the skin, strengthen skin elasticity, spray rate should depend on the guests' absorb state of oxygen.

Cautions

The product should be liquid when using oxygen sprayer.

Avoid spraying into mouth or nose when spraying oxygen.

Clean the small cup and the pipe inside the sprayer every time use it, to avoid jam.

Clean method:

pour the purified water into the small cup, and press the control button and then pull back towards.

1. Supply nutrients and moisture for the skin rapidly, smooth skin, lightening stains, improve chlorosis, dull conditions, make skin rosy, white, transparent, tender, tightening indeed, with special spa oils will be better.

3, *Vacuumpen*

Vacuum pen is use vacuum/suction to suck the blackheads from the pores, makes our pores more cleaner. And our vacuum pen is in new technique that can be massage our skin during working , because it is not always in suction , it will suck on and off, good for lymphatic drainage and active our cells .

4, *Bio micocurrent*

Microcurrent is a safe and effective technology for clients who wish to attain/maintain a healthier and younger looking appearance.

Benefits:

Non-invasive facelift

Reduce and eliminate fine lines and wrinkles Improve skin circulation

Improve hyperpigmentation and acne Increase product penetration

5. *High frequency*

High frequency electric device can produce high frequency microcurrent via glass electrode tubes . it can used on face and hair . Good for acne treatment and hair regrowth.

6, *Microcurrent*

Microcurrent is use the micro current to do the face lifting and skin tightening treatment. It also use around eyes for eyes lifting .

7, Ultrasound with big and small head

1.Ultrasound: Ultrasonic therapy (3 million times per minute) is suitable for the skin on facial, eyes and body therapy. Ultrasonic high frequency waves cause proper heat and physical stimulation and movement changes of human cell. It stimulates and adjusts cell membrane. It enhances performance of epidermis and its heat speeds up metabolism and regeneration of epidermal tissue. Meanwhile it helps Ion- and Ion+ better lead in and out any type of skin.

2.Physical Massage: Physical vibration(6500-7000 times per minutes) can accelerate blood circulation, dilute melanin and extra moisture and remove dropsy. Working with Ultrasonic function, it can whiten and tighten your skin.

Functions:

- 1) Clean skin deeply
 - 2) Eliminate wrinkles, moisturize skin
 - 3) Firm and tighten skin
 - 4) Reduce fat, shape slim face contour
 - 5) Dilute spots, make skin be white
 - 6) Remove dark circles, eye bags
- ?)Promote blood circulation and metabolism

8. Ultrasonic skin scrubber

Ultrasonic scrubber use 25KHZ ultrasonic vibration with Nano technology, this vibration can penetrate into skin around 3-7mm, make skin temperature increasing, improve cell metabolism, accelerate microcirculation and make metabolism reflux. By converting of Ion, increasing permeability of membrane, make the fatty acid mutate into CO₂ and H₂O, the volume of fat cells become smaller. If you match with the skin care cream or liquid, it will be more effective for face beauty.

9, Co2 bubble

In contact with the skin , millions of tiny Co2 bubbles will be produced , sending oxygen-rich blood to the skin , increasing capillary flow and skin metabolism. And the natural oxygen supply of the skin is awakened, multiple absorption products.

CONTRAINDICATIONS

- People who has fever, infectious diseases, acute diseases.
- People with heart disease or configured cardiac pacemaker.
- Patients with severe high blood pressure, tumor disease, asthma, deep vein thrombosis, varicose veins, thyroid, cancer, falling sickness.
- People with hemorrhagic disease, trauma, vascular rupture, skin inflammation, skin Disease.
- Pregnant women
- Do not use at the abdomen during menstrual period
- Medical Plastic parts, or parts with metal inside
- People with an abnormal immune system

PRODUCT SERVICE AND MAINTENANCE

1. Warranty: The warranty date is one year from the date of purchasing. We are responsible for the maintenance in our facility in UK. When the warranty date is expired, we will charge the service fee or the cost of the component.
2. Professional Use: The merchandise of our Products is designed for professional use only and must be installed by a licensed professional electrician only. Failure to do so will void customer warranty.
3. Manufacturer Defects: In the event of defects in material and workmanship our obligation shall be limited to free labor and parts repair or replacement, solely at our option, of any defective part for a period of one year from the date of purchase.
4. Disclaimer of Damages: The remedies provided herein are the buyer's sole and exclusive remedies and in no event shall our products be liable for direct, indirect,

special, incidental, or consequential damages (including loss of profits) whether based on contract, tort, or any other legal theory.

5. Returns: All damaged machines must return in 30 days after receiving the confirmation from our service department.

6. Shipping Policy: For your protection, the acceptance of this shipment by the Transportation Company is an acknowledgement that the articles were delivered to them in good working condition and properly packed.

7. Exceptions

- Inappropriate damage such as: dramatic impact, vibration, drop, condenser chamber burning, crystal rods destroying, etc.

- Inappropriate damage or loss of key components such as the handpiece, protective glasses, power lines, you will be charged for the cost of accessories.

- Consumer who repairs the machine caused by improper by himself.

- Due to irresistible forces (such as fires, earthquakes and other natural disasters) of damage.

8. Beyond the period of warranty products.