

Please don't shoot if the head is not against skin or Acrylic (don't empty shooting)

Treatment Direction

- Mark: 1. The black lines to help you separate the treatment area, easy to operate for beginner.
2. The arrows are the direction of moving the cartridge during the treatment.
3. "X" is the treatment forbidden area.
4. Treatment needs to avoid all bones, lymph, spine, throat...



Face

In one session, firstly use 4.5mm cartridge pass 1 time, then use 3.0mm cartridge pass 2 times.

Totally pass 2~3 times for each area.

Reason: Use 4.5 cartridge first, it's for pull-up SMAS layer; Then use 3.0 cartridge, its for stimulation of collagen, the result will be better.

Eyes around and forehead, use 1.5mm pass 2 times.




Neck

In one session, first use 4.5mm cartridge one time, then use 3.0mm cartridge one or two time.


Totally pass 2~3 times for each area.

Reason: Use 4.5 cartridge, its for pull-up SMAS layer firstly; Then use 3.0 cartridge, its for stimulation of collagen, the result will be better.

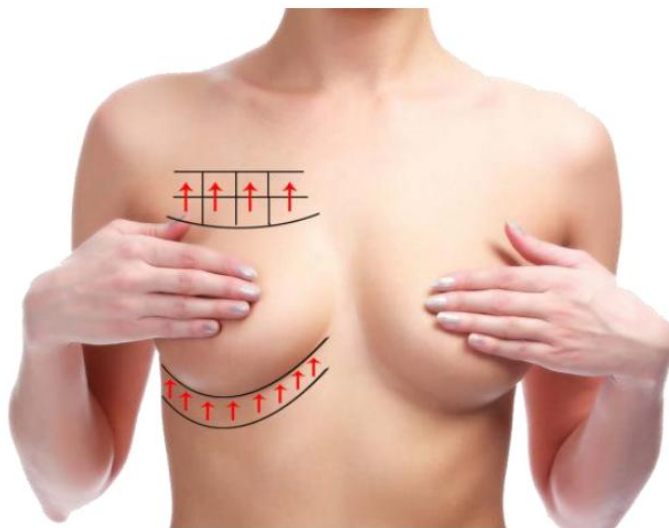

Chest

In one session, use 4.5mm or 6mm cartridge, according to clients fat thickness. Pass 2~3 times for each area. *Do not treat on the breast, but only need to treat at the bottom and upper part of the breast, in this case, it can achieve the effect of lifting the chest.

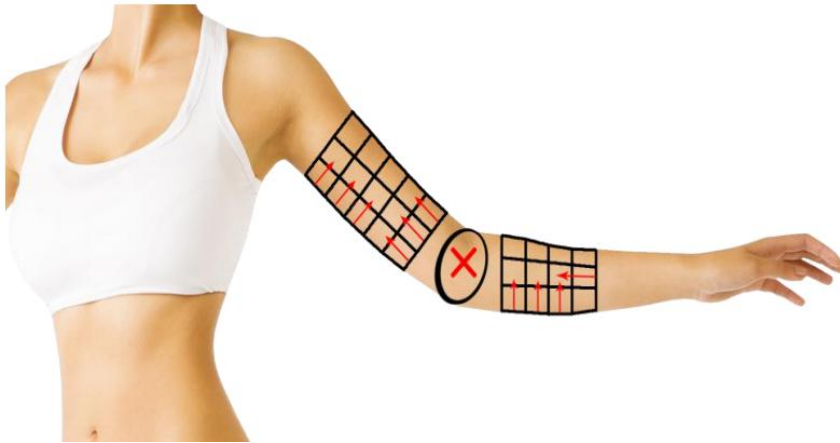

Arms

In one session, use 6mm or 8mm cartridge, according to clients fat thickness. Pass 3~4 times for each area. *3 fingers distance from Armpit, 2 figures distance from elbow, avoid the bones and muscles.




Abdomen

In one session, for belly and waist treatment, we suggest use 13mm and 10mm to do treatment, if fat layer is too thick, you can use 16mm depth.

Use 13mm cartridge do 1 time firstly, then use 10mm cartridge do 2~3 times.

Pass 3~4 times for each area.

*Forbidden area: Navel, stomach, ribs, Iliac crest

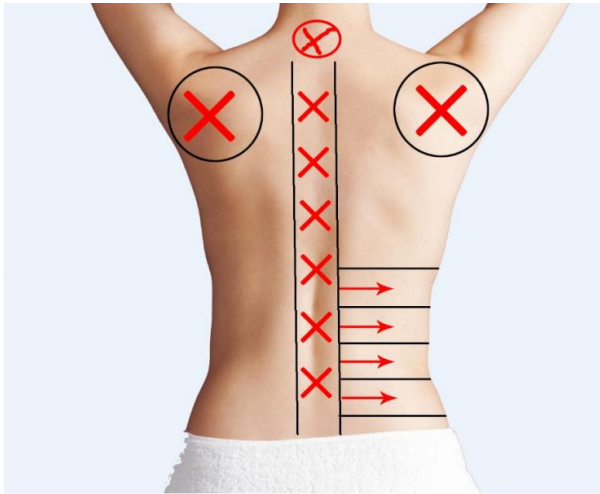


Back

In one session, we suggest use 8mm and 10mm to do treatment, if fat layer is too thick, you can use 13mm cartridge.

Use 10mm do 1 time firstly, then use 8mm do 2~3 times.


Pass 3~4 times for each area.

*Forbidden area: Spine, scapula, the top of scapula (it's like a raised area)

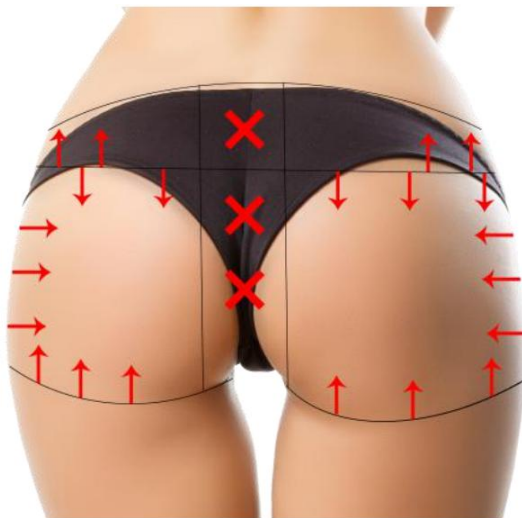

Buttock

In one session, we suggest use 13mm and 10mm for Buttock, you can combine with each 2 cartridges to do the treatment, according to customer's situation. Use the cartridge from deep to shallow. Pass 3~4 times for each area.

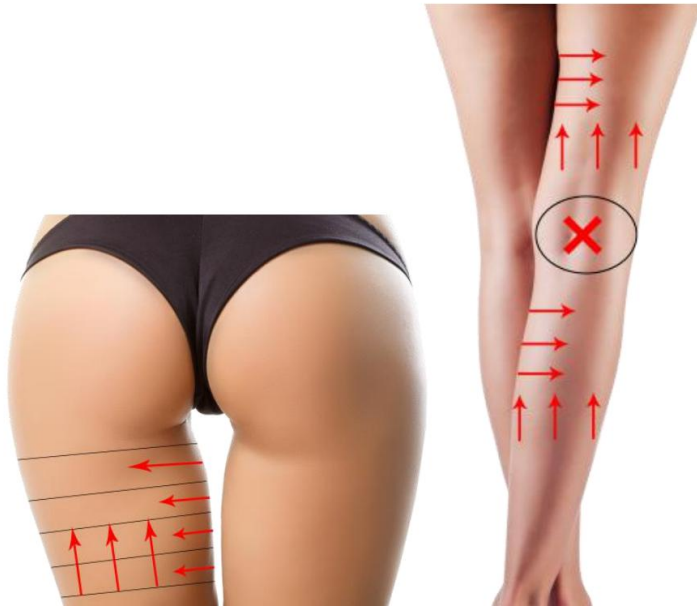

Leg:

In one session, we suggest use 10mm cartridge first, then use 8mm cartridge, from deep to shallow.

The picture shows the inner thigh, which is the main area of the leg treatment, please don't do treatment on knee woof.

Pass 3~4 times for each area.

Parameters setting

Treatment area	Cartridge	Energy suggestion	Length	Lateral point spacing	Vertical point spacing	Width
Eyes around & Forehead	1.5MM	0.1-0.3J	15-25MM	1.5-2.0MM	1.5-2.0MM 4-5lines	6-10MM
Face	3.0MM 4.5MM	0.4-1.0J	15-25MM	1.5-2.0MM	1.5-2.0MM 6-9lines	6-10MM
Neck	3.0MM 4.5MM	0.4-1.0J	15-25MM	1.5-2.0MM	1.5-2.0MM 6-9lines	6-10MM
Chest	4.5MM 6.0MM	0.8-1.6J	20-25MM	1.0-2.0MM	1.0-2.0MM	10MM
Arm	6.0MM 8.0MM	1.0-1.6J	20-25MM	1.0-2.0MM	1.0-2.0MM	10MM
Abdomen	10.0MM 13.0MM	1.2-2.0J	20-25MM	1.0-2.0MM	1.0-2.0MM	10MM

	16.0MM					
Back	8.0MM 10.0MM	1.2-2.0J	20-25MM	1.0-2.0MM	1.0-2.0MM	10MM
Buttock	10.0MM 13.0MM	1.2-2.0J	20-25MM	1.0-2.0MM	1.0-2.0MM	10MM
Leg	8.0MM 10.0MM	1.2-2.0J	20-25MM	1.0-2.0MM	1.0-2.0MM	10MM

Postoperative care:

1. After HIFU treatment, use moisturizing facial mask every day in the first week, then two times a week in following time.
2. In the first week, wash face with warm water, not too hot or too cold.
3. No sauna, no Bikram Yoga, no strenuous exercise, no smoking and drinking in one week.
4. Don't to be exposed to the sun in the first month.
5. Avoid high-tech treatment items such as anti-aging, pigment removal...
6. Don't eat dark color and spicy stimulation foods, eat more fruits and vegetables.
7. A few people will appear slight redness on the skin after treatment, this is normal, please smear some aloe vera gel or erythromycin ointment, the redness will disappear within 3-5days.
8. One week before and after the treatment, take some nano-collagen would enhance the treatment result.

Notes for use and maintenance

1. Disinfect the cartridge with alcohol before treatment, clean up the cartridge with a wipes after treatment.
2. Use the plug with ground pins, and make sure that the socket has been connected to the ground truly.
3. Make sure the voltage is stable.It is suggested increasing the regulated power supply with power matching to the device if the local power voltage is unstable.
4. Please use the specified accessories provided or suggested by the original factory so as to ensure the treatment effect and normal service life of the device.
5. Do not place the device in humid place or close to the water source, or expose apparatus to the sun either.
6. Never keep the device closed to strong thermal source because that may influence the service life and normal application of the device.
7. Use the current loop arm set in winter or dry air in avoidance of poor contact.
8. Mediums such as water and oil are forbidden in device, as well as striking, knocking and cracking that may damage the device.
9. Please clean the cartridge and cut off the power timely after using the ultrasonic beauty device.

Taboos for apparatus users

This apparatus shall be used with enough caution in the following cases, suggestions from

doctors or professionals are necessary prior to use, some details are listed as follows:

1. Clean the skin at physiotherapeutic part before using the apparatus to prevent dirt from entering into the skin with the ultrasound or compromising the penetrability of the ultrasound. Please remove all metallic objects before the treatment lest unexpected situation impacts curative effects.
2. It is strongly recommended to use medium with certain viscosity to facilitate better fusion between ultrasound and skin and prevent any reflection arising from gap from hampering acoustic and energy absorption.
3. Generally each physical therapy period shall be maintained for 15 to 20 minutes, over long time or improper use will cause skin fatigue or bad absorption. The user may select appropriate duration considering the size of physiotherapeutic part.
4. The degree of heat at the probe cannot be treated equally as the ultrasonic power output; it is not advisable to penetrate low-concentration agent or drug directly, which will lead to dry skin.
5. When this apparatus is being used, the probe must be kept away from the eye and prohibited from passing the eyeball, the pregnant and patients with severe heart disease.
6. Unless the doctor's permission is obtained, the patients in the course of treatment shall be cautious enough to use this apparatus.
7. Patients with malignant tumor, hemophilia or severe bleeding;
8. Patients with the major disease, skin disease or infectious disease shall be cautious enough to use this apparatus.
9. Enough caution must be given when such apparatus is used in the heart, brain, neck nerve centre, spine, eyeball, bleeding and injection.
10. Patients with the hyperthyroidism, asthma and hypertension;
11. Ultrasonic scalpel treatment is not allowed within 15 days after injection of hyaluronic acid.
12. Please do not wash you face with hot water and don't massage in one week, please apply mask every day during the first week, please pay attention to sun protection, please don't operate other beauty equipment on treatment skin in one month, such as RF\Vacuum\ultrasound\mechanical roller etc.

Troubleshooting

1. No display on the on the apparatus interface: Please check if the power plug is inserted well or the power switch is turned on.
2. The apparatus operates normally but the press key malfunctions: Please check if the power supply is connected well.

Repair and maintenance of the apparatus

1. Please clean the apparatus with clean water after its use and preserve properly;
2. Power plug with ground pin must be used before application of this apparatus and ensure that the power outlet is truly grounded.
3. The apparatus voltage is adaptive; in case of unstable local supply 5 voltage, we suggest users

to add power matching stabilized voltage supply.

4. Please use consistently components provided or suggested by original manufacturers to ensure curative effects and normal lifetime of the apparatus.

5. Neither place the apparatus in damp places or near the water source nor expose them directly to sunshine.

6. The apparatus shall be avoided to approach strong heat source body, because it may affect the life and normal use of the apparatus.

7. Please remove firstly all metallic objects lest unexpected situation impacts the curative effects.

8. Avoid poor contact and ensure a certain degree of wetness at the part contacting with the skin, which may be achieved by wiping with water soluble product or water.

9. Please turn off the power switch when the apparatus remains idle and ensure that the general power of the apparatus is turned off before/after its use and the safety of electrical products.

10. Please use this apparatus or train its operators in strict conformity with the instructions in users' manual.