


Mustang Masks

Simple Adjustable Mask

1. Place front and back cotton pieces right sides together.




2. Stitch across the top and sides. Leave a one inch opening on each corner.

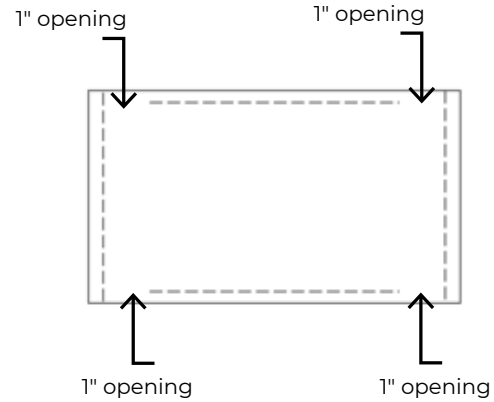


3. Turn right side out and iron flat.



4. Sew the sides leaving a 1/2" space on each end. Top stitch the top. Be careful not to sew beyond the 1/2" space on each side.

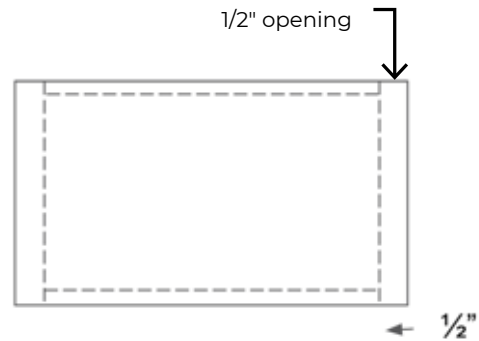

5. Pull the tie through the opening on each side.

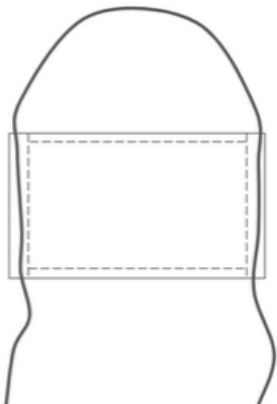

View Video Instructions:

Join the Facebook Group:

<https://www.facebook.com/groups/mustangmasks>

Website:

www.mustangmasks.org