
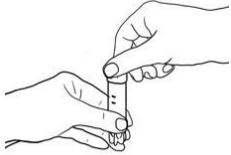
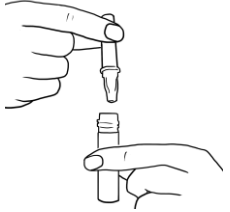


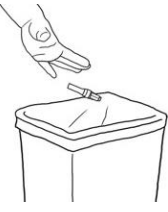
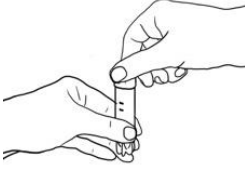
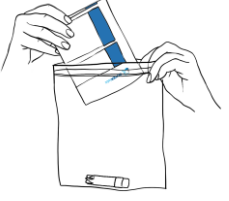


Step 1	Consent	 Consent Link
	<p>➤ Please fill out the online Consent at https://app.rover-labs.com/consent/NJDOH</p> <p>➤ For those unable to complete the online Consent:</p> <p><i>By filling out this form you are acknowledging that you have read, understood, and agreed to the full consent language linked by the QR code to the right.</i></p>	

Step 2	Fill out Patient Information	
	Patient Name: _____	School/Test Location: _____
	Patient Date of Birth: (mm/dd/yyyy) _____	Parent Name: (if patient is student) _____
	Mobile #: (use parent's if student) _____	Residence ZIP Code: _____
Email: (use parent's if student) _____		Date of Sample: _____
Please PRINT carefully and fill ALL sections, or your test cannot be processed.		

***The following steps (3-4) are used only for Independent Testing**

Step 3*	Collect Saliva Sample			
	You will need: 1. Saliva vial 2. Saliva funnel 3. This sheet 4. Ziploc bag	 Step 1 – Unscrew vial cap	 Step 2 – Insert saliva funnel into vial until snug fit	 Step 3 – Supply sample (2 drops or approx. 0.25ml)
	 Instruction Video	 Step 4 – Remove funnel and discard	 Step 5 – Screw on vial cap	 Step 6 – Put vial & folded registration sheet in bag & seal
	Have you completed and put this sheet in the sample bag? If not, your test cannot be processed.			

Step 4*	Drop off Sample at School
	<p>Your school has a drop-off location for samples. Please drop off sample within 24 hours of supplying the sample.</p> <p style="text-align: center;"><i>Contact your school administrator to find the location of the sample drop box.</i></p>

Need help? Email us at njtest@rover-labs.com