

DIS IN FECT 5

BACTERICIDE VIRUCIDE FUNGICIDE DIN#02264021

DESCRIPTION DISINFECT 5 is a ready-to-use, highly effective cleaner and disinfectant for use on the surfaces of inanimate objects. It is especially useful to disinfect non-porous hard environmental surfaces in livestock facilities, health care facility (hospitals, dental clinic, nursing homes) and / or a food processing plant.

APPLICATION For cleaning medical equipment and surfaces Apply directly to surface. Wipe surface cleaner using a paper or cloth towel or rinse surface and either wipe dry or allow to air dry. Discard dirty towel.

For disinfecting medical equipment and surfaces Apply directly to precleaned surface, thoroughly wetting area to be disinfected. Allow surface to remain wet for 10 minutes at room temperature (20°C). Follow by wiping surface using a clean paper or cloth towel; or rinse and either allow surface to air dry or wiped rinsed surface dry using a clean paper or cloth towel. Discard dirty towel.

For disinfecting surfaces in food processing establishments:

To Disinfect: Apply directly to precleaned surface, thoroughly wetting area to be disinfected. Allow treated surfaces to remain wet for a minimum of 10 minutes in a single application. Can be applied with a mop, sponge, or cloth as well as coarse (trigger) spray. For food plant use, food contact surfaces should be rinsed with abundant potable after disinfection.

PROPERTIES Appearance: Clear liquid

Odor: Alcohol

LIQUID DISINFECTANT FOR HARD SURFACES GUARANTIE / GUARANTEE:

Alkyl (C14 50%, C12 40%, C16 10%) Dimethyl Benzyl Ammonium ChlorideE...0.20% Octyl decyl Dimethyl Ammonium Chloride...0.15% Dioctyl Dimethyl Ammonium Chloride....0.06% Didecyl Dimethyl Ammonium Chloride.....0.09%

DIS IN FECT PRODUCTS LTD

P.O Box 338 Balmoral, Manitoba ROC-0HO