

Birthstone Trauma Healing Alpha-Stim Rental Agreement

Thank you for choosing self-care and symptom treatment through Alpha-Stim at Birthstone Trauma Healing! This document is a rental agreement for you to reference during the period you are using an Alpha-Stim Device.

Directions for use. Please refer to the usage manual for utilizing and caring for the device. You and I will determine a recommended length of time and frequency for your daily use. Please hold to that usage recommendation and contact me if you have any questions or concerns.

Pricing. The weekly fee to rent an Alpha Stim Device for a three-week period is \$150, or \$50 per-week. Payment is accepted via IvyPay or with cash.

Incidentals. As with any important item you own, please do all you can to keep the device safe and clean. Store it in the case, keep out of reach of children and pets, and follow directions for cleaning and sanitizing the ears. Should something happen to the device outside of normal use, please let me know. A maximum \$250 fee toward replacement of the device will incur if the device is broken or neglected.

Return. Please return the device once your third week has concluded. If you are interested in using the device again or integrating it into sessions, please let me know. If you are interested in purchasing a device of your own, I can supply you with a referral to Alpha-Stim to do so.

Statement of Agreement. "I agree to the terms of this rental agreement. I understand and will adhere to the policies and procedures regarding device use and care, rental pricing, incidentals and device return.

Printed Name: _____ **Rental Duration:** _____

Signature: _____ **Date:** _____



ALPHA-STIM® AID

QUICK START GUIDE


for treating anxiety and insomnia



To watch instructional videos, scan with your phone


HOW TO USE EARCLIPS

for treating insomnia and anxiety

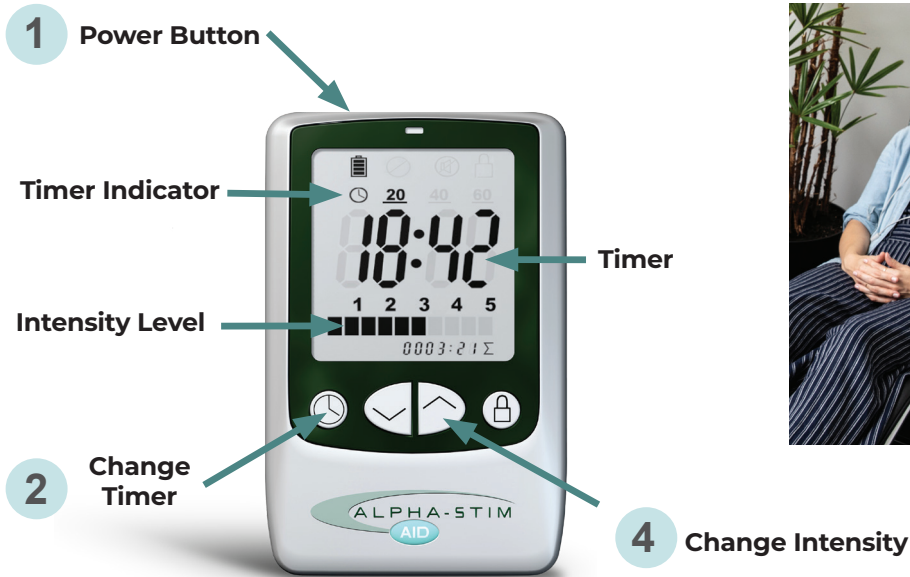

BEFORE YOU START

- Clean ear lobes with mild soap and water, alcohol pads or antibacterial wipes and allow skin to dry.
- Saturate new Earclips Electrode Pads (EEPST[™]) thoroughly with several drops of Alpha-Stim[®] Conducting Solution.

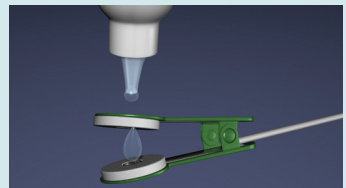


1



2



3

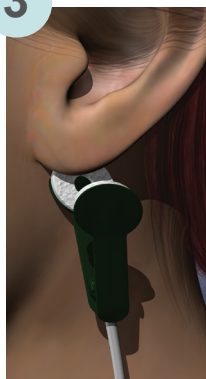

4



1. Power on device.
2. Set timer to 20 minutes if the current is at 2.5 or higher or 1 hour if the current is at 2 or lower.
3. Apply one saturated Earclip to each earlobe (pre-wet with several drops of Alpha-Stim® Conducting Solution).
4. Increase current until a slight dizzy feeling is experienced, then decrease immediately until the dizziness stops.

AFTER TREATMENT

- Remove and discard earclip pads.
- Clean earclips with alcohol, let dry, and replace 4 earclip pads for the next treatment.

ALWAYS COMPLETE YOUR TREATMENT

When the timed session ends, and a “heavy” feeling is still experienced, resume treatment until heaviness lifts and a light feeling develops.

For help with how to use your Alpha-Stim, call 1-800-367-7246



Made in the USA and in China to the exacting specifications of EPI, USA. © 2026 Electromedical Products International, Inc. All rights reserved. Alpha-Stim and the Alpha-Stim logo are registered trademarks of Electromedical Products International, Inc. Manufactured under US Patents 8,612,008 and 8,457,765.