# PM-720

## Digital TENS/EMS (Combination)

- ◆ Pain Relief and Muscle Stimulation
- ◆ LCD Display
- ◆ EMS Three Modes
- ◆ TENS Six Modes
- ◆ Nine Preset Programs
- ◆ Adjustable Timer, 5 minutes each step and constant
- ◆ Cumulative (Compliance) Timer
- ◆ System Includes:  
Stimulator, Lead Wires, 9V Battery,  
4 Self-Adhesive Reusable Electrodes,  
Instruction Booklet, Hard Carrying Case
- ◆ 9V AC Adapter; 9V Rechargeable Battery Kit Available

### Technical Specifications

Six TENS Modes:

B – Burst rate: Adjustable 0.5 – 5Hz.

N – The pulse rate and pulse width are adjustable.

MRW – Modulated Rate and Width.

S1 – Modulation Width – Pulse is varied to 40% of initial setting.

S2 – Modulation Width – Pulse is varied to 70% of initial setting.

P – 9 Preset programs.

Three EMS Modes:





S – Synchronous Mode – Output from both channels occurs synchronously. The “ON” time includes “Ramp Up” and “Ramp Down” time.

A – Alternate Mode – The stimulation of the CH2 will occur after Mode (A) the 1st contraction of CH1 is completed.

C – Constant Mode – The pulse rate and pulse width are adjustable. Continuous stimulation is delivered.

*All values have 10% +/- tolerance*

Features and Controls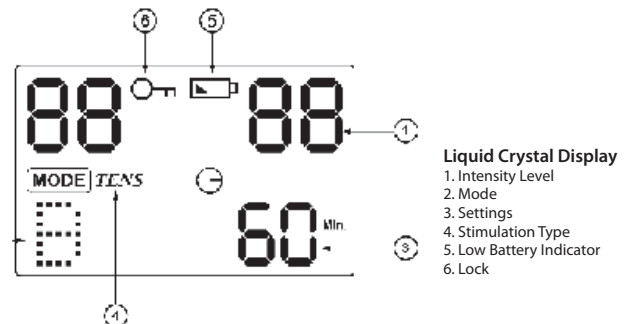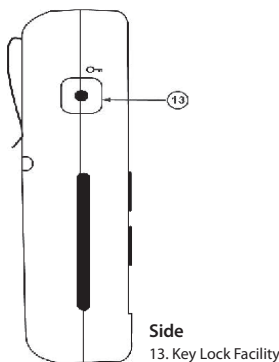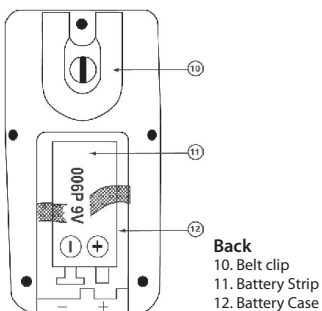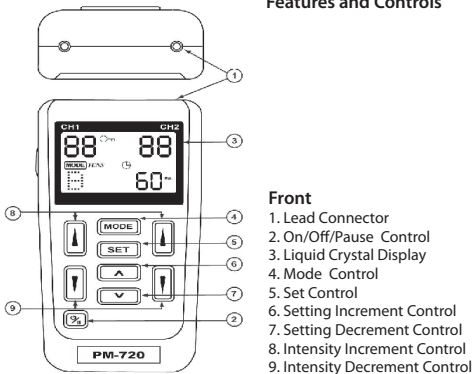


**Voice: 215.938.0200 ~ Toll-Free 1.800.472.9190 ~ Fax: 215.938-9309**

www.ProMedSpecialties.com ~ 1800 Byberry Rd. Unit 905 ~ Huntingdon Valley, PA 19006 ~ USA

## QUICK REFERENCE GUIDE

- Electrodes are reusable, and last approximately 1 month; After using your electrodes, place them back onto the storage sheet
- Intensity ranges from 1 – 100; Start around a 6 or 7 and adjust from there to either your doctor/therapists instructions, or to your comfort level
- Turn unit off before removing electrodes
- Make sure that the intensity (number) of Channel is the same as Channel 2
- Place electrodes AROUND the site of pain or injury, NOT ON THE PAIN/INJURY
- Reference the instruction manual to choose the Mode and Settings for that mode to choose the right settings for you- Follow the directives given by your doctor/therapist, or adjust to find the right settings for your needs

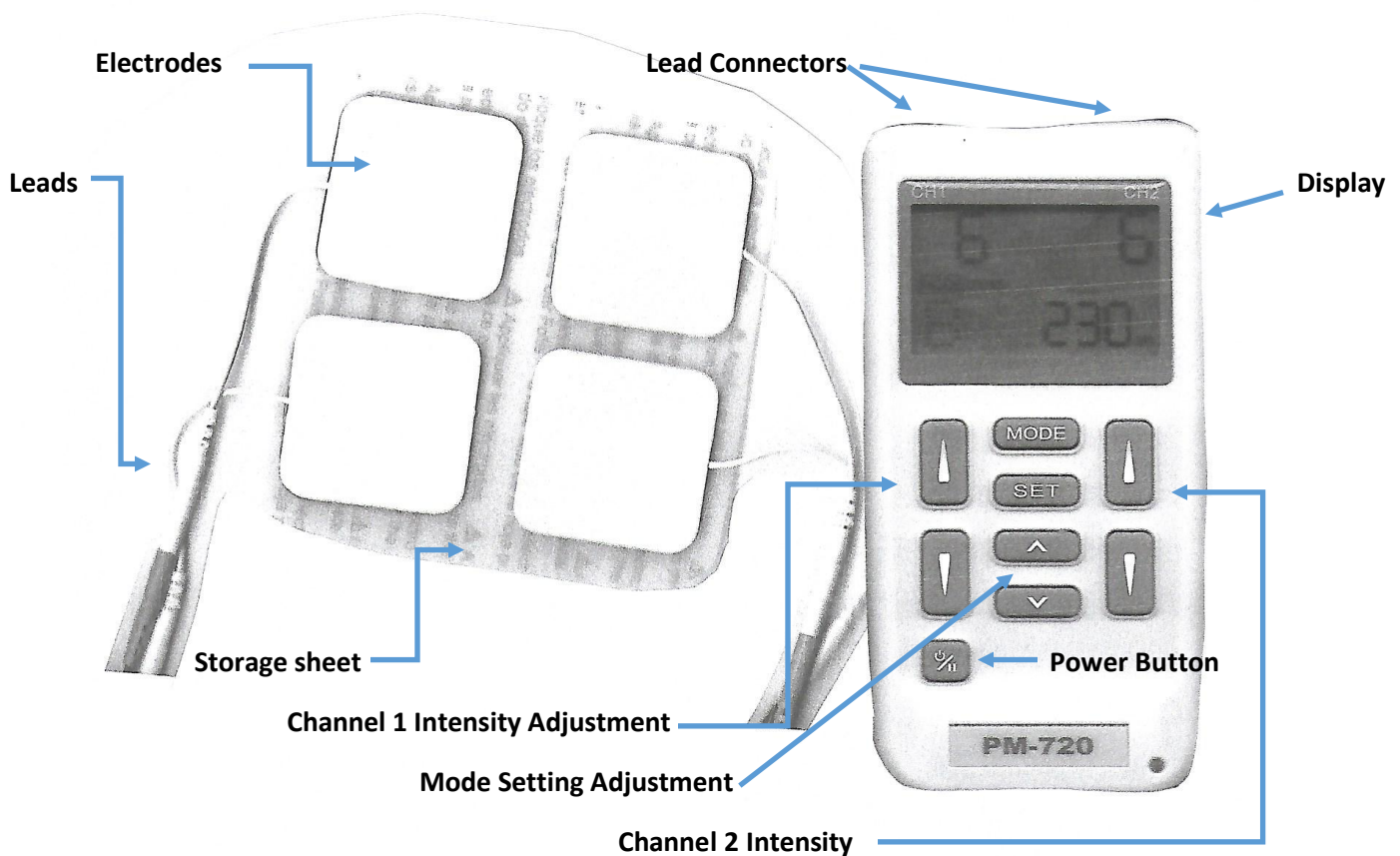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