



The Clear Solution To *Stop The Splash*

INSTRUCTIONS FOR USE

CLEANING & MAINTENANCE:

- Looney® may be cleaned and disinfected using facility-approved disinfectant wipes or bleach.
- The frequency of cleaning and disinfection should follow your facility's established cleaning and disinfection protocols.

USE:

- Lower the Looney® shield and bedpan washing diverter arm to the down position.
- Pro Tip: Hold the vessel close to the shield and spray nozzle to minimize outward spray.
- Keep the faucet and Looney® in the upright position when not in use.
- Follow your facility's protocols for cleaning and disinfecting Looney® and clinical measuring containers.

WARNINGS:

- Do not sit, lean, pull, twist, or place objects on Looney®.
- The Looney® shield is not weight-bearing and should not be used to support any weight.
- Do not use Looney® if the shield is cracked or damaged, hardware is missing, or the shield is not securely attached to the diverter arm.
- Improper installation or use may result in inadequate splash protection.