

Timeless
EVOLUTION



AquaFacial Professional Unit with 6 modalities

by Timeless Evolution

(800) 595-1925

WHAT IS AQUAFACIAL?

- **Skin Resurfacing**
- **Skin Rejuvenating**
- **Skin Hydration**
- **Skin Nourishing**
- **Pre Laser Protocols**
- **Pre Microneedling Protocols**

INTEGRATION OF

- **Cleansing**
- **Exfoliation**
- **Extraction**
- **Hydration**
- **Stimulation**
- **Infusion**



A woman with a glowing complexion and a geometric network overlay.

THE FEATURES

Immediate, visible results

Treat skin types I –VI

Combine with other protocols

Promotes Lymphatic drainage

Promotes muscle stimulation

Timeless
EVOLUTION



INDICATIONS

Hyperpigmentation

Fine lines

Management of Acne - Grades I and II

Dehydration

Aging Signs




Timeless
EVOLUTION




Tone+Contour+Smooth

Microcurrent uses low-voltage electricity to stimulate muscle, adenosine triphosphate (ATP) cell growth, and collagen development in the dermis. Improve Facial contour, promote blood circulation and improves lymphatic drainage

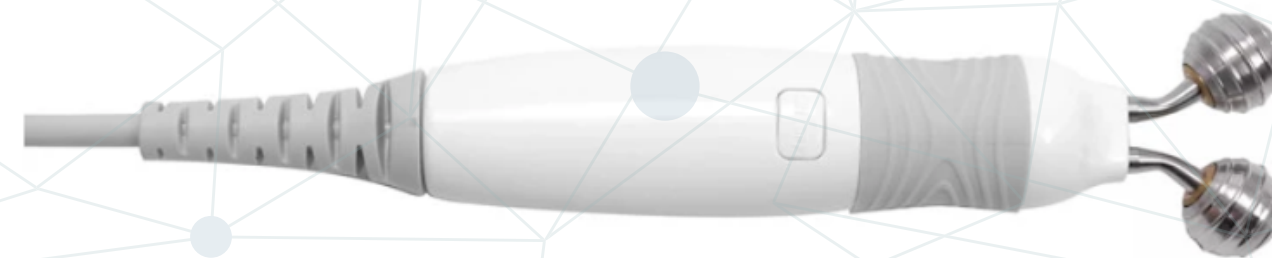


3D Bicornal Microcurrent Roller

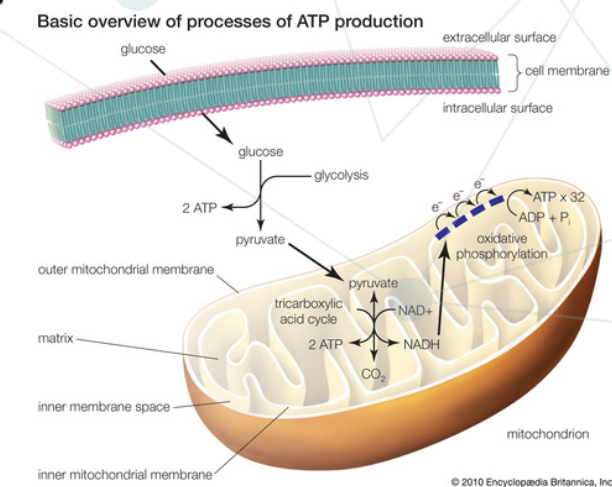

Non-invasive face-lift

Bipolar RF therapy uses low energy to heat the deep layer of the skin called the dermis. This heat stimulates the production of collagen to help improve signs aging. RF therapy is safe and can be effective used in treating mild or moderate signs of aging.

Timeless
EVOLUTION

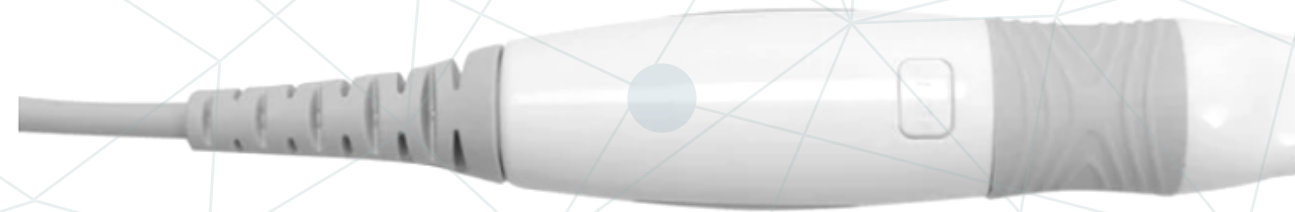

Bicorneal RF



Exfoliation and Extraction with the Power of Vacuum and Water



- Cleansing
- Exfoliation
- Extraction
- Hydration



Aqua Suction and Extra

Suction increases blood circulation, may relieve muscle tension, which can improve overall blood flow and promote cell repair. It may also help form new connective tissues and create new blood vessels in the tissue.




Oxygen and Hydration

Timeless
EVOLUTION

Intraceuticals Oxygen Infusion Treatment:

- Promotes collagen production – The infusion of oxygen and vitamins into the skin encourages the production of collagen.
- Helps detox the skin – Everyday toxins like air pollution and cigarette smoke rob the skin of much-needed oxygen.
- Speeds cell turnover – Medical researchers believe that the infusion of oxygen into the skin increases cell regeneration helps speed the healing of facial scars, acne and blemishes

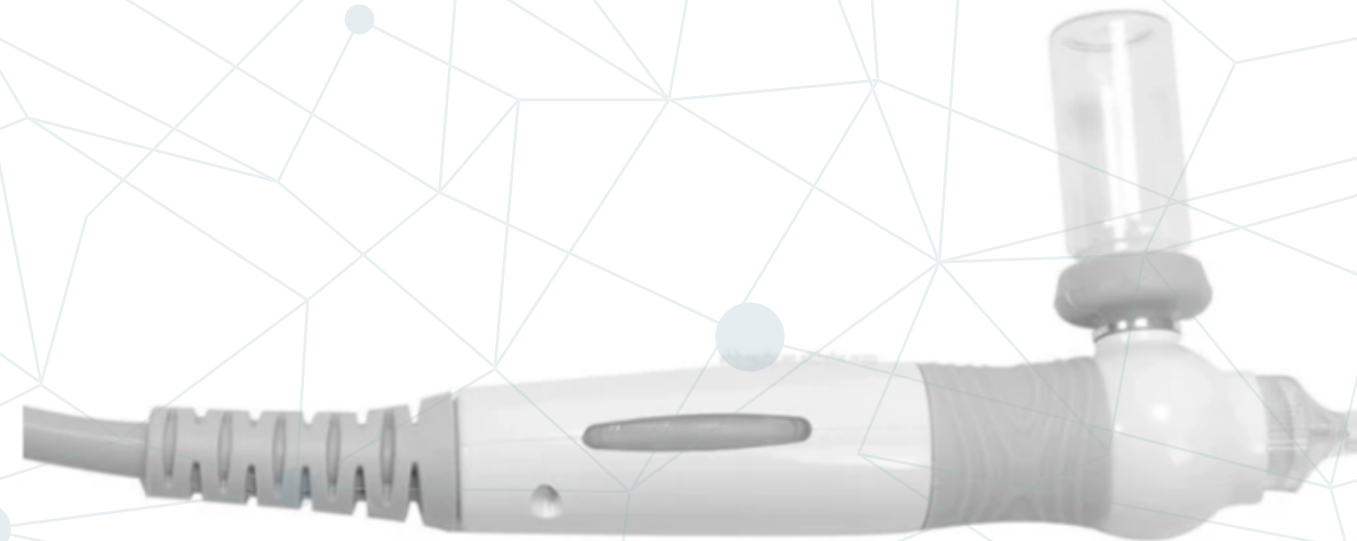

Fine Jet Stream

High polymer water oxygen gun for deep water and oxygen injection

Combined with nutrients, in the form of fine molecular atomization, the cells are injected into the basal layer of the skin to supplement nutrition and water.




Nano Infusion Mesotherapy is non-invasive delivery system that insures absorption of nutrients. It has built-in pump to dispense the serum and circular bipolar microcurrent that promotes absorption of any formula by breaking down molecules into smaller fragments that are of suitable size for absorption across cell membranes

Timeless
EVOLUTION




Nano Infusion Mesotherapy

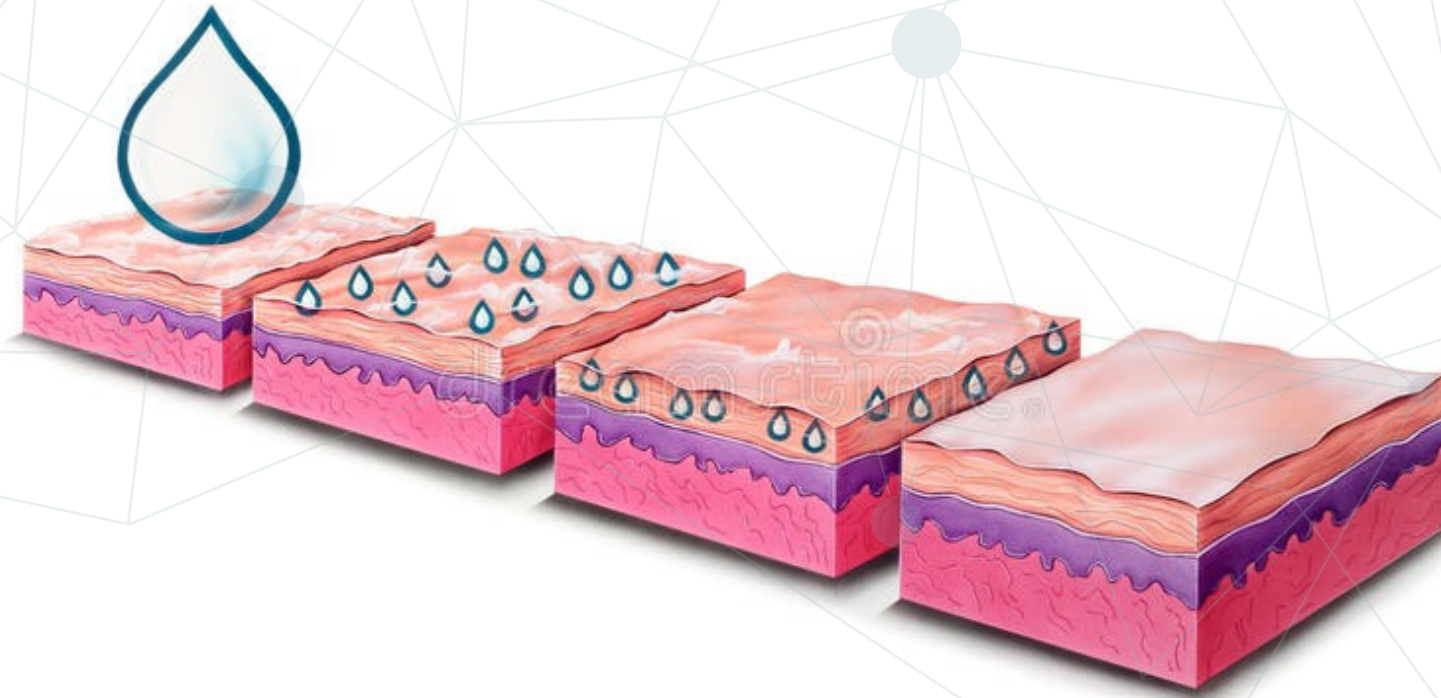

Cryotherapy Hammer
An essential tool to finish AquaFacial. By shrinking the pores and exercising capillaries of the dermis.

Timeless
EVOLUTION

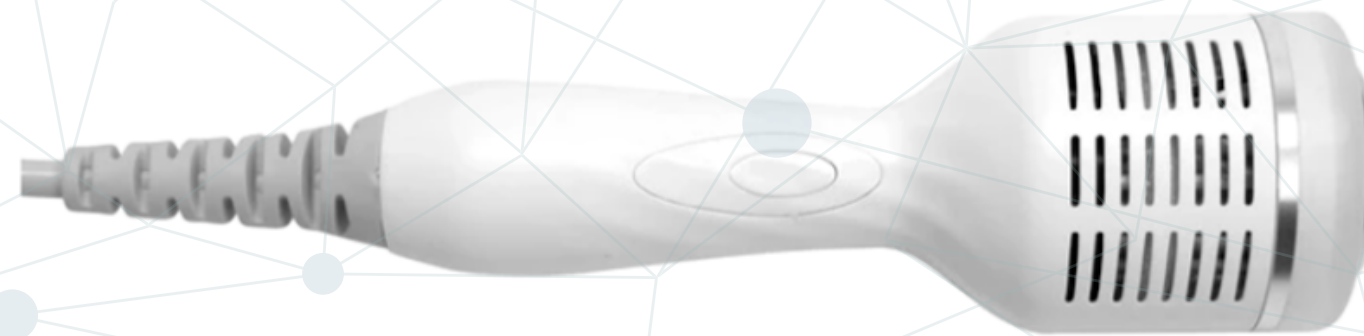

AQUAFACIAL CLINICAL FACIAL MENU

By Timeless Evolution

Signature AquaFacial

Deeply cleanses, extracts, and hydrates the skin through super serums filled with antioxidants, amino acids, and hyaluronic acid. Includes, Suction/extraction, Fine Jet Stream, and cold hammer therapy.

Clarifying AquaFacial

The Clarifying AquaFacial Treatment will include extended extractions, microcurrent and ultrasonic therapy to help aid in the removal of congestions and to kill acne-causing bacteria.

Radiance AquaFacial

The Radiance AquaFacial Treatment will add a Vitamin C Boost to minimize the appearance of dark spots aiding in a clearer complexion. Ideal for hyperpigmentation. Includes Bicornial RF and Nano Infusion Mesotherapy with AquaDeArus Revitalizing Serum

Age-Refinement AquaFacial

The Age-Refinement AquaFacial Treatment will stimulate skin and facial muscles to reduce the appearance of fine lines and wrinkles. Includes Suction/extraction, Fine Jet Stream, 3D Microcurrent Roller, Nano Infusion Mesotherapy, Bicornial RF, Cryotherapy Hammer

Restorative AquaFacial

The Restorative AquaFacial Treatment improve the appearance of skin tone, texture, and elasticity. Includes Suction/extraction, Fine Jet Stream, 3D Microcurrent Roller, Nano Infusion Mesotherapy, Bicornial RF, Cryotherapy Hammer.

Micro-Needling Treatment

Collagen Induction Therapy is a minimally invasive procedure where small needles are used to create micro-perforations in the skin. These tiny “holes” give the skin the opportunity to react as if it were injured, which stimulates collagen and elastin production. Great for fine lines and wrinkles, deep wrinkles, sagging skin, scars and acne scarring, ice pick scarring, and overall rejuvenation.

Treatment For Minor Skin Irregularities

Treatment utilizing direct high-frequency technology to remove and/or correct various skin irregularities such as Skin Tags, Telangiectasis, Cherry Angiomas, Keratosis, Fibromas, Sebaceous Hyperplasia, Sun Damage/Spots, and Age Spot.

Contraindications It is intended for external use on intact skin only, subject to the following contraindications and warnings:

- Do not use over active acne, such as severe, inflammatory acne.
- Do not use over active infection or open wounds (e.g., herpes simplex, excoriations, or open acne cysts).
- Do not use on client with autoimmune or communicable disease.
- Do not use on client with cold sores or fever blisters without pre-medication.
- Do not use over abnormal, unidentified facial growth or mark.

Side Effects Client may experience temporary irritation, tightness, or redness at the treatment area. This usually dissipates within 72 hours depending on skin sensitivity.


Warnings

- Not recommended for severe acne scarring.
- If the skin becomes irritated beyond normal, discontinue the treatment and seek medical advice.
- Do not use within the orbital area or over any body orifice, such as over the eyelids. Eyes should be closed during the procedure and covered with protective eyewear. Sterile eye rinse solution should be available in case products accidentally get in the eyes.
- This System is not intended to be used on infants, children, pregnant or lactating women because it has not been evaluated for this group of users.

AquaFacial Tips



Timeless
EVOLUTION

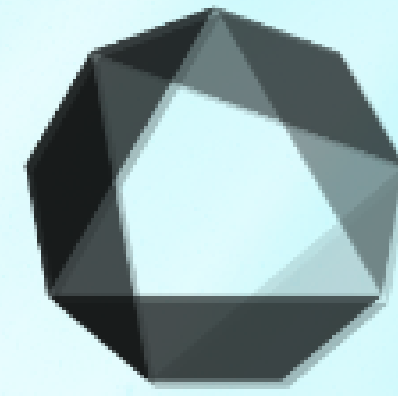

Are You offering AquaFacial Yet?

Clean → Tone → Brighten → Hydrate



800) 595-1925