

Timeless
EVOLUTION

Micro Needle RF

Monopolar & Bipolar RF Application in a Single Platform







Most Innovative | Timeless Evolution




Application

One Device | All Skin Tones | Face and Body

Face


Fine lines 	Wrinkles 	Acne scars 	Elasticity 
Reducing Aging Signs 		Pore Reduction 	

Body

Stretch Marks 	Scars from injury or surgery 	Laxity 
---	---	--



How Does RF MicroNeedling Work?

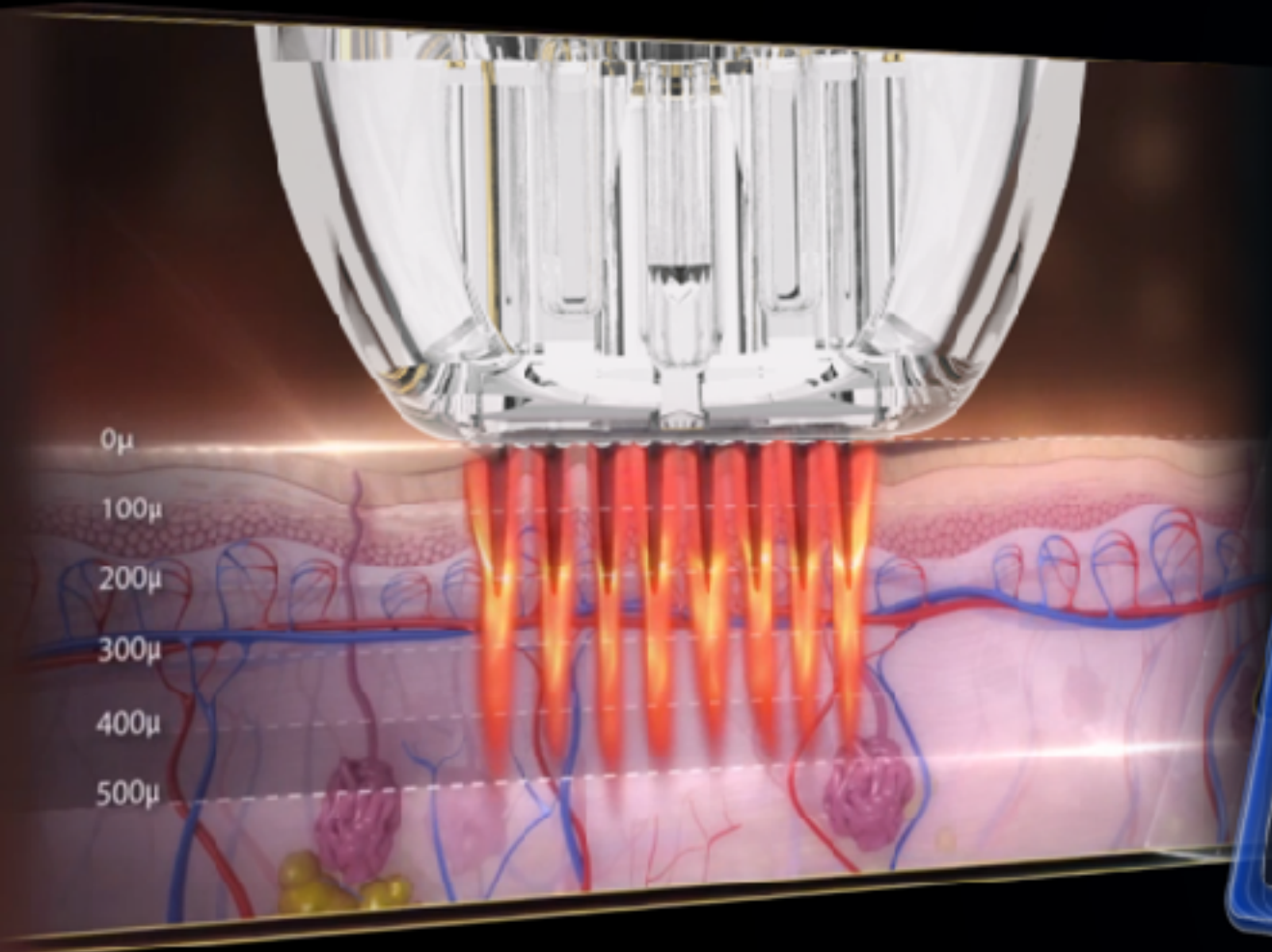

The RF energy heats the underlying layers of skin, causing constriction and tightening of the skin, as well as the stimulation of collagen and elastin production. Because the RF energy is delivered so deeply, it can produce faster results that are more dramatic. RF powered microneedling can be use on all skin types even IV V VI tones, and sensitive areas like around the eyes.

UI Interface



10.4 inch Full-size touch operation | Sliding menus




Advantage- With Dual Suction Chamber

Monopolar & Bipolar RF Application in a Single Platform

Advanced RF Micro Needle with build in one of a kind dual air chamber cartridge. This feature allows more precise depth penetration to deliver the RF energy efficiently. This feature can be used to deliver topical biologics to desired depth of the skin.

This advanced Micro Needle RF Platform will take you to new levels in Aesthetic Medicine. The versatility of this system provides wide range of customizable treatments with evolving client needs on face or anywhere on the body, any time of the year.

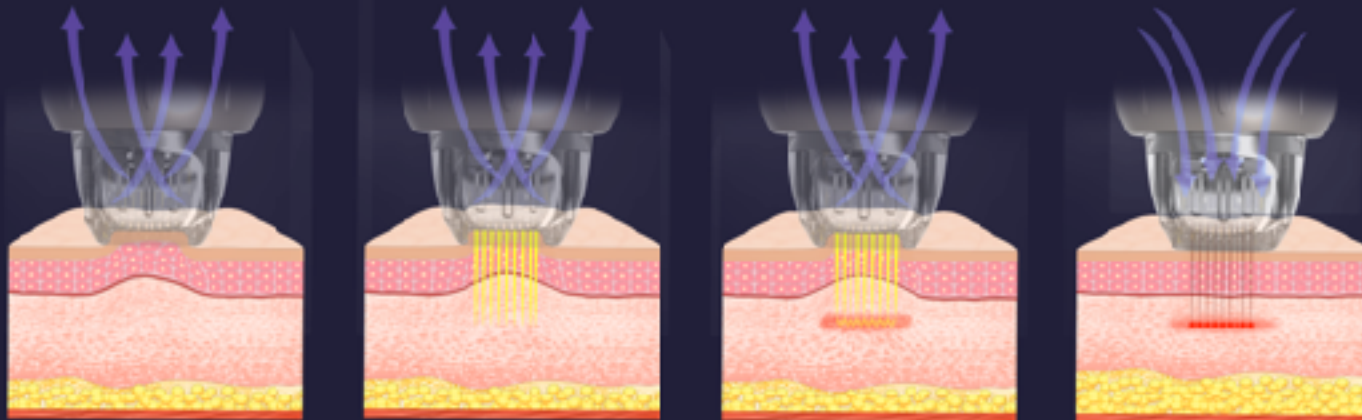

Auto Suction
Capture

Create channels

Deliver RF
Auto Suction Release
Auto Needle Retraction

After RF Delivered



Gold Plated Semi Insulated & Non Insulated Cartridges



M-18



M-25



M-49



M-81





- Stepping Motor Type, different to existing solenoid type, the needle inserts to the skin smoothly with no shock, and causes no bleeding and no pain after the procedure
- Needle is durable with high Biocompatibility because Gold Plated feature.
- Patient with Metal allergy could also use it without Contact Dermatitis.

- 5-4.0mm [0.2mm Step]
- Deliver epidermis layer and dermis layer by controlling the needle depth in unit of 0.2mm.
- Sterilized Single use cartridges
- operator can easily notice the applying RF energy from LED light.
- Needles designed to penetrate the skin with minimum resistance for painless treatment.

Semi Insulated & Non Insulated Application


RF Target Area

- Dermis

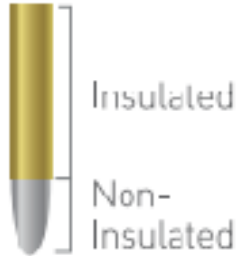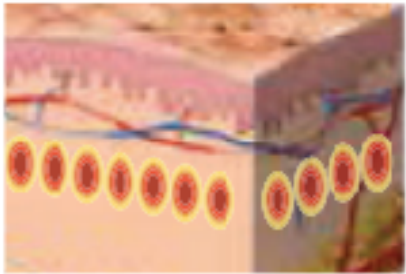

RF Target Area

- Epidermis & Dermis

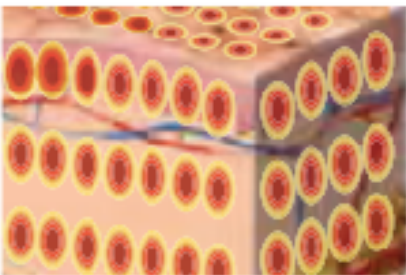

Semi Insulated

Semi Insulated microneedles have a layer of insulation around them, allowing the heat only to be emitted from the end tip, heating just the lower layers of the skin.

Non-insulated

Non-insulated microneedles emit Radiofrequency energy over their entire surface area allowing both the upper and lower layers of the skin to be heated all at once.



Thermal RF

Timeless
EVOLUTION

- Thermal RF applicators are for non-invasive skin rejuvenation, can be used by it self or in combination with RF micro needle



F-3*3

- Eyes
- Nose Area



F-5*5

- Face Area



F-9*9

- Body Area




Advanced Micro Needle RF

-- Negative polar (MONO/BI POLAR)


Bipolar

50% percent needle is positive and the other 50% is negative polar. this make the energy only works on depth of the needle. This mode is more suitable for face treatment.

Face : Evenly

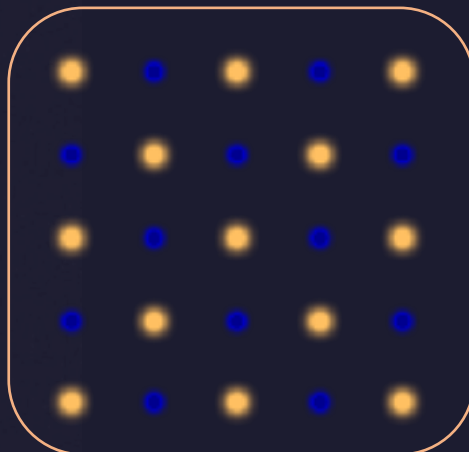

Body: Deeper

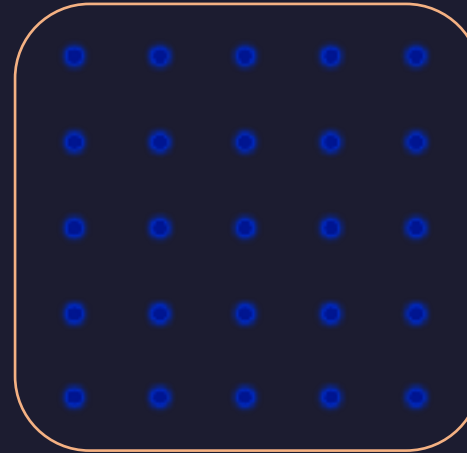

Mono mode

all the needles on the tip is positive polar. The customer need to hold the negative polar in hand during the treatment. it's more suitable for body treatment.

Negative Handle: Used with MONO polar application



Cooling handle: to Reduce Pain




Timeless
EVOLUTION



■ NEGATIVE POLE

■ POSITIVE ELECTRODE

Before & After

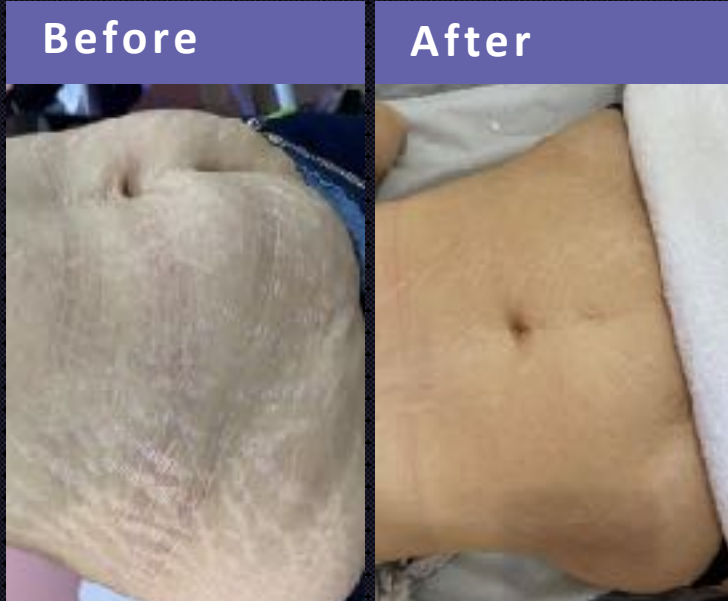

25 tips non insulated ●

Level 4
1.0 mm tip
140 ms
2 pass
3 weeks interval
3 treatment times



● 25 tips non insulated

Level 4
1.0 mm tip
140 ms
2 pass
3 weeks interval
3 treatment times



● 25 tips non insulated


Level 4
1.0 mm tip
140 ms
2 pass
3 weeks interval
3 treatment times

Design and Engineering Features


2MHz RF Frequency



Metal Frame

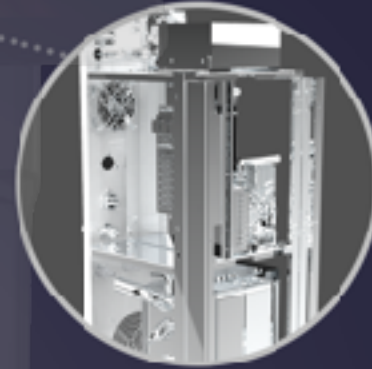

Stepping Motor Driving

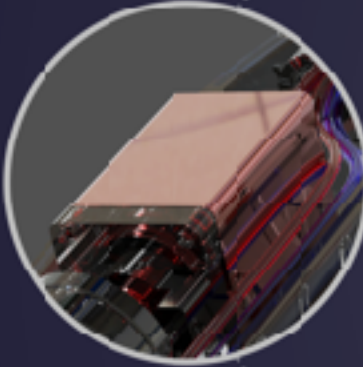

Button Switch



Modular component



360-degree silent wheel



Product Advanced Micro Needle RF



Power max 50 watts

Handle Fractional RF handle
Micro Needle RF handle

Frequency 2MHz

Energy 1-10 Level

Pulse Duration 10-900ms

Needle Depths 0.4-4mm adjustable

Pressure Level 1-4

Working mode SINGLE/AUTO

Mode MONO & BIPOLAR

Cooling 1-3 Level

Tips Insulated & Non-insulated

Invasive Tips M-18, M-25, M-49, M-81

Non invasive Tips F-3*3 , F-5*5, F-9*9



THANKS

www.timeless-evolution.com

2020 N. Glenoaks Blvd., Burbank CA 91504

(800) 595-1925

Timeless
EVOLUTION

