

Jeunetique Eyes Pro

Advanced rejuvenation for delicate under-eye skin with polynucleotides, hyaluronic acid, and peptide complex



A close-up photograph of a person's eye, looking directly at the camera. The image is heavily filtered with a solid blue color, which is semi-transparent, allowing the details of the eye and surrounding skin to be visible. The eye is dark, and the eyelashes are clearly defined. The skin around the eye appears smooth. The overall composition is centered, with the eye occupying most of the frame.

Rediscover the beauty
of your gaze with this
advanced skinbooster!

Experience
the transformative
power of Meamo labs
Jeunetique Eyes Pro
and reveal smoother,
brighter, and more
youthful-looking eyes

A large, light teal decorative shape that resembles a stylized 'M' or a wide, shallow 'U' with curved sides, positioned at the top of the page.

Contents

- 4** What is Meamo labs
Jeunetique Eyes Pro?
- 6** When Can Meamo labs
Jeunetique Eyes Pro help?
- 10** How Does Meamo labs
Jeunetique Eyes Pro Work?
- 15** Where and How to Use
Meamo Labs Jeunetique Eyes Pro?
- 20** How to Store Meamo labs
Jeunetique Eyes Pro?

What is Meamo labs Jeunetique Eyes Pro?



Meamo labs Jeunetique Eyes Pro is an advanced skinbooster designed specifically to rejuvenate the delicate skin under the eyes

This innovative formula helps eliminate dark circles, smooth fine lines, and restore the youthful appearance of the eye area.



The effectiveness of Meamo labs Jeunetique Eyes Pro is powered by active signaling molecules — polynucleotides (PN) — that penetrate deep into the dermis, stimulating cell regeneration. Enhanced by hyaluronic acid and a selection of ultra-short peptides (USP), this product offers a comprehensive anti-aging solution for the skin around the eyes



**The Beauty
of your eyes!**


**When Can
Meamo labs
Jeunetique
Eyes Pro help?**

Effectiveness of Jeunetique Eyes Pro




● WRINKLE REDUCTION

Smoothes fine lines and firms skin around the eyes.

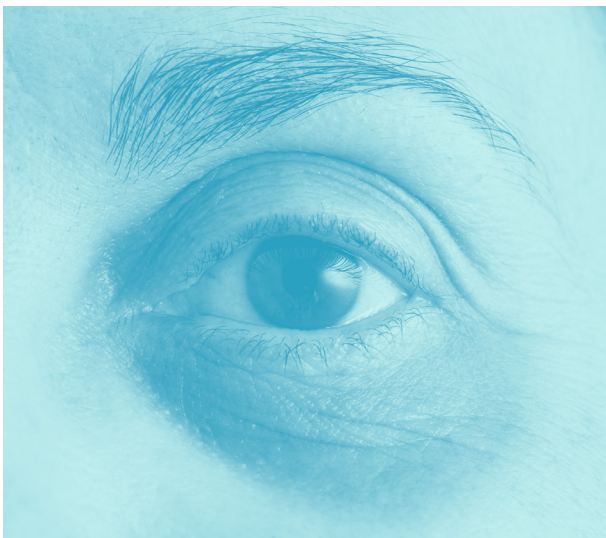

● BRIGHTENS SKIN

Effectively combats dark circles and tired eyes.

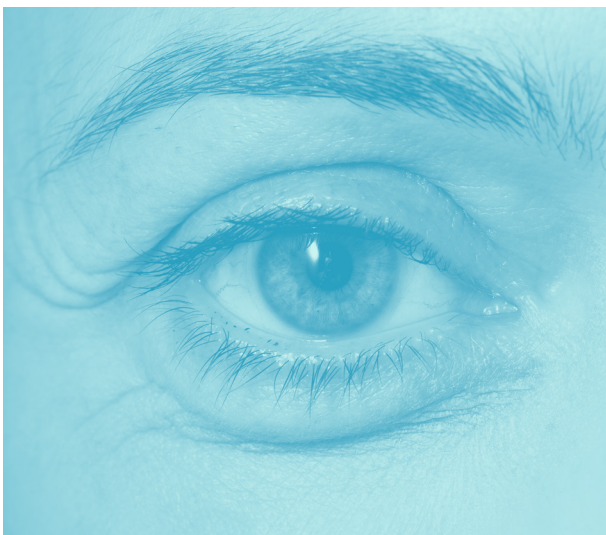


● ANTI-AGING EFFECTS

Improves skin elasticity, tightening sagging and reducing "bags under the eyes".

The three components of your beauty: What Meamo labs Jeunetique Eyes Pro contains?


Polynucleotides (PN)

Extracted from salmon gonads, PN promotes cell regeneration and stimulates fibroblast activity, helping to rebuild collagen structures in the skin.

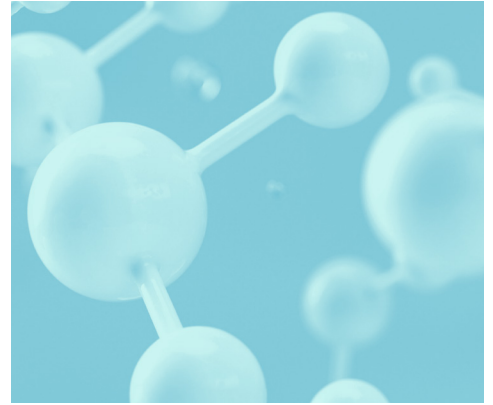

Hyaluronic Acid

A naturally occurring substance that hydrates the skin and replenishes lost volume, reducing the appearance of wrinkles and fine lines.

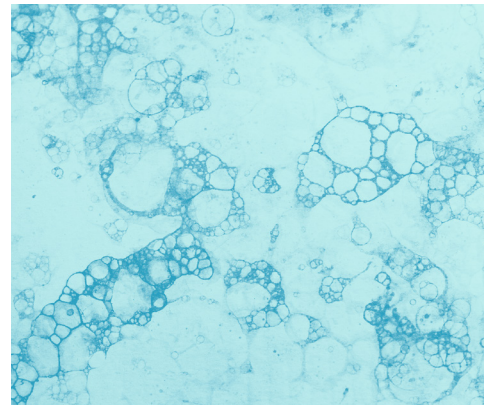

Ultra-short Peptides (USP)

Small protein molecules that regulate gene activity, stimulating collagen production and providing a lifting effect while reducing elastin degradation. They offer a local "Botox-like" myorelaxing effect, smoothing skin and improving its texture.



PDRN vs. PN – What's the Difference?

Polynucleotides (PN) are an enhanced form of PDRN, derived from the gonads of salmon rather than sperm. PN offers several advantages:

1.

More viscoelastic

Forms a longer-lasting porous structure, providing structural support in the skin.

2.

Stronger fibroblast activation

Stimulates collagen production more effectively than PDRN.

3.

Improved anti-inflammatory properties

Reduces inflammation more efficiently.

4.

Broader application

Ideal for skin rejuvenation across various anti-aging treatments.



How Does Meamo labs Jeunetique Eyes Pro Works?




How Does Meamo labs Jeunetique Eyes Pro Works?

The skin around the eyes is the thinnest and most sensitive, making it highly vulnerable to aging. Meamo labs Jeunetique Eyes Pro uses a gentle yet effective combination of ingredients to address these concerns.

Polynucleotides (PN)

In vitro and in vivo experiments have shown that the mechanism of action of PN is very similar to that of PDRN: activation of adenosine A2A receptors. PN also triggers the so-called nucleotide “salvage” pathway, which accelerates the formation of new fibroblasts and, consequently, **the synthesis of endogenous collagen.** However, PN is a purer and more concentrated form of PDRN and has stronger regenerative properties.

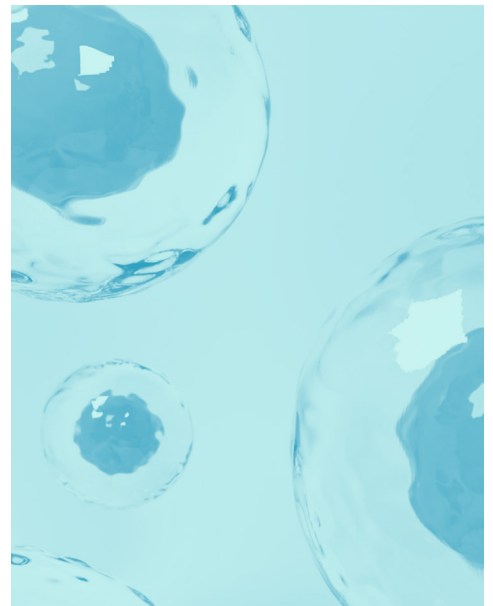


Ultra-short Peptides (USP)

The mechanism of action of ultrashort peptides is epigenetic. They are signal molecules that regulate the activity of genes and thus influence the synthesis of functional proteins.


Meamo labs Jeunetique Eyes Pro contains a complex of peptides including:

- Pentapeptide-13;
- Tripeptide-29;
- Dipeptide-2.

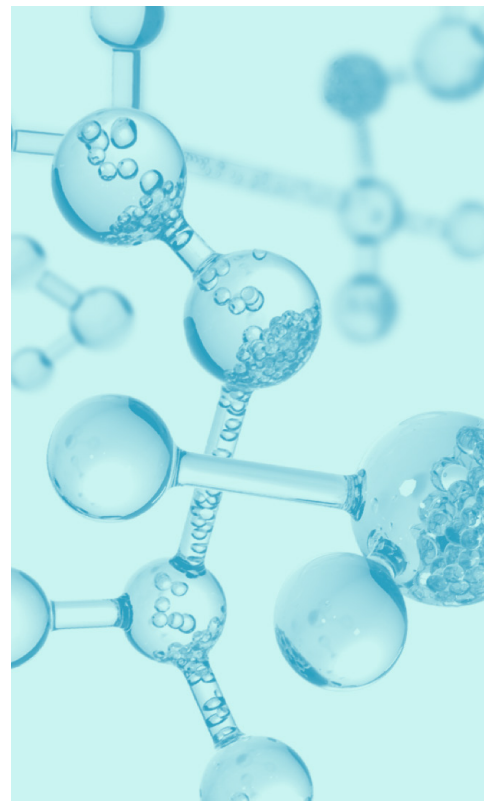


These short peptide molecules effectively eliminate wrinkles and hyperpigmentation, stimulate collagen synthesis and inhibit elastin degradation. In addition, by penetrating into the underlying muscle layer, they have **a local myorelaxing effect**, for which they are called "Botox-like peptides".

According to clinical studies, the use of ultra-short peptides for 1 month increases skin smoothness by an average of **21%**, reduces the number of fine lines by **15-20%** and improves skin microrelief in more than 80% of subjects.

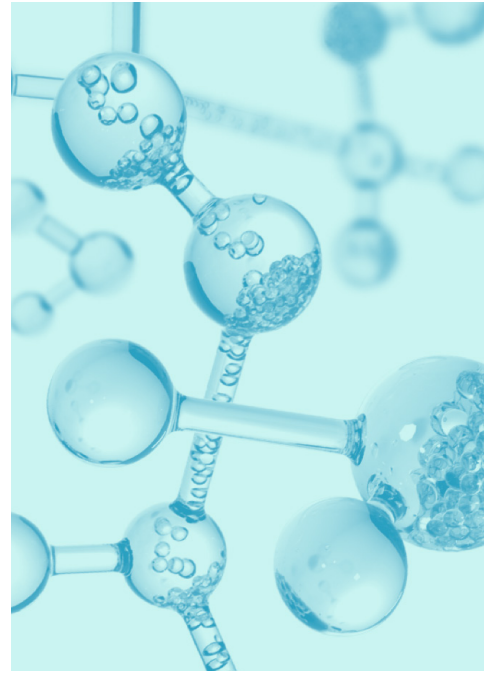

Hyaluronic Acid

Age-related loss of hyaluronic acid is one of the leading causes of moisture loss, skin laxity and wrinkles. Meamo labs Jeunetique Eyes Pro injections rehydrate soft tissues, restore lost volume and reduce the appearance of wrinkles. Adequate hydration of the skin increases the activity of fibroblasts. This triggers **the synthesis of new collagen.**

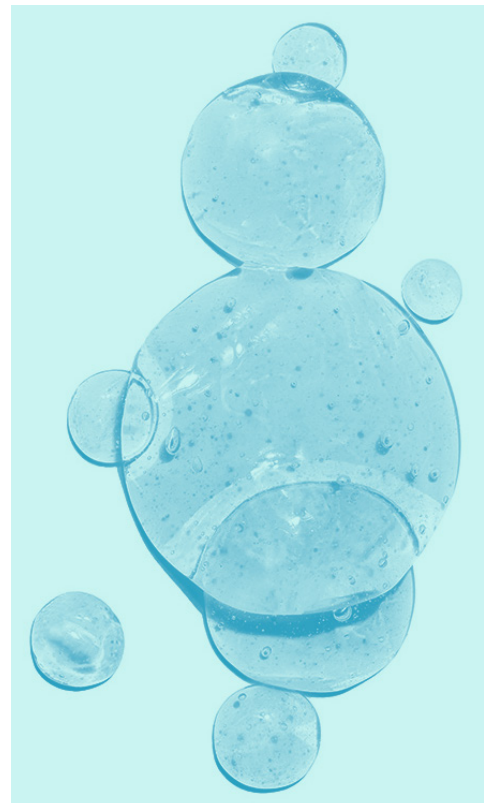

An additional advantage of Eyes Pro



is that it contains hyaluronic acid in the form of a hydrolyzed gel. It dissolves more slowly in the tissues, which prolongs the rejuvenating effect of the preparation

How Soon Will You See Results?

The first changes will be visible 3-4 days after the first injection. This time is necessary for the skin cells to react to the action of PN and USP. Fine lines and wrinkles under the eyes will gradually begin to smooth out.

The ingredients of Meamo labs Jeunetique Eyes Pro have a cumulative effect. Therefore, **the results will increase with each new injection.** After the second treatment, the dark circles under the eyes will gradually begin to disappear. At the end of the course, the lifting effect will be noticeable due to the formation of a new collagen matrix.

Only 1 session per month!
Meamo labs Jeunetique Eyes Pro
Course of Treatment

For optimal results, it's recommended to undergo **3-5 sessions,** with each session spaced **4 weeks apart.** With each session, the effects will be enhanced, offering a more youthful and refreshed appearance.

Where and How to Use Meamo Labs Jeunetique Eyes Pro?



Injection Techniques of Jeunetique Eyes Pro

Meamo Labs Jeunetique Eyes Pro is specially designed to rejuvenate the delicate skin under the eyes. It should be injected directly into the area beneath the lower eyelids, covering the entire **periorbital area** to target puffiness, dark circles, and fine lines.

1. Superficial Mesotherapy Technique

● Needle Type

Use **ultra-fine needles** to minimize discomfort. The recommended needle is a **33-34G, 4-6 mm** in length.

● Injection Depth

Injections should be superficial and subcutaneous, at a depth of **1-2 mm**. The needle should be held at a **45-degree angle**, with the bevel facing upwards to ensure precise placement.

● Injection Sites and Volume

From **0.05 to 0.2 ml** of the product should be injected over the entire **periorbital area**, with a depth of **1-2 mm**.

● Total Dosage

The total dosage for each eye is typically **1 ml** to achieve optimal results.

2. Infiltrative Technique

● Needle Type

Use **ultra-fine needles** to minimize discomfort. The ideal needle is a **33-34G, 4-6 mm** in length.

● Injection Method

0.1 ml of the product is injected directly into the "bag" under the eyes, pinched between the fingers in a fold. The injection is performed at a depth of **2-4 mm**.



What to do after the injection?

Minimal Recovery Period!


After receiving Meamo labs Jeunetique Eyes Pro injections, you can resume daily activities the following day. Here's what to keep in mind:

● Aftercare



Light makeup and gentle washing of the face are allowed **the next day**.

● Possible Reactions

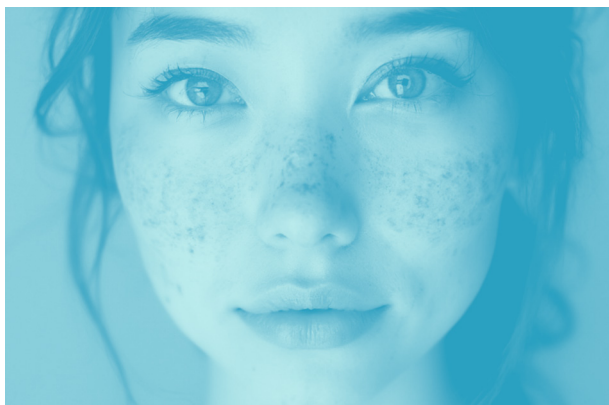

Redness and mild swelling may occur but will subside within **1-2 days**.

● Avoid



Heavy drinking, sauna, hot baths, and intense exercise for **7 days** post-treatment.

● Alleviate discomfort



Ice packs can help alleviate discomfort and speed recovery.



Results from scientific studies and clinical trials

A large-scale 2022 survey of 557 aesthetic physicians found that more than **96%** use PN to improve the appearance and rejuvenate the skin of their patients.

In 2016, PN were shown to improve the skin in several ways:



eliminate excessive keratinization



reduce pore size



even out skin tone



eliminate wrinkles



make skin more elastic and firm



88% of the physicians surveyed rated the results of using PN as "effective" or "very effective". In addition, scientists have not recorded a single case of serious complication in any of the studies conducted to date, underscoring the safety of PN-based products.

How to Store Meamo labs Jeunetique Eyes Pro?

Precautions for safe storage and use of the advanced bioreparant Meamo labs Jeunetique Eyes Pro

Keep away from direct sunlight
and out of reach of children.



Store in a cool,
dry place below 30°C.







Do not freeze.



Only use if packaging is intact.





 www.meamo.space
 info@meamo.co
 [@meamolab](https://www.instagram.com/meamolab)
 +82 (70) 4513-1496