

ATOM 250 WINCH USER GUIDE

1



Position the wheelchair centrally at the bottom of the ramp. Holding one of the winch remote controls in your hand, enter the vehicle and locate the winch unit.

2



Press the 'OUT' button on the remote control and guide the winch belt out of the vehicle and down the ramp towards the wheelchair.

3



Click the wheelchair attachment (either T bar or Y harness) onto the end of the winch belt.

4



Secure the wheelchair attachment to the wheelchair. The pictures shows a tongue & buckle fitting but it could be a J hook or karabiner. Ensure you are using the correct anchor points. Every wheelchair is different, refer to the wheelchair handbook if you are not sure where they are.

5



Locate and press the switch that activates and releases the wheelchair restraints.

6



Guide the restraint belts out of the vehicle, down the ramp and attach them to the wheelchair frame. Once attached, press the switch again to lock the restraints from further release.

7



Using the winch remote, press the 'IN' button, just until the webbing becomes taut from the winch unit to the wheelchair.

8



Move behind the wheelchair and hold the handles. Press the 'IN' button on the remote control and slowly enter the vehicle. It is important to remain behind the wheelchair for guidance at all times.

9



Use the 'IN' and "OUT" buttons on the winch remote as necessary, until the wheelchair is positioned correctly inside the vehicle.

10



Disconnect the winch attachment from the wheelchair and the winch unit, and store it somewhere safe and secure. Ensure the front restraints are still attached to the wheelchair correctly. Then fit the remainder of your restraint system, referring to your restraint handbook for more information should you need to.

