



Cordclear® Instructions for Use

For In-Patient Beds and Operating Tables
Cordclear® Ag – Multi-Use Cord Management Product

Product Composition

- SABIC Noryl™ 731S (Polyphenylene Ether + Polystyrene)
- Million Guard™ PG721S Silver Glass (Antimicrobial Additive)

Purpose

Cordclear® provides a non-fixed, reusable holder for organizing and storing electrical cords and cables on in-patient hospital beds and operating tables when not in use.

Directions for Use

1. Preparation

- Remove Cordclear® from packaging.
- Wipe with a hospital-approved germicidal wipe before first use.

2. Application

- In-Patient Beds: Attach Cordclear® to the bottom of the handrail opening (preferably the lowest point) or as needed on the headboard, side rail, or footboard.
- Operating Tables: Attach to the metal side rail or a remote hanging apparatus.

3. Cord Placement

- Gather cords to be secured and pull taut toward the Cordclear® hook.
- Loop cords together and hang the loop over the Cordclear® hook.

Cleaning Instructions

Before first use, between patients, and between cord storage cycles, clean with hospital-approved germicidal wipes per facility disinfection standards. If visibly soiled, wash immediately with soap and water.

Storage Instructions

Keep Cordclear® attached to the bed or table for immediate access. If not in use, store in a cool, dry location, protected from excessive heat, cold, or moisture.

Disposal


Dispose of used Cordclear® in general non-hazardous waste.

cordclear_{Ag}

Important Information

- For hospital staff use only – not intended for patient use.
- Designed to hold electrical cords only, during storage or transport.
- Weight tested to 10 lbs. Do not exceed this limit.
- In-Patient Bed Model: holds up to three cords.
- Operating Table Model: holds one cord.
- Bed model is symmetrical and reversible (either end up).
- Operating Table model: square hook end attaches to the table; rounded hook end holds the cord loop.

Proposition 65 Notice

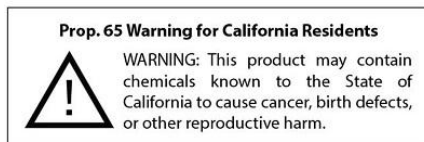

Cordclear® products have passed testing for Phthalates, Cadmium, and Lead. This warning is included out of an abundance of caution.

For Cordclear Support

Visit: www.cordclear.com

Manufacturer:

Cordclear, LLC.

Las Vegas, NV 89148

www.cordclear.com | info@cordclear.com

Designed in the USA. Made in China.

IFU-2025 Rev B – Updated October 2025