

cordclear_{Ag}

Cordclear® Instructions for Use

For In-Patient Beds and Operating Tables
Cordclear® Ag – Multi-Use Cord Management Product

Product Composition

- SABIC Noryl™ 731S (Polyphenylene Ether + Polystyrene)
- Million Guard™ PG721S Silver Glass (Antimicrobial Additive)

Purpose

Cordclear® provides a non-fixed, reusable holder for organizing and storing electrical cords and cables on in-patient hospital beds and operating tables when not in use.

Directions for Use

1. Preparation

- Remove Cordclear® from packaging.
- Wipe with a hospital-approved germicidal wipe before first use.

2. Application

- In-Patient Beds: Attach Cordclear® to the bottom of the handrail opening (preferably the lowest point) or as needed on the headboard, side rail, or footboard.
- Operating Tables: Attach to the metal side rail or a remote hanging apparatus.

3. Cord Placement

- Gather cords to be secured and pull taut toward the Cordclear® hook.
- Loop cords together and hang the loop over the Cordclear® hook.

Cleaning Instructions

Before first use, between patients, and between cord storage cycles, clean with hospital-approved germicidal wipes per facility disinfection standards. If visibly soiled, wash immediately with soap and water.

Storage Instructions

Keep Cordclear® attached to the bed or table for immediate access. If not in use, store in a cool, dry location, protected from excessive heat, cold, or moisture.

Disposal


Dispose of used Cordclear® in general non-hazardous waste.

cordclear_{Ag}

Important Information

- For hospital staff use only – not intended for patient use.
- Designed to hold electrical cords only, during storage or transport.
- Weight tested to 10 lbs. Do not exceed this limit.
- In-Patient Bed Model: holds up to three cords.
- Operating Table Model: holds one cord.
- Bed model is symmetrical and reversible (either end up).
- Operating Table model: square hook end attaches to the table; rounded hook end holds the cord loop.

Proposition 65 Notice

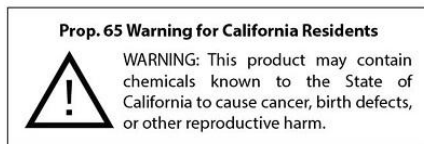

Cordclear® products have passed testing for Phthalates, Cadmium, and Lead. This warning is included out of an abundance of caution.

FDA Disclosure

Cordclear, LLC products are designed for organizational and safety purposes only. They have not been evaluated by the U.S. Food and Drug Administration (FDA), and Cordclear makes no medical claims regarding their use.

For Cordclear Support

Visit: www.cordclear.com

Manufacturer:
Cordclear, LLC.
Pala, CA 92059
www.cordclear.com | info@cordclear.com

Designed in the USA. Made in China.

IFU-2025 Rev B – Updated January 2026