



DMD CLEARFIT ANESTHESIA MASK

INSTRUCTIONS FOR USE

MR

PREPARED BY DELTA MEDICAL DEVICES

INTENDED USE

DMD ClearFit masks are intended to direct anesthesia or medical gases into the patient's upper airway without introducing any apparatus into the mouth or trachea. This device is designed for use in hospital and pre-hospital settings. This mask should only be used by appropriate medical personnel.

This guide covers the instructions for use of the DMD ClearFit masks.

MASK SPECIFICATIONS:

Disposable
Non-sterile
Non-conductive
Single patient use
Phthalate-free
Latex-free
MRI safe

INSTRUCTIONS:

1. Remove the mask from the package and inspect the device for any damage. If any damage or deformation is found, discard the mask and obtain a new mask.
2. Check the cushion's inflation level to ensure an appropriate fit and seal to the patient's face.
3. If present, the mask collar may be removed if it is not needed
4. Discard the mask after use; DO NOT re-use.



Warnings

- Keep the mask away from sharp objects that could damage the cuff
- Restricted use to only one patient – reuse will cause cross-infection.
- This mask is not designed to be used with automatic cleaning or sterilization processes; this will cause damage to the mask and leave harmful residues.
- Do not use on patients with any allergies to plastics.
- Not intended for use as a respirator mask or with respirators

Caution

- Over-inflation or excessive pressure on the mask to hold it in-place may restrict or close the patient's airway.
- Under-inflation may result in poor seal and leakage
- This mask is designed to connect to a standard fitting.
- Restricted to use by licensed health care practitioners