

skytron. 3603 UltraSlide Surgical Table

Loaded weight distribution instructions – Warning: Do not exceed

The maximum lifting capacity of the 3603 table is 450 kg and the maximum articulation weight capacity is 360 kg. When lifting or articulating large patients, pay close attention to the patient position as well as the positioning guidelines and limitations listed in the operation instructions. **Caution:** accessories may limit weight capacities.

DO NOT unlock brakes when a patient is on the table. Uneven patient weight load may cause instability.

If circumstances demand table brakes be unlocked with a patient aboard:

Patient must be cantered and evenly distributed on the table in a supine or prone position with the table lowered to its lowest height position.

Table top slide must be cantered (indicated by a red light on the pendant control) prior to unlocking brakes.

Patient's head must be on the head section. Head section must be attached in its normal orientation.

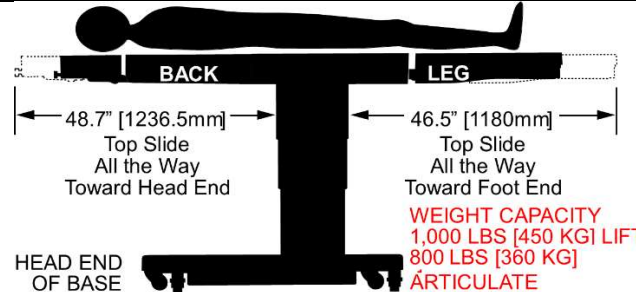
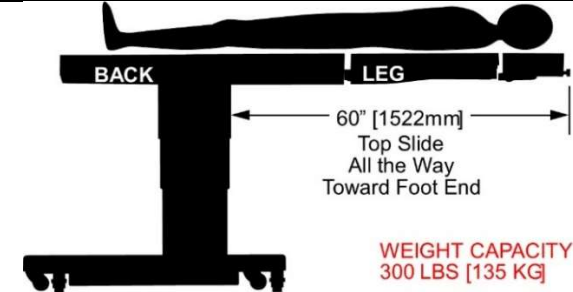
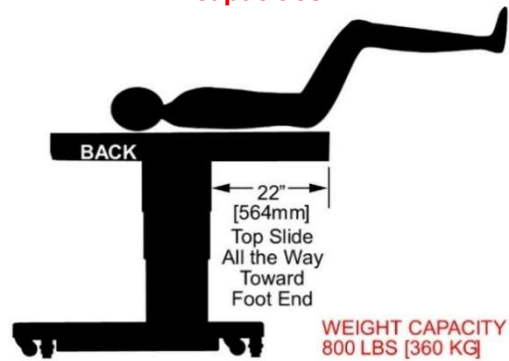
Prior to unlocking brakes, check for obstructions on the floor that might prevent the table from moving smoothly to a new location.

Re-lock the brakes immediately once the final position is reached and before commencing surgery.

To move the table safely, one staff member should be positioned at the head end and one at the foot end. If the patient weight exceeds 114 kg, four (4) staff members are required to move the table and ensure patient safety.

If patient weight exceeds 227 kg, the brakes should remain locked at all times.

DO NOT reverse the patient on the table without first consulting with SKYTRON product management.

 BACK LEG 48.7" [1236.5mm] Top Slide All the Way Toward Head End 46.5" [1180mm] Top Slide All the Way Toward Foot End WEIGHT CAPACITY 1,000 LBS [450 KG] LIFT 800 LBS [360 KG] ARTICULATE HEAD END OF BASE Suitable for heavy patients	 BACK LEG 60" [1522mm] Top Slide All the Way Toward Foot End WEIGHT CAPACITY 300 LBS [135 KG] Not suitable for heavy patients
MOVING – Table must be lowered to lowest position & centred above base of table / Breaks off (only if under 227kg) then moved into position / Breaks on and bed raised into final position. 	Certain accessories may limit weight capacities.  BACK 22" [564mm] Top Slide All the Way Toward Foot End WEIGHT CAPACITY 800 LBS [360 KG]

 skytron. 3603 UltraSlide Surgical Table

Loaded weight distribution in-service -

Please sign upon successful completion of training relative to the Skytron 3603 UltraSlide Surgical table weight distribution limits and operational considerations as shown within this document. In signing, you also agree that of a copy of this document has been given to you and that you know where to access it as needed.

[illegible]