

AquaSept™/AquaFit™ Limited Warranty Policy

Scope of Warranty

Genesis Dental Technologies, LLC, manufacturer of AquaSept/AquaFit (“Genesis”) warrants to the original retail purchaser that the AquaSept/AquaFit branded products will be free from defects in material and workmanship under normal use and service for a period of one year from the date of invoice. Warranty includes parts and labor. Shipping costs are excluded. This limited warranty shall only apply to defects that are reported to Genesis within the warranty period and which, after examination by Genesis, prove to be defective. If Genesis finds the product to be defective during the applicable warranty period, Genesis shall correct any such defect by either, at Genesis’ sole option, repairing or replacing any defective or non-conforming product, or defective or damaged part, as applicable. This warranty is not transferable or assignable.

Discontinued parts and accessories, and parts not supplied by AquaSept are excluded from this limited warranty. Genesis shall not be responsible for dismantling or re-installation of any AquaSept/AquaFit part. Any defects in a repaired or replaced part shall be covered to the extent of the unexpired term of the original warranty period. Warranty does not cover consumable parts, or normal wear, or any goods that have been supplied by any company other than Genesis. Please refer to the AquaSept website for updated warranty and product specifications (www.aquasept.com).

Exclusions-Limitation of Liability

Genesis shall be liable under this limited warranty only for defects that appear under normal conditions of operation and proper use. Genesis shall not take any responsibility whatsoever for the product if stored or moved carelessly; tampered with, altered, or misused. This limited warranty is void and Genesis disclaims all liability if customer fails to seek and obtain repair or replacement in a timely manner, or if the product or part is excessively used. In addition, if the Product has been installed, serviced, or repaired by anyone other than an authorized service provider or Genesis, any defects caused or contributed to the Product by these unauthorized parties will void this warranty.

This limited warranty does not cover, and Genesis shall not be liable for, accessories or parts not manufactured by Genesis. Genesis shall not be liable for products that are not properly cleaned and maintained as

required or recommended by Genesis in the AquaSept/AquaFit installation and/or operation manual. This limited warranty shall not apply to customer use outside of the Manufacturer’s recommended use. Genesis shall not be liable for damages of any kind, under any legal theory arising out of or in connection with any non-intended use of the AquaSept/AquaFit products. Any off-label use shall be the sole responsibility of the customer, who will be held singularly liable for damage, injury, or other loss.

EXCLUSIVE REMEDY; DISCLAIMER

GENESIS’ ONLY OBLIGATION UNDER THIS LIMITED WARRANTY IS THE REPAIR OR REPLACEMENT OF DEFECTIVE PARTS. GENESIS SHALL NOT BE LIABLE FOR AND HEREBY DISCLAIMS ANY DIRECT, SPECIAL, INDIRECT, INCIDENTAL, EXEMPLARY OR CONSEQUENTIAL DAMAGES OR DELAYS, INCLUDING BUT NOT LIMITED TO, DAMAGES FOR LOSS OF PROFITS OR INCOME, LOSS OF USE, DOWNTIME, COVER AND EMPLOYEE OR INDEPENDENT CONTRACTOR WAGES, PAYMENTS, AND BENEFITS. THIS LIMITED WARRANTY IS GENESIS’ ONLY WARRANTY FOR THE AQUASEPT/AQUAFIT PRODUCTS AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED. GENESIS MAKES NO IMPLIED WARRANTIES OF ANY KIND INCLUDING ANY IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.

No agent, distributor, dealer, or other party is authorized to make any warranty on behalf of Genesis related to AquaSept/AquaFit products or to assume any other liability with respect to its products.

STATUTE OF LIMITATIONS; CAP ON DAMAGES

No action may be brought against Genesis for breach of this limited warranty for the AquaSept/AquaFit products or for any claim arising out of or relating to the products, more than ninety (90) days following expiration of the limited warranty period. Genesis’ total liability to the purchaser shall not exceed the original product price paid by the purchaser to Genesis.

AquaSept™



Installation Instructions



20-70-114-INS F

Genesis Dental Technologies, LLC
200 Main Street
Elmwood, WI 54740




888.539.3907 | info@aquasept.com | www.aquasept.com

©2023 Genesis Dental Technologies, LLC. Patented.

Installation Instructions for AquaSept™ High Speed Handpiece System

Installation Instructions:

- 1. Install the Quick Connect following the Mounting Instructions.
- 2. Fasten Quick Connect air supply tube to turbine drive air hose barb on dental unit. **Do not operate handpiece above 40 psi.**
- 3. Turn off, disconnect or plug existing water and chip air.

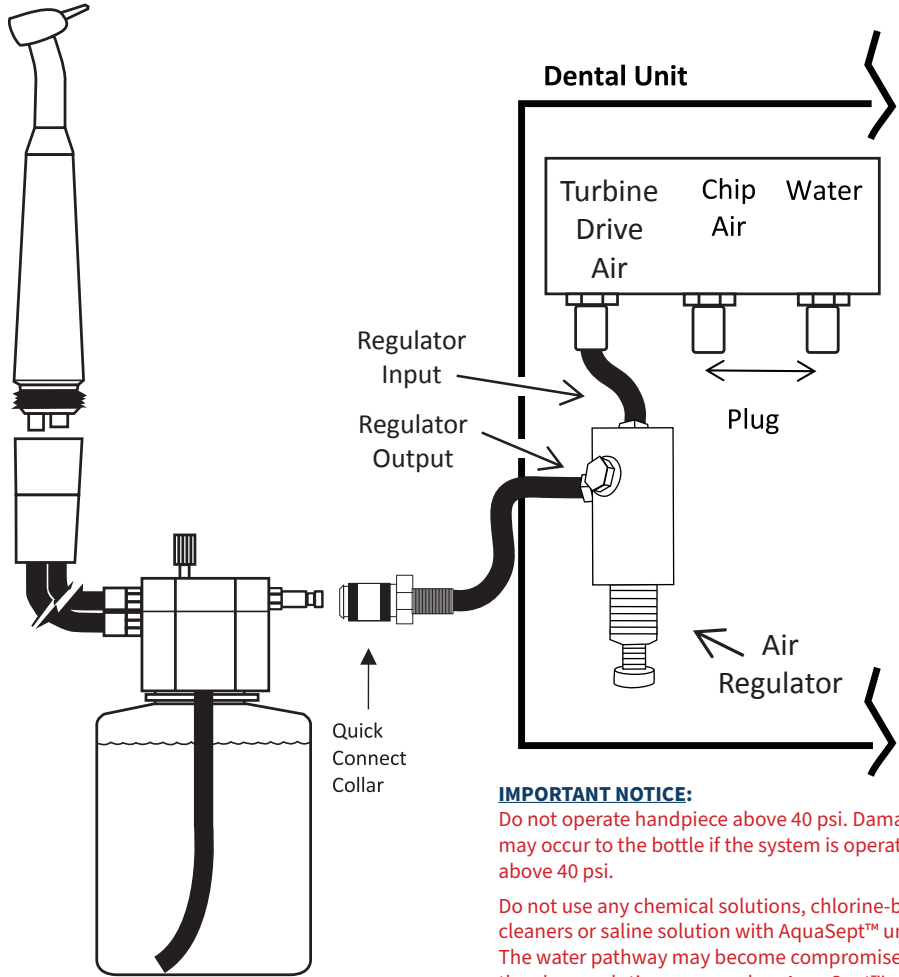

IMPORTANT NOTICE:

Do not operate handpiece above 40 psi. Damage may occur to the bottle if the system is operated above 40 psi.

Do not use any chemical solutions, chlorine-based cleaners or saline solution with AquaSept™ units. The water pathway may become compromised if the above solutions are used on AquaSept™ units.


Visually inspect your AquaSept™ system for tears or holes in the tubing before each use. If any of these defects are observed, do not use the system. If any other defects are observed, do not use the system. Contact technical support at 888.539.3907 for instructions on how to return the system for repair.

CAUTION: Do not plug a High Speed Handpiece System into a syringe mount, the turbine will rotate immediately.

Installation Instructions for AquaSept™ Air / Water Syringe System

Installation Instructions:

- 1. Install the Quick Connect following the Mounting Instructions.
- 2. Turn off master air supply to dental unit.
- 3. Connect dental unit air supply to Regulator Input.
- 4. Turn on air supply.

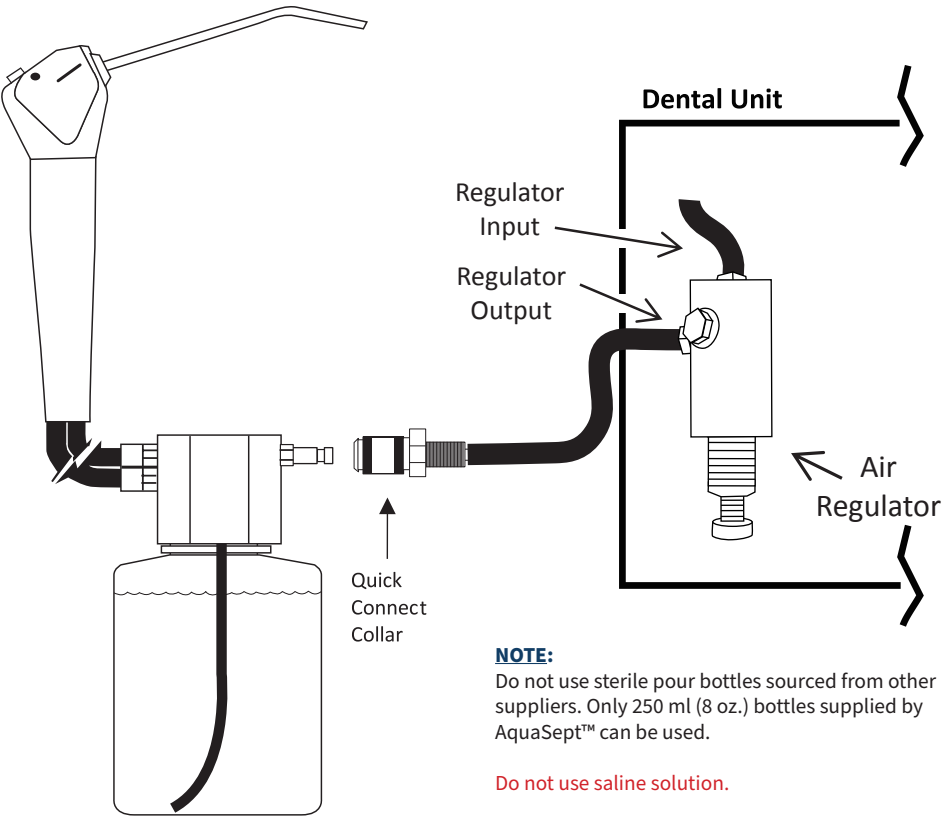


NOTE: Do not use sterile pour bottles sourced from other suppliers. Only 250 ml (8 oz.) bottles supplied by AquaSept™ can be used.

Do not use saline solution.

CAUTION: Regulator is fixed to deliver 12-15 psi air to syringe rather than line pressure. Regulator must be used. Supply flow restrictor (if used in the unit) should be opened or bypassed.

Mounting Instructions & Options

Panel Mount Instructions:

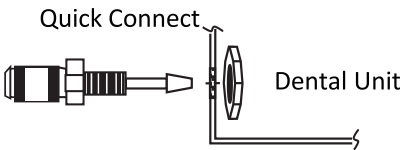

Drill a 7 mm hole into dental unit, remove nut and insert quick connect through hole. Fasten nut onto quick connect with 11 mm wrench.

The AquaSept™ system is operated by air through a quick connect. There are two mounting options: Panel Mount or Hanger Mount.


Panel Mount Instructions:

- 1. Follow the Panel Mount Instructions in diagram.
- 2. Proceed to the High Speed Handpiece or Air / Water Syringe System Installation Instructions.

Hanger Mount Instructions:

- 1. Follow the Hanger Mount Instructions in diagram.
- 2. Select a hanger mount option: Swing Arm, Standard or Surface.
- 3. Follow the appropriate diagram for the selected option.
- 4. Proceed to the High Speed Handpiece or Air / Water Syringe System Installation Instructions.


Hanger Mount Instructions:

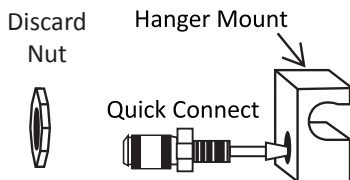

Remove nut from quick connect and discard. Thread quick connect into hanger mount with 12 mm wrench.


Hanger Mount Options

Swing Arm:

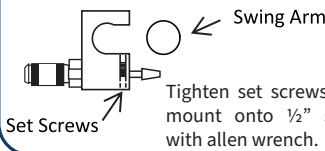


Tighten set screws of hanger mount onto 1/2" swing arm with allen wrench.

Standard:

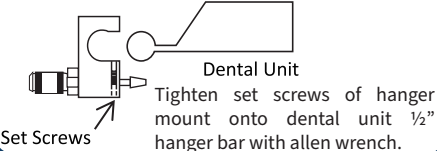

Tighten set screws of hanger mount onto dental unit 1/2" hanger bar with allen wrench.

Surface:

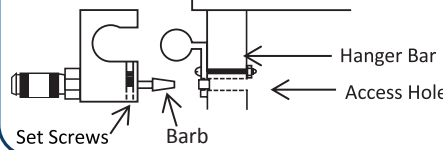

Mark where barb exits hanger mount. Drill a 9 mm access hole on this mark. Thread hose through access hole and fasten tubing to barb. Install hanger mount and tighten set screws of hanger mount onto 1/2" hanger bar with allen wrench.