

Operation Procedure

The Inject-Safe™ Barrier Bandage is a patented “pre-procedure” self-adhesive bandage that would be applied to a patient’s skin PRIOR to being given an injection by a healthcare provider for the purpose of containing any post injection bleeding.



Step 1 Prepare injection site with alcohol wipe.



Step 2 Remove Inject-Safe™ Barrier Bandage from sterile pouch.




Step 3 Hold the tab on the Inject-Safe™ Barrier Bandage & remove paper backing.



Step 4 Apply Inject-Safe™ Barrier Bandage to skin without touching the membrane area of the Inject-Safe™ Barrier Bandage.



Step 5 Wipe the exterior of the barrier with the alcohol wipe if you touch the center of the barrier.

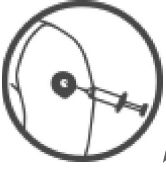

Step 6 Inject needle through the center of the Inject-Safe™ Barrier Bandage.



Step 7 Remove and safely dispose of needle in a safe container.

**When the needle is removed, the Inject-Safe™ Barrier Bandage self seals and protects both the patient and healthcare provider from possible exposure to blood from the injection site*

**Meets injection administration guidelines set forth by the American Pharmacist’s Association*