

Wellness[®]+

p l u s

START LIVING A

**PAIN
FREE
LIFE!**



For more information contact:

Health and Wellness Systems, Inc.

Michael Lofrano, customer service

customer@globalwellnesspro.com

www.globalwellnesspro.com

732-276-7500 Code# GWP 1246

ElectroMedical
technologies[®]



WellnessPro® Plus

Get Results™

Wellness⁺
plus^{pro}

#1 CHOICE OF HEALTH CARE PROFESSIONALS

The Problem: Pain is a silent epidemic

- Over 60 million Americans live with chronic pain caused by a disease, a disorder or an accident
- An additional 25 million people suffer acute pain resulting from surgery or accident
- 4 out of 10 chronic pain patients are currently not seeing a physician for pain relief as they think there is nothing more their doctor can do

The Solution: WellnessPro® Bio Electrotherapy



The last decade has seen a major shift in the way Americans view medicine. Despite the benefits of pharmacological treatments and surgical procedures, the associated side effects and addictions have demonstrated a need for additional approaches to patient care. Today, a huge number of patients are turning to complementary therapies like acupuncture, chiropractics, nutritional superfoods, physical therapy, yoga and psychology.

In the United States, more and more people are discovering alternatives to drug treatments and have found a very powerful and effective modality that works - Electromedicine! The bio electrotherapy system known as the WellnessPro® is a complementary treatment modality that has been FDA cleared and is a safe and effective device for pain management.

If you want to find the secrets of the Universe, think in terms of energy, frequency and vibration. -Dr. Nikola Tesla

ElectroMedical
technologies®

THE ISSUES

**83
MILLION**

people report that pain frequently affects their participation in daily activities.

28%

feel that there was no solution for their pain.

4 OUT OF 10

**PEOPLE
SUFFERING**

from moderate to severe pain are unable to find adequate pain relief.

**A New Breakthrough in Pain
Management is HERE!**

An increasing number of clinics and private practices, including practitioners in the allopathic and natural healing areas of medicine, are discovering a device that is unlocking endless possibilities. The WellnessPro® has shown to be the most effective bio frequency device on the market and is the #1 choice of professionals for the relief of pain, even in the most difficult cases.

What is Electromedicine?

Electromedicine is a discipline within the field of medicine that deals with the use of electricity and frequencies to aid in the treatment of chronic or acute pain conditions associated with a variety of physical ailments.

- **Non-invasive**
- **Non-toxic**
- **Safe**
- **Effective**

**Ask your doctor TODAY
and Live Pain FREE!**



(888) 880-7888

www.electromedtech.com

How Does it Work?

The human body is like a machine; it wears out in direct proportion to its abuse. A well balanced body reflects a strong immune system. Longevity is the result of natural care and maintenance of the body.

In basic terms, electricity is found naturally in all of us. Certain electrical impulses in our bodies help facilitate bodily functions including actions needed for healing. By mimicking the electrical impulses that occur in us, we can help facilitate a specific effect.

Electromedicine is able to trigger these impulses by varying the frequency, wave length, intensity, and location of the electricity applied to the patient. Understanding how these elements interplay to create a desired effect is the basis for the science.

"I had a patient come to me complaining of pain in the ball of her right foot for a year. She did not remember when the pain started or how. She has seen several doctors, physical therapists, and a chiropractor. She also used several kinds of rubs and nothing was helping. The doctor I work with referred her to me for a treatment. I treated her for 3 days. That was 3 months ago, and she has had no pain since treatment! She is promoting the WellnessPro® to all her fellow workers and friends."

Nancy R.



Wellness[®]
plus^{pro}

Here's what people are saying about the WellnessPro® unit!

"It's a Miracle Instrument!"

"The WellnessPro® has been a God-send for me. I have Fibromyalgia, and most things I've tried have failed. A friend suggested this unit to me; I didn't know what to think. I told her I'll give it a try. At first I felt a little improvement, the next day we tried a different frequency and the pain in my legs was gone within 10 minutes!! I am amazed to say the least!!! I may still have fatigue and some leg stiffness, but there is absolutely NO pain as long as I use the machine once a week. More people should know about this!"

Jan K.



"...No Pain Since Treatment!"



I have been plagued with back problems most of my adult life. I have gone through back surgery three times. Each time it has given me temporary relief; several years after the second surgery the pain returned. To my wife's dismay, our social activities began to fade away. I could not comfortably walk any distance, sit, or stand for an extended period of time.

My niece introduced me to this unit. She has used the WellnessPro® and was enthusiastic about the excellent results she has had using one. She begged me to try it before taking a chance on surgery again. The device was delivered to my home and I began using it immediately. No rash promises were made at the time the Electromedical device was left with me. In just a few days I began to feel improvement. In less than one week, the pain in my back and legs was gone. I feel a huge weight has been lifted from my mind and my body. Mentally and physically I feel alive again. I no longer dread going to bed, only to toss and turn in pain all night.

G. McLarney

"...I Feel Alive Again!"

Caution: Federal law restricts this device to sale by or on the order of a practitioner licensed by the law of the State in which he/she practices to use or order the use of the device.

Electro[®]Medical
technologies[®]

Electro[®]Medical
technologies[®]

Sports Medicine

Sports Medicine is a branch of medicine that cares for the prevention and treatment of sport-exercise related injuries. It plays a fundamental role in all athletics ranging from high school to the professional level. The WellnessPro® Plus is the premier choice of athletics professionals across the country and around the world. It is a revolutionary therapy device that achieves extraordinary results. Our proven track record has allowed the WellnessPro® Plus to become a vital component for physicians, physical therapists, chiropractors and athletic trainers. Most importantly, the WellnessPro® Plus provides FAST and LONG LASTING results that help players and teams quickly recover and **GET BACK IN THE GAME!**


CALL NOW and find out about special athletic program pricing!



The basic unit of life, the cell, exists only because it is held together by energy. The atoms and molecules that make up a cell also exist because their various parts are held together by a bond of energy. Essentially, everything in the universe as we understand it is energy. Your entire body is made up of “energy fields” or energy centers that form the foundation of your health. The body’s energy centers – meridian, chakras, sefirot, etc (depending on one’s belief system) – bring vital life energy to every organ in your body. Keeping these energy fields balanced and removing blockages in the flow of energy, plays a major role in determining your body’s health. Balance is a key concept within energy wellness. With the WellnessPro’s primary benefit of pain relief, your body can move to a state of harmony and energetic balance.

If every day you make sure your energies are not blocked and are flowing freely, your body’s innate healing system will counter virtually everything. Your body’s wellness reflects the health, clarity, and flow in your energy field. By relieving the pain, your body is able to stabilize and maintain a natural healthy energy flow that helps to create a state of physical and mental wellness.

The WellnessPro® instrument is for wellness. It does not have any side effects. It is non toxic and not invasive. It is FDA cleared for a form of pain relief: acute, chronic or post-operative. The WellnessPro is in perfect harmony with the new self-care model of natural health.

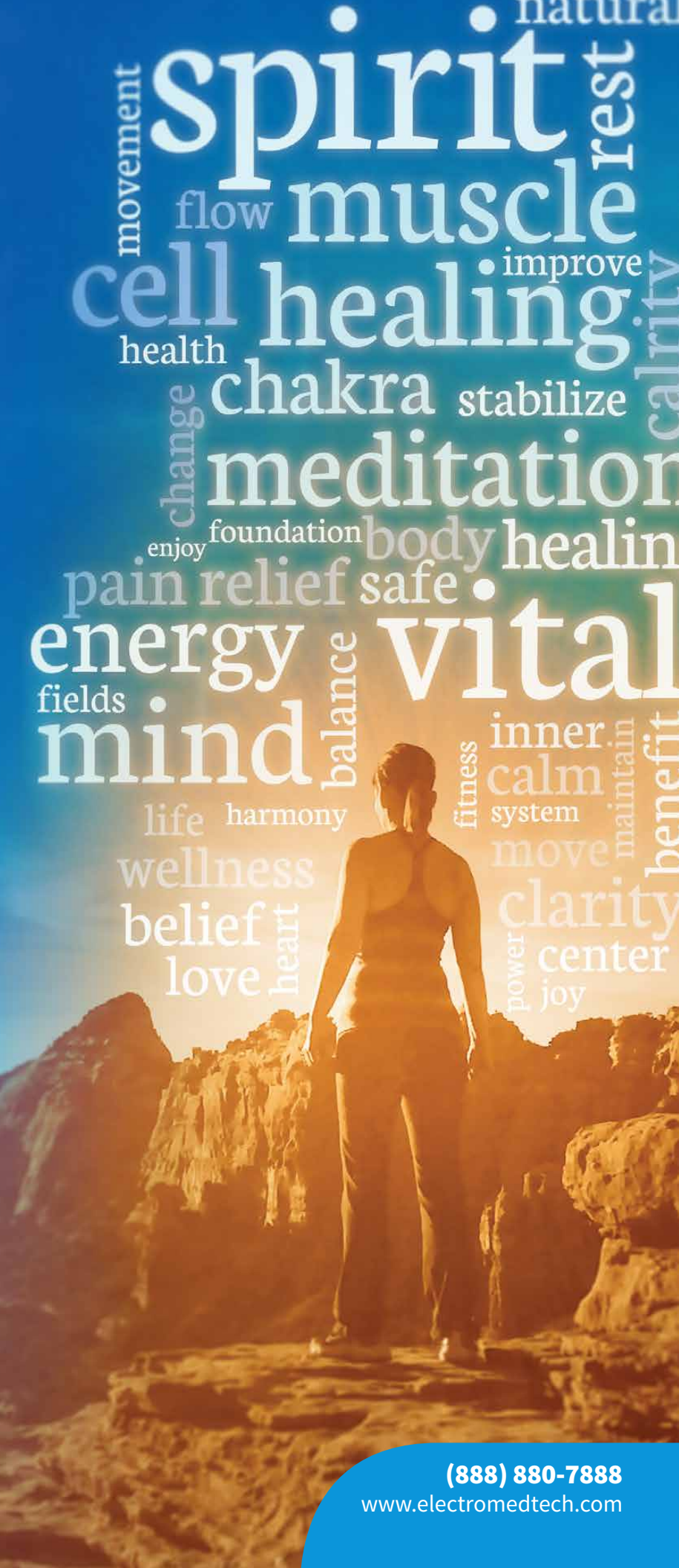

New Breakthrough

In Technology & Medicine

What makes WellnessPro[®] the #1 choice of health care professionals world wide?

- Easiest to use on the market! The built in "i-Guide™" system walks you through all steps of usage and programming.
- FDA cleared, CE 0459, ISO 13485, ETL, CAN/CSA, UL, IEC certified!
- Up to 8 lead outputs for treatment flexibility and multiple stimulation points.
- 7" easy-to-read color LCD display with detailed session information.
- STIMCARD™: Create, Backup & Store Personalized Patient Programs.
- Illuminated control panel for added safety and clarity.
- Portable and Convenient! The WellnessPro[®] can be used anywhere in the world (100V to 240V) and comes with a case to protect your investment.
- 3 Year Warranty (\$795 value) included with purchase.



Built with the future in mind!

- Designed to expand its capabilities and support future devices and attachments!
- One MILLION frequencies to choose from in TENS or Microcurrent mode!
- DeepPulse ION™ technology for unique frequency delivery to problem areas.
- FULLY independent outputs - Like having 2 units in 1.
- Enhanced and innovative wave types: Burst, Spectrum, Sweep, Duplex or Continuous.
- Multi Auto - run multiple auto codes without user intervention.
- DUAL mode (Interferential) - Frequency independent outputs
- FLUX mode - set frequency range parameters for unit to target.
- Dedicated extension port for new upcoming accessories and new devices


Electrotherapy Chronology

46AD

46AD (Ancient Rome) - Scribonius Largus describes the use of torpedo fish (aquatic animals capable of electrical discharge) for medical applications. "The live black torpedo when applied to the painful area relieves some chronic and intolerable protracted headaches... carries off pain of arthritis... and eases other chronic pains of the body."

1600 (England) - William Gilbert, physician to Queen Elizabeth, published De Magnete, in which he describes the use of electricity in medicine. Gilbert re-discovers that when certain materials are rubbed, they will attract light objects - originally known to be true of Amber by the ancient Greeks. He coins the name 'electricity' from the Greek 'electron' for amber.

1780 (Italy) - Galvani, Professor of Anatomy at the University of Bologna, first observes the twitching of muscles under the influence of electricity (prepared from the leg of a frog). Galvani then proves that atmospheric electricity, as manifested in lightning, will produce the same effects on muscular movement.

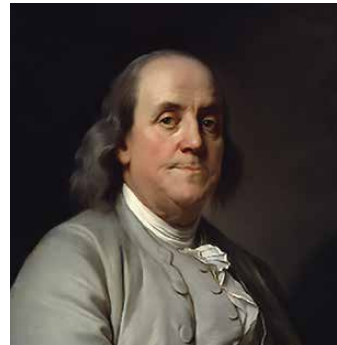


BENJAMIN FRANKLIN

1800 (Italy) - Carlo Matteucci shows that injured tissue generates electric current.

1840 (England) - England's first electrical therapy department is established at Guy's Hospital, under Dr. Golding Bird. The electrical discovery of Galvano leads to the use of mechanically pulsed Galvanic currents.

1891 (USA) - Nikola Tesla presents a paper in 'Electrical Engineer', about the medical application of high frequency currents. He notes that when the body is transverse by alternating currents above a certain frequency, heat is perceived.

1900 (Worldwide) - The electrical discoveries of Galvano, Faraday and Tesla were therapeutically adopted by activating the electrical phenomena of the human body with the use of Galvanic, Sinusoidal and Faradic currents. These become the standard methods of applying Electrical Body Stimulation. There becomes a preference to use the more comfortable and less hazardous Faraday method.

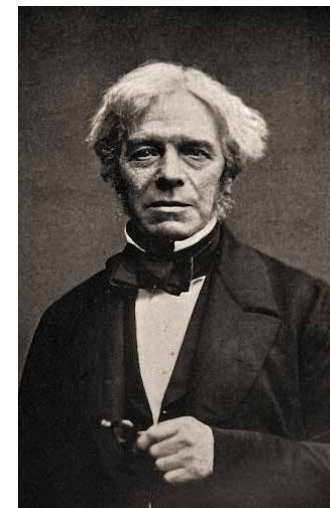

MICHAEL FARADAY

1908 (Germany) - Von Berndt, Von Priers and Von Zeyneck publish a paper on the treatment of the pain of joint disease by high frequency currents.



GEORGES LAKHOVSKY

1929 (Russia) - Lakhovsky publishes his book "The Secret of Life". Conducts studies and concludes that cells possess resistance, capacitance, inductance and function like tuned resonant circuits, capable of resonating to a resonant frequency when exposed to a range of frequencies.

1938 (USA) - Clinics are opened in San Diego, Los Angeles and Pasadena, California. Phenomenal results are reported. A special medical research committee is organized at the University of Southern California in Los Angeles to oversee and manage Dr. Rife's "energy medicine" research and clinical success. The newspaper articles are cautious but optimistic.




LUIGI GALVANI

1950's - 1990's (Russia) - "Russian Stimulation" is developed - 4,000Hz tone bursts, 50% ON, 50% OFF, alternating currents with 50 tone bursts per second. This is used by athletes for building muscle and increasing power.

1970's (England & Canada) - Melzak and Wall propose the 'Pain Control Gate' theory, by which strong afferent nerve stimulation by chemical, mechanical or electrical means overrides painful sensations at hypothetical pain control 'gates' in the spinal cord.

1970's (USA) - Transcutaneous Electrical Nerve Stimulation (TENS) is acknowledged as a viable method of pain management by America's Food and Drug Administration (FDA). Many American companies begin production of TENS devices. The heart pacemaker is developed.

1970's & 80's (Sweden) - Clinical researchers Ericsson and Sjolund publish a series of articles in various scientific and medical journals. They compare constant, high frequency "TENS to bursts of high frequency TENS (termed acupuncture-like TENS), finding that the latter offers better pain relief and does in fact instigate a release of endorphins into the bloodstream.

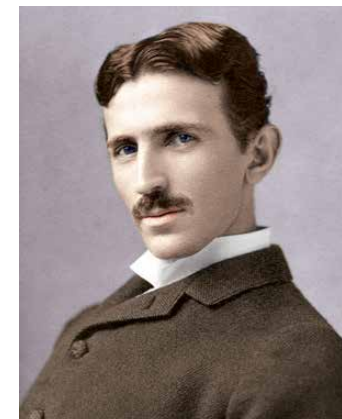

NIKOLA TESLA

1981 (USA) - Becker publishes "The Body Electric". Challenges the established mechanistic understanding of the body. Finds clues to the healing process in the theory that electricity is vital to life.

1991 (Germany) - Nobel Prize recipients Erwin Neher and Bert Sakmann developed a technique that detects electrical currents to the trillionth of an Ampere in the membranes of the cell, establishing the existence of ion channels.

2000 (USA) - John McDonald of Washington University uses intensive applications of Electrical Muscle Stimulation (EMS) to exercise the muscles of a quadriplegic of 8 years. The patient defies medical science by regaining limited sensation and movement in his body. The regimen includes 2 hours of EMS and hydrotherapy weekly.



DR. ROYAL RIFE

A Brief History of Electromedicine

Electromedicine, the use of electricity to treat physical ailments, is considered one of the oldest and most documented sciences known. Medical professionals of ancient Greece learned that the electrical impulses emitted from electric eels in clinical foot baths relieved pain and produced a favorable influence on the blood circulation. Doctors Largus and Dioscorides (cc 46 AD) documented substantial therapeutic results with electrical currents in the management of pain from a variety of ailments.

In the 1700s, European physicians used controlled electrical currents from electrostatic generators almost exclusively for numerous medical problems involving pain and circulatory dysfunction. During that period, Benjamin Franklin also documented pain relief by using electrical currents for "frozen shoulder."

Today electromedicine is recognized as a very safe, efficacious and certainly one of the most cost effective treatments of choice in 21st century that has ZERO side effects.



2007 July (USA) - After 4 years of development, Electromedical Technologies receives FDA clearance for the WellnessPro®! This new generation bio electrotherapy device combines decades of research with new breakthrough technologies in the electromedical field. It is also the only unit of its kind on the market to go through the rigorous laboratory tests for safety, accuracy and precision, receiving ISO13485, CE0459, ETL, UL and IEC marks. The WellnessPro® has "Deep Pulse ION™" Technology, a large frequency range and unique features that show astonishing results with patient's pain control issues; putting this device light years ahead of any other frequency unit on the market. The future of bio electro medicine begins.

1930 1950

1970

1980

1990

2000

What is a WellnessPro[®] 2010 PLUS device?

The WellnessPro[®] is an electrical stimulation device indicated for the relief of chronic, intractable pain and as an adjunctive treatment of post-surgical or post-traumatic acute pain. The unit is compact, lightweight, and very user friendly. The WellnessPro[®] is FDA cleared and CE0459 marked as a Class II medical device. It has been laboratory tested against ISO and IEC standards to ensure protection, accuracy and precision. The unit has the most flexibility of use and largest frequency range in its class on the market today. It is digitally controlled with a unique “Deep Pulse ION™” technology. Designed by world renowned scientists, the WellnessPro[®] has been created for the precise treatment to minimize the pain associated with various conditions.

What does PLUS stands for?

PLUS stands for “Built with the Future in Mind.” Electromedical Technologies will announce a series of NEW electrotherapy devices and attachments that are designed to work with the WellnessPro[®] Plus upon FDA clearance. These attachments and new devices are designed to upgrade, enhance and expand the modalities and capabilities of the WellnessPro[®]! The WellnessPro[®] Plus will open doors to all electrotherapy modalities and give you the most effective, innovative, upgradable and expandable device on the market today.

How does this device work to reduce pain?

The WellnessPro[®] Plus electrotherapy device produces the FASTEST LONG TERM RESULTS by alleviating acute and chronic pain symptoms. It is widely used around the world and is the premier choice of health care professionals, athletes, coaches, and medical research facilities.

WellnessPro[®] Plus impulses stimulate the peripheral nerves that send information about touch and vibration. The signals from the stimulated nerves interfere with pain signals traveling to the brain which reduces the brain’s perception of pain. In addition, the frequencies generated by the WellnessPro[®] stimulate endogenous opioids. These can eliminate pain through the same biological processes as prescription pain medications but without the harmful side effects.

Is this a TENS device?

YES. The WellnessPro[®] has been FDA cleared as a TENS (Transcutaneous Electrical Nerve Stimulation) device. These devices use electrical stimulation to relieve pain. Unlike other TENS devices, the WellnessPro[®] has a very large frequency range coupled with a unique delivery system which stimulates the body to block pain signals. The WellnessPro[®] has demonstrated superiority to other comparable devices by achieving longer term pain relief. The WellnessPro[®] allows more flexibility for users and practitioners.

Partial list of conditions that have been known to cause pain:

- | | | |
|-----------------------------------|--------------------------|-------------------------------|
| • Arthritis: Osteo and Rheumatoid | • Herpes/Shingles | • TMJ |
| • Carpal Tunnel Syndrome | • Chronic Fatigue | • Neuropathy |
| • Back Pain / Sciatica | • Wound pain | • Cancer |
| • Parkinson’s Disease | • Neurological Disorders | • Nerve Entrapment |
| • AIDS related Pain | • Multiple Sclerosis | • Bell’s Palsy |
| • Broken Bones | • Joint Pain | • Fibromyalgia |
| • Soft Tissue regeneration Pain | • Sports injuries | • Acute and Chronic Back Pain |
| | • Spinal cord injuries | • Headaches / Migraines |
| | • Plantar Fasciitis | • Disc injuries/Tendonitis |

NOTE: TENS devices are not recommended with all conditions. Ask your doctor to determine if the WellnessPro[®] is appropriate for you.

Can I buy the WellnessPro[®] for myself?

YES. If you are a licensed medical provider, proof of license must be provided. If a non-licensed individual would like to purchase the WellnessPro[®] for personal use at home, a prescription from a licensed medical provider must be on file. This is a FDA requirement for use in the United States only. No prescription required if the individual lives outside the United States. Additionally, if you do not wish to personally use the device and would like to become a sales representative for Electromedical Technologies, you can purchase a unit with distributor paperwork on file.

I have a specific condition. Can these treatments help me with my pain?

YES. Electromedical Technologies, the university research centers in Europe and participating US doctors/clinics have overseen treatments using the WellnessPro[®] and have had outstanding results in the reduction of pain for a wide range of acute and chronic conditions.

When can I expect results from treatments?

A licensed professional will attach either self adhesive or carbon rubber electrodes to specific areas of your body. An electrode gel may be applied to facilitate better conductivity of the device’s output. There should be no discomfort during treatment and you may or may not feel a slight tingling. Electrode placement and treatment duration should be determined by a licensed professional. Typical treatments can last between 10-60 minutes. Contact your practitioner for more information.

Are there side effects?

The most common side effect is the reduction and/or elimination of pain. Sometimes patients may experience atypical side effects including, but not limited to: diarrhea, night sweats, headaches, nausea, flu-like symptoms, etc. Make sure you consult your practitioner if you experience any symptoms.

If it’s so effective, why haven’t I heard of it before?

The technology has been available to the elite, practitioners, professional athletes and celebrities for over 60 years. It has been used for decades throughout the world and is common practice in various hospitals and clinics in Europe today. In the past few years, bio electrotherapy has become more available in the United States. It has quickly opened new possibilities and frontiers in the medical field for research and overall wellness.

What is the function of the WellnessPro 2010 Plus Microcurrent (µA)?

The human body functions through electro-chemical processes. Currents flow through cells, connective tissues, muscles, ligaments, and bone by neurological pathways to the brain. When an injury occurs at a particular site, the “Electrical Grid” is disrupted causing the healthy current flow to change pathway conductivity.

The WellnessPro Plus microcurrent (millionth of an amp) works with the body’s electric current on a cellular level. The unique sub-sensory micro currents increase the production of ATP, protein synthesis, oxygenation, ion exchange, absorption of nutrients, elimination of waste products and neutralizes the oscillating polarity of deficient cells. The WellnessPro Plus utilizes a unique combination of the largest range of frequencies and waves to achieve fast pain relief in the most difficult cases.

No other unit can compare.



Ask Your Doctor TODAY!

Start Living a Pain Free Life!

Your WellnessPro® pack includes:

- WellnessPro® 2010 Plus unit
- Universal power adapter (100-240V)
- 1 set of Large carbon-rubber electrodes
- 1 set of Small carbon-rubber electrodes
- 2 sets of self adhesive electrodes
- Tube of conductivity gel
- Operating manual
- 3 Year Warranty
- And MORE!



BUY NOW & SAVE

\$3,495



Recommended by doctors and medical professionals around the world!

The NEW generation WellnessPro® Plus bio electrotherapy unit combines decades of research from the original work of electromedical pioneers together with new breakthrough technologies in this field. The WellnessPro® Plus is also the only unit of its kind that upgrades and expands into various electrotherapy modalities giving you a truly universal electrotherapy device.

The unique breakthrough “Deep Pulse ION™” technology of the WellnessPro® has a frequency range and expandable therapeutic features that have generated astonishing results. It has helped relieve people’s pain around the world and has put this device light years ahead of any other frequency unit on the market!

ElectroMedical
technologies®

...Your Health & Wellness™

“The WellnessPro® money-back guarantee made my decision easy!”


Frank and Karen tried other frequency instruments on the market but were never satisfied with the results. To their amazement, they discovered an FDA cleared bio frequency unit - The WellnessPro®. The results were stunning!

Like them, you can try the WellnessPro® at absolutely no risk. With the 30-day money back guarantee. You have nothing to lose and much to gain from trying the WellnessPro®.

Live healthier and happier!



Order your Wellness Pro Plus Today!

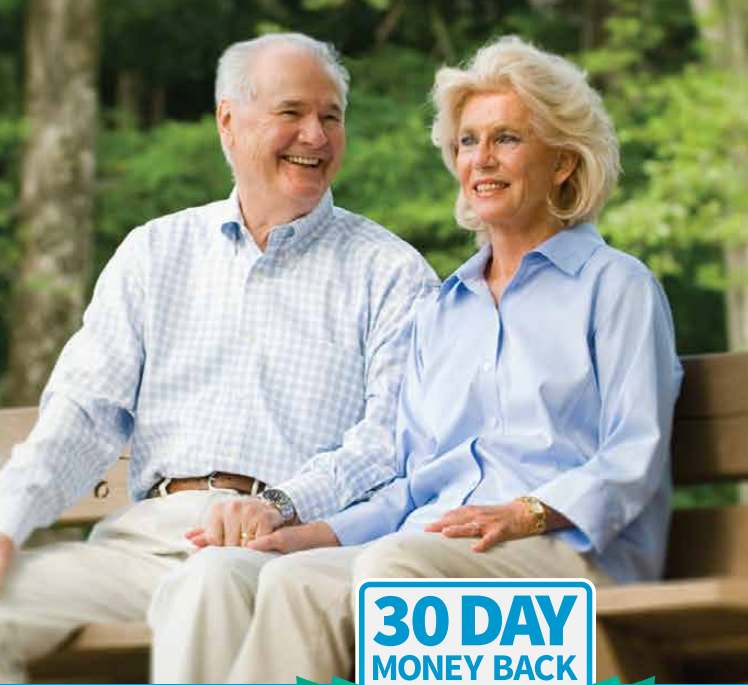

Customer Information:

Company Name:			
Contact Name:			
Shipping Address:			
			Suite Number:
City:	State/Province:	Postal Code:	Country:
Telephone:		Fax:	
Billing Address If Different:			
E-mail Address:			

Payment Information - Please choose one of the following:

Credit Card: <input type="checkbox"/> Visa <input type="checkbox"/> Master Card <input type="checkbox"/> American Express <input type="checkbox"/> Discover Card			
Credit Card Number:		Expiration Date:	
Name As It Appears On Card:		CVV Code:	
Money Order / Cash / Check Payment		Total: U.S.\$	
Make funds payable to: Electromedical Technologies			
Mail to: 3104 E. Camelback Rd #528 Phoenix, AZ 85016 OR Fax to: 480-452-1518			

Authorization:

By signing below, I agree to the terms and conditions and specifically authorize Electromedical Technologies (EMT) and/or its designee(s) to charge for product(s) in the amount of U.S. \$ [] (as indicated). I understand that EMT and/or its designee(s) will obtain Credit Card Authorization for the credit amount requested.		
Print Name:	Signature:	Date:

RX ONLY: Caution: Federal law restricts this device to sale by or on the order of a practitioner licensed by the law of the state in which he/she practices to use or orders the use of the device

(888) 880-7888
www.electromedtech.com

Unprecedented ground floor

Opportunity

for distributors & professionals!

- Highest Paid Industry Commissions
- Sub-distributor Sales Tracking Program
- Most Demanded Unit On The Market
- Proven To Provide the Fastest Results
- Easiest To Use
- Clinically Documented & Laboratory Tested
- Unbeatable Industry Guarantee
- Insurance Billable
- FDA Cleared
- European Union, ISO & Health Canada Certified

Promo Code:

GWP 1246

Contact Info:

Health and Wellness Systems, Inc.
Michael Lofrano, 732-276-7500
customercare@globalwellnesspro.com
www.globalwellnesspro.com



ElectroMedical
technologies®

©Copyright 2013 Electromedical Technologies