



Maiden Biosciences, Inc.

Collagen for Advanced Wound Care

MAIDEN BIOSCIENCES TOPICAL COLLAGEN POWDER WOUND DRESSINGS

PRODUCT DESCRIPTION:

Maiden Biosciences Topical Collagen Powder Wound Dressings are absorbent type 1 microfibrillar collagen pellets indicated for the management of moderate to heavily exuding acute and chronic wounds. The collagen powder dressings serve as a wound filler that is applied directly onto wounds to absorb excess exudate that may impair wound healing. The collagen dressings form a gel on contact with the wound surface to control minor bleeding, while maintaining a moist, physiologically favorable wound environment that encourages wound healing. The dressings consist of off-white loose collagen powder pellets packaged in convenient sterilized single-use 1gram peel open packets.

INDICATIONS FOR USE:

Maiden Biosciences Topical Collagen Powder Dressings are indicated for the management of moderate to heavily exuding wounds and to control minor bleeding.

Maiden Biosciences Topical Collagen Powder Dressings may be used for the management of exuding wounds such as: pressure ulcers, venous stasis ulcers, diabetic ulcers, acute wounds, e.g., trauma and surgical wounds, and partial-thickness burns.

DIRECTIONS: (Please use under the guidance of a healthcare professional)

TO APPLY:

1. Cleanse the wound following a wound appropriate preparation, procedures and guidelines.
2. Ensure the intact skin surrounding the wound is dry.
3. Peel open Maiden Biosciences Topical Collagen Powder packet(s) and apply sufficient powder to fill approximately ¼" depth of coverage within the wound bed. Do not overpack or compress powder to allow for expansion during absorption of wound exudate.
4. Apply appropriate secondary sterile dressing to secure and protect collagen powder placement.



Maiden Biosciences, Inc.

Collagen for Advanced Wound Care

CHANGING THE DRESSING:

1. Maiden Biosciences Topical Collagen Powder Dressings should be changed when fully saturated.
2. Remove Secondary Dressing.
3. Gently remove collagen gel with moist wipe or irrigation with approved wound cleanser or saline.
4. Perform wound cleansing and wound bed preparation per facility protocol.
5. Dry surrounding intact skin.
6. Reapply Maiden Biosciences Topical Collagen Powder Dressing Application Step 3.
7. Apply appropriate secondary sterile dressing to secure and protect collagen powder placement.

PRECAUTIONS:

Maiden Biosciences Collagen Topical Powder Dressings are intended for topical use only. Avoid inhalation and eye contact. For single use only. Do not reseal or re-sterilize. Discard open packaging and unused product per facility hazardous waste protocols.

Some reactions such as transitory pain, bleeding, blistering, swelling and redness have been reported in isolated cases using a similar product. If these conditions continue for more than 3-5 days, discontinue use of the product and notify physician.

CONTRAINDICATIONS:

Maiden Biosciences Collagen Powder Dressings should not be used on patients with known sensitivity or allergy to bovine Collagen.

Rx Only



See Instructions
For Use



Temperature
Limitation



Do Not Use
If Damaged



Keep
Dry



Do Not
Reuse

STERILE R

Sterilized Using
Irradiation



Not Made With
Natural Rubber Latex

Maiden Biosciences, Inc.

101 Lakeforest Blvd, Suite 390
Gaithersburg, MD 20877 USA

www.maidenbio.com

240.306.7272

MB Control No. Maiden-IFU-100-PowderMBCN07182019

Rev.3.ApprovedJD2020.11.05 | PC156031a