



Maiden Biosciences, Inc.

Collagen for Advanced Wound Care

MAIDEN BIOSCIENCES TOPICAL SHEET COLLAGEN WOUND DRESSINGS

PRODUCT DESCRIPTION:

Maiden Biosciences Topical Sheet Collagen Wound Dressings are an absorbent microfibrillar white/off white Type 1 bovine collagen topical dressing indicated for the management of moderately to heavily acute and chronic exuding wounds. The dressing is applied directly to the wound to absorb excess exudate that may impair tissue regeneration while maintaining a physiologically favorable moist wound environment. The product is supplied sterile, in single-use individually wrapped collagen sheets.

INDICATIONS FOR USE:

Maiden Bioscience Topical Sheet Collagen Wound Dressings are indicated for the management of moderately to heavily exuding wounds and to control minor bleeding.

Maiden Biosciences Topical Sheet Collagen Wound Dressings may be used for the management of exuding wounds such as: pressure ulcers, venous stasis ulcers, diabetic ulcers, acute wounds, e.g., trauma and surgical wounds, and partial-thickness burns.

DIRECTIONS: (Please use under the guidance of a healthcare professional)

PREPARE WOUND SITE AND SURROUNDING SKIN:

1. Cleanse and debride wound according to wound requirements and facility infection control protocol. Deep wounds should be well irrigated.
2. Skin around wound should be clean and dry.

TO APPLY:

1. Apply medication to the wound, if indicated.
2. Apply the collagen dressing to the entire wound surface.
3. Cover with secondary dressing.



Maiden Biosciences, Inc.

Collagen for Advanced Wound Care

CHANGING THE DRESSING:

- Dressing may be left in place for up to 5-7 days.
- Change the dressing if drainage seeps through the back of the dressing, or if the edge of the dressing becomes loose; or if the condition of the wound or surrounding skin causes concern; or as directed by a qualified healthcare professional.
- Dressing may be moistened to ease removal.
- Carefully remove adhesive dressings by holding the edge of the dressing and slowly pulling it parallel to the skin.
- Dispose of in accordance with local infection control guidance / hazardous waste protocols.

WARNINGS:

- An increase in drainage may be seen in the first several days of treatment.
- Discard all open and unused portions of the product.
- Some reactions such as transitory pain, bleeding, blistering, swelling and redness have been reported in isolated cases using a similar product. If these conditions continue for more than 3-5 days, discontinue use of the product and notify physician.

CONTRAINDICATIONS:

Do not use on patients with a known allergy or sensitivity to bovine Collagen.

PRECAUTIONS:

For topical use only. Not for ophthalmic use. Avoid eye contact.

R_x Only



See Instructions
For Use



Temperature
Limitation



Do Not Use
If Damaged



Keep
Dry



Do Not
Reuse

STERILE R

Sterilized Using
Irradiation



Not Made With
Natural Rubber Latex

Maiden Biosciences, Inc.

101 Lakeforest Blvd, Suite 390
Gaithersburg, MD 20877 USA

www.maidenbio.com

240.306.7272

MB Control No. Maiden-IFU-CollagenSheetMBCN07182019

Rev.2.ApprovedJD2020.11.05 | PC156032a